



Better Living Care

Operation and Maintenance Manual for Owners

Full Electric Beds

Model: 500 series

Distributor: This manual must be provided to the user of the bed.

User: Read this manual and keep for future reference, before using the bed.



TABLE OF CONTENTS

| | |
|----------------------------|-----|
| Safety Information..... | 1 |
| Assembly..... | 2 |
| Operating..... | 3-5 |
| Specifications..... | 6 |
| Cleaning Instructions..... | 7 |
| Warranty Card..... | 8 |





Safety Information:

- Check all parts for shipping damage and test all the function before using.
- Before moving the bed, the plug **MUST** be removed from the power point. Always keep the cable clear from the floor and castors during transportation.
- **DO NOT** use the bed for transporting a patient. Otherwise, injury or damage may occur.
- When the bed is used with children under 12 years, psychiatric patients and physically challenged individuals, close supervision is required.
- The bed is for nursing use only.
- **DO NOT** allow any individual under the bed at ANY time.
- Extra care should be taken to ensure spills are avoided, when using liquids in or around the bed. If a liquid is spilt on the bed, **UNPLUG** the bed immediately. Wipe the spill with a dry towel. **DO NOT** use the bed until the bed is dried thoroughly.
- The castors **MUST** be all locked before putting a patient onto the bed.
- Ensure all moving parts, including the mattress platform, main frame and moving actuators are free of obstruction (i.e. bed sheets, blankets, drainage bags/tubing, and other products with electric cords which can get tangled with the frames, safety rails, limbs and etc.) during operation.
- In order to reduce the risk of electric shock, **DO NOT** use the bed outdoors or on wet surfaces. It is not suitable for use in the presence of a flammable reagent.

SAFE WORKING LOAD: 160Kg

Operating

1. HANDSET

| | |
|--|---|
|  | Backrest control buttons lift and lower the backrest. |
|  | Knee Break control buttons lift and lower the knee break. |
|  | Contour function buttons lift and lower the backrest and knee break simultaneously. |
|  | Whole body control buttons lift and lower the bed platform. |


O
P
E
R
A
T
I
N
G

Operating (continued)

2. Castors

Individually Lockable Castors

Following illustrations show a castor at locked and unlocked position:

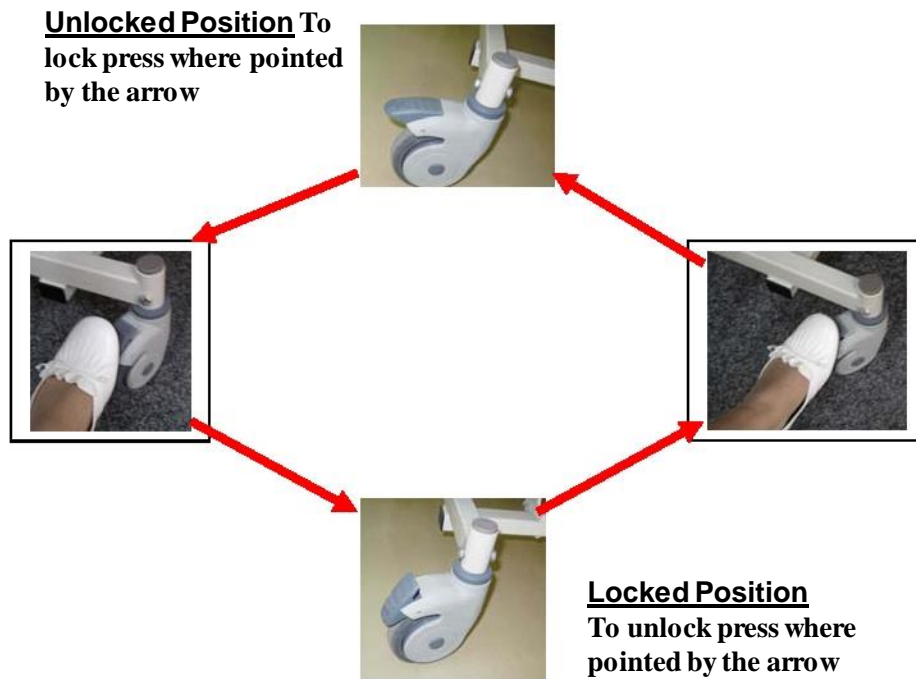

O
P
E
R
A
T
I
N
G


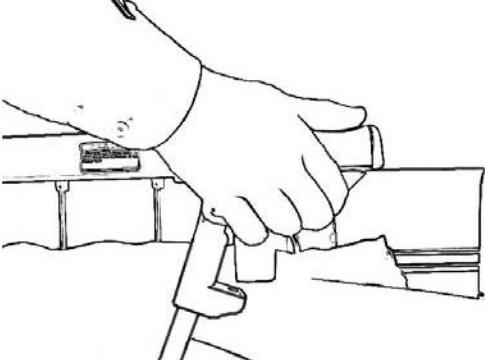
Central Locking Systems

To lock the castors, push the central locking bar to the lowest. To unlock, pull the bar upwards.



Operating(continued)

3. Safety Rails



Safety Rail Lock Release

WARNING
There is a risk of entrapment on installation and operation of the safety rails
REV. 02/06 LK B114

To fold up the rail, press the rail release button and collapse the rails.

To unfold the safety rails, lift the horizontal bar as shown in the picture. The rails are locked at vertical position when a "click" sound is heard.

O
P
E
R
A
T
I
N
G

SPECIFICATIONS

Specifications:

Product code: **B02-500**

OPERATION: ELECTRIC

FUNCTIONS:

Height Range: 330-680mm

Knee Brake: Yes

Central Locking: Yes

Back Rest: Yes

Auto-PCR No

MEASUREMENT(mm):

Mattress Base: 950X2010

Overall without Wall

Buffers: 950x2040

ELECTRIC PART

BRAND: **Moteck**

CONTROL BOX

ACUTUATORS

HANDSET

S
P
E
C
I
F
I
C
A
T
I
O
N
S

Cleaning

REMOVE MAIN POWER PLUG.

HAND WASH WITH CLOTH OR BRUSH USING CLEANSING AGENT AND DISINFECTANTS. DRY AFTER CLEANING. ALL PLUGS MUST BE FITTED DURING CLEANING.

WARRANTY CARD

Full Electric Bed

Model: 500 series

Please retain this Warranty Card and Invoice as proof of purchase. These documents must be presented in the event of making a claim under the following terms. A copy of completed Warranty Card needs to be presented to the distributor within 3 month from receipt of the product.

PURCHSER'S NAME

ADDRESS

SUBURB

STATE

POST CODE

PRODUCT NAME:

ELECTRIC BED

PRODUCT CODE

SERIAL NO

DATE OF INVOICE

INVOICE NO

DISTRIBUTOR:

AUTHORIZED SERVICE CENTER:

TERMS OF WARRANTY

1 YEARS WARRANTY ON MOTOR, CONTROL BOX, AND ACTUATORS

5 YEARS WARRANTY ON STRUCTURE

6 MONTHS WARRANTY ON HANDSET

Should a defect in this due to either **FAULTY MATERIAL** or **WORKMANSHIP** become apparent within the above period from the date of delivery of the product, such defect will be rectified by the authorized service centre, without cost to you for either labor or material, at the premise of the distributor, subject to:

1. The Warranty being rendered invalid if the product is misused or if any unauthorized alteration, modification, or substitution of any part of the product be made or the serial number is lost or altered.
2. Cost of transportation both ways to and from the authorized service centre being met by the owner if returning the product or any part to the authorized service centre is necessary.
3. The warranty only applying to the servicing within Australia of products purchased in Australia.
4. All (a) damage resulting from incorrect installation or use other than in accordance with the operating instructions issued by the manufacturer (b) consequential damage, being excluded from this Warranty.
5. A charge being accepted by the owner for traveling expenses incurred in connection with warranty services at the user's premises.

IMPORTANT NOTICE

1. Warranty starts from the date of delivery.
2. If the product or any parts returned to the authorized service centre for any cause not covered by this warranty, all costs involved, must be paid to the authorized service centre.
3. Changes not covered by the term of this Warranty must be borne by the owner. Such charges may arise through the cost of any service call where (a) the product is found to be in incorrect installation, maladjustment of user controls, or failure to follow operating instructions.