
*No. 1 choice  
of hospitals  
& mothers*

# Symphony<sup>®</sup>

EN Instructions for use – home use | DE Gebrauchsanweisung – für den Heimgebrauch  
FR Mode d'emploi – Usage à domicile | IT Istruzioni per l'uso – Uso domestico  
NL Gebruiksaanwijzing – voor thuisgebruik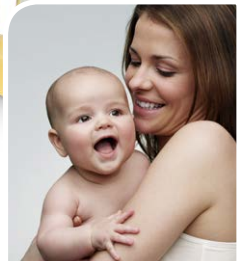

Thank you for choosing the Symphony Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers' needs. For mothers who insist on the best for their babies and themselves. Because you care.

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NL  
124–153

# Table of Contents

<b>1. Intended use/Population – Contraindication</b>	<b>5</b>
<b>2. Meaning of symbols</b>	<b>6</b>
<b>3. Important safety information</b>	<b>8</b>
<b>4. Product description</b>	<b>10</b>
<b>5. Cleaning</b>	<b>12</b>
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning the breastpump and case	15
5.4 Disinfecting the breastpump	15
<b>6. Preparing for pumping</b>	<b>16</b>
6.1 Card and Cord Protector	16
6.2 Mains operation	16
6.3 Changing the program card	17
6.4 Assembling the pump set	18
<b>7. Expression</b>	<b>22</b>
7.1 Preparing for single pumping	22
7.2 Expression	22
7.3 Preparing for hands-free expression with the Easy Expression Bustier	24
7.4 Preparing for double pumping	26
<b>8. Storing and thawing breastmilk</b>	<b>28</b>
8.1 Storage	28
8.2 Freezing	28
8.3 Thawing	28
<b>9. Feeding breastmilk</b>	<b>29</b>
<b>10. Troubleshooting</b>	<b>30</b>
<b>11. Warranty and maintenance/disposal</b>	<b>31</b>
<b>12. Complete care during breastfeeding time</b>	<b>32</b>
<b>13. Technical specifications</b>	<b>154</b>

# 1. Intended use/Population – Contraindication

5

## **The intended use and indications of breastpumps**

Breastpumps are intended to be used by lactating women in the hospital or home environment for expressing breastmilk. Breastpumps are indicated to alleviate the symptoms of engorgement by expressing milk. In case of mastitis breastpumps support the healing process by removing breastmilk from the affected breast. Breastpumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also, breastpumps enable mothers to provide breastmilk for babies who cannot feed directly at the breast for a variety of reasons such as latch-on problems, cleft palate or premature babies.

## **Intended patient population/user**

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breastpump to express and store their breastmilk once they have returned to work, are travelling or are otherwise separated from their baby. Women are often delighted to learn that a breastpump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the suckling of a nursing baby.

## **Contraindications**

There are no known contraindications for the Symphony breastpump.

## 2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury or damage to the breastpump. When used in conjunction with the following words, the warning symbol stands for:

### Warning

Can lead to serious injury or death.

### Caution

Can lead to minor injury.

### Note

Can lead to material damage.

### Info

Useful or important information that is not related to safety.

### Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN).

## Symbols on the device

This symbol indicates to follow instructions for use.

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the manufacturer.

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates a type BF applied part.

This symbol indicates manufacturer's serial number of the device.

This symbol indicates manufacturer's catalogue number of the device.

IP21

This symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates that the Symphony is a class II device.

This symbol indicates fuse (to identify fuse boxes or their location).

This symbol indicates that the fuse box is safety tested.

This symbol indicates open the cover of the mains socket.

## 3. Important safety information

8

Failure to follow these instructions/safety information can lead to danger from the device. Subject to technical changes.

### Device

Danger of electrocution! Keep the device dry! Never immerse in water or other liquids!

The Symphony breastpump is not heat-resistant: keep away from radiators and open flames.

Do not expose the motor unit to direct sunlight.

There are no user-serviceable parts inside the Symphony. Repairs must be performed only by an authorised service agency. Do not repair yourself! No modifications to the device are permitted.

Never use a damaged device. Replace damaged or worn parts.

The operating life of the Symphony breastpump is 3000 hours. The service life is seven years if used normally.

### Electrical use

Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.

Keep the power cord away from heated surfaces.

Do not reach for any electrical device if it has fallen into water. Unplug immediately.

The Symphony breastpump should never be left unattended when plugged into a power source.

## Use

Use the Symphony breastpump only for its intended use as described in this manual.

Never operate an electrical device if it has a damaged power cord, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use the Symphony breastpump while bathing or showering.

Do not use the Symphony breastpump while sleeping or overly drowsy.

Do not drive whilst hands-free pumping.

Supervision is necessary when the Symphony breastpump is used in the vicinity of children.

Contact your lactation consultant or physician if you experience problems or pain.

Radio frequency communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations and walkie-talkies can affect the Symphony breastpump. Due to electromagnetic compatibility (EMC) a distance of at least 1 meter between the Symphony breastpump and the wireless equipment is recommended.

## Important:

- I Plastic bottles and component parts become brittle when frozen and may break when dropped.
- I Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- I Take appropriate care in handling bottles and components.
- I Do not use the breastmilk if bottles or components become damaged.

## 4. Product description

### 10 **Symphony single or double pump set (Applied parts)\***

The Symphony single or double pump set is an accessory for the Symphony breastpump. The pump set must only be used by one mother; use by more than one mother could pose a health risk.

The double pump set contains two pieces of each part.

### The pump set contains:

The PersonalFit breastshield, size M (24mm). You can find more information on the correct choice of breastshield at [www.medela.com/personalfit](http://www.medela.com/personalfit) or scan the QR code on page 32.

**1x PersonalFit™ Breastshield M (24 mm)**  
(for other breastshield sizes see Chapter 12, page 32)  
810.7084

**1x valve head**  
810.0475

**2x valve membrane**  
820.7013

**1x membrane cap**  
810.0667

**1x protective membrane**  
820.0094

**1x lid with opening**  
810.7062

**1x disc**  
810.0473

**1x PersonalFit™ Connector**  
810.7088

**1x milk bottle (150 ml, printed)**  
811.0021

**1x silicon tubing (105 cm)**  
800.0828

**1x protective cap**  
810.0474

## Rental pump required: Symphony

**Bottle holder**  
810.0552

**Card & Cord Protector**  
810.0589

**Power cord**  
928.0040 Euro  
928.0041 AUS/NZ  
928.0042 UK

**Socket for car connection cable**

**Case**  
600.7073

**Program card, Standard 2.0**  
800.0540 DE  
800.0541 EN  
800.0542 FR  
800.0543 IT  
800.0544 NL

### Ideal accessories\*

**Calma** – the unique breastmilk feeding solution, the ideal way to feed your child expressed breastmilk. Calma is the result of extensive research and development.

**Easy Expression Bustier** makes hands-free pumping easier, especially with simultaneous expression from both breasts. It is available in 4 different sizes (S, M, L, XL).

**More information on all Medela products can be found at [www.medela.com](http://www.medela.com)**

\* Accessories available for order can be found on page 32. Articles may not be available in all countries. For further information about Medela products, visit [www.medela.com](http://www.medela.com) and look for location finders in your country. Health Canada device identifiers: Symphony: 0240108 and 0240208. Symphony Kits: 008.0292, 27098S and 27292

## 5. Cleaning

12

### Caution

- | Only use drinking-quality water for cleaning.
- | Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

### Note

- | The pump set requires no maintenance.
- | Take care not to damage parts of the pump set during cleaning.
- | If the individual parts of the pump set are cleaned in a dishwasher, parts may be discoloured by food pigments. This has no impact on the function.
- | Inspect the tubing after each pumping session for condensation and/or milk. Follow the instructions on page 30 if there is condensation or milk in the tubing.

\* Medela recommends to clean all parts of the pump set (incl. tubing, etc.) before used for the first time as shown in Chapter 5.2.


### 5.1 Before using for the first time and after every use

1

Disassemble the pump set into individual parts (see diagram). The tubing, protective membrane and cap do not normally need to be washed.\*

4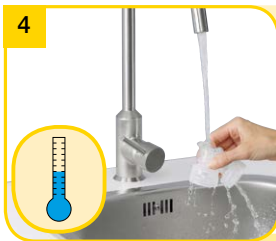

Rinse all parts with cold, clear water (approx. 20 °C) for 10–15 seconds.

2

Rinse all parts with cold, clear water (approx. 20 °C).

3

Clean all parts with plenty of warm water and basic dishwasher agent (approx. 30 °C).

5

Dry with a clean cloth or leave to dry on a clean cloth.

or

6

If you clean the individual parts of the pump set in the dishwasher, place them on the top rack or in the cutlery section.

As an alternative to steps 2, 3 and 4.

## 5. Cleaning

14

### Caution

- | Only use drinking-quality water for cleaning.
- | Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.
- | **for chapter 5.3 Cleaning the breastpump and case:**  
Unplug the breastpump before cleaning.

### Note

- | **for step 2a (Chapter 5.2):**  
When boiling, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale.
  - | Store the pump set in a clean bag/ container until next use. Or store it in clean paper or a cloth.
  - | Do not spray or pour liquid directly onto the pump.
- \* Medela recommends to clean all parts of the pump set (incl. tubing, etc.) before used for the first time as shown in Chapter 5.2.

### 5.2 Before using for the first time and once a day

1

More information  
on page 12, chapter  
5.1, steps 2–4.

Disassemble the pump set into individual parts.  
The tubing, protective membrane and cap do not normally need to be washed.\*

3

Dry with a clean cloth or leave to dry on a clean cloth.

2a

Cover all parts with water and boil for five minutes.

or

2b

Use QuickClean bags\* in the microwave in accordance with the instructions.

### 5.3 Cleaning the breastpump and case

1

Wipe off with a clean, damp (not wet) cloth. Water and basic dishwasher agent or non-abrasive detergent may be used.

### 5.4 Disinfecting the breastpump

1

Wipe off with a clean, damp (not wet) cloth. Alcohol based disinfecting agent may be used.

\* Accessories available for order can be found on page 32.  
For more information please refer to [www.medela.com](http://www.medela.com)

## 6. Preparing for pumping

16

### Warning

#### I for chapter 6.2 Mains operation:

- Only use the power cord that comes with the Symphony.
- Make sure the voltage of the power cord is compatible with the power source.

### Info

- Only use the pump with the appropriate program card (Standard or Premie+\*).

#### Optional for pump with battery:

- Built-in battery must be charged for 12 hours on receipt of the device.
- The pump can be used during charging.
- If the pump is to remain unused for a long time, charge the battery every two months (for 12 hours each time).
- The optimum service life of the battery is achieved if it is not fully discharged on a regular basis or if the pump is charged while not in use.

Pumping time (fully-charged battery)	approx. 60 min.
Charging time (for full charge)	12 hours

### 6.1 Card and Cord Protector

1

To remove, unscrew the card and cord protector and slide it out of the guide rail.

### 6.2 Mains operation

1

Insert the power cord into the power socket on the back of the pump.

Insert the cord into the card and cord protector in a loop around the pin.  
 → Make sure there is enough play to plug the cord in.

Push the card and cord protector over the guide rail on the back until it stops and tightens the screw.

### 6.3 Changing the program card

Plug the power cord into the mains socket.

As soon as the card compartment is revealed, the card can be removed and replaced or changed.

## 6. Preparing for pumping

18

### Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, clean pump parts and clean pump set.

### Note

I Use Medela original accessories only.  
I Check pump set components for wear or damage before use.


### Info

I Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

### 6.4 Assembling the pump set

1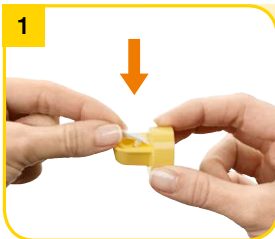

Carefully push the membrane onto the valve head until it engages.

4

Screw the bottle onto the connector.

2

Push the breastshield onto the connector.

3

Push the valve head and membrane onto the connector.  
→ Position valve head laterally.

5

Insert the tubing into the corresponding opening in the connector.

6

Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.

## 6. Preparing for pumping

20

### i Info

#### I for step 11:

Always close the lid when pumping.  
The lid keeps the membrane cap pushed down to prevent a loss of vacuum that could occur if the membrane cap came off.

7

Insert the tubing adapter into the opening in the membrane cap.

10

Centre the tubing connector in the recess in the pump housing.  
→ Make sure that the membrane cap engages and lies flush to the surface.

8

Open the lid of the breastpump.  
→ Press the oval button on the top of the Symphony and lift the handle.

9

Position the membrane cap firmly on the breastpump membrane to form a seal. Centre the tubing connector in the recess in the pump housing.  
→ Make sure that the membrane cap engages and lies flush to the surface.

11

Close the lid.

## 7. Expression

22

### Caution

**I before step 2 (chapter 7.1):** Wipe the breast with a warm cloth (do not use alcohol).

**I for step 2 (chapter 7.1):** Answer the following questions to find out whether the available **breastshield size** will fit:

- I Is the nipple moving freely in the tunnel?
- I Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- I Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- I Do you feel the breast emptying all over?
- I Is your nipple pain-free? If the answer to any of these questions is “No” or if you experience problems or pain during expression, consult your breastfeeding and lactation consultant or midwife.

**I for step 3 (chapter 7.1):** Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

### Info

**I** If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.

### 7.1 Preparing for single pumping

Mount only one membrane cap to the breastpump membrane.

### 7.2 Expression

Switch on the breastpump with  .  
→ Stimulation phase begins.  
The stimulation phase will automatically change to the expression phase after two minutes.

2

Place the breastshield on your breast so that your nipple is properly **centred** in the tunnel.

3

Hold the breastshield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

2

If your milk starts to flow before the end of the two minutes, press  to change to the expression phase.  
→ Make sure that the milk flows into the bottle properly.

3

Set vacuum comfort: Rotate the vacuum to the right and increase until there is slight discomfort. Rotate to the left to reduce the vacuum slightly.

## 7. Expression

24

### Caution

| Contact your midwife or breastfeeding and lactation consultant if you can express only minimal or no milk or if expression is painful.

### Note

| Always disconnect the Symphony from the power source after expression.

### Info

| Do not kink the tubing during expression.

| Press the “Let down” button  to revert from the expression phase to the stimulation phase if necessary. If milk has not begun to flow after two successive stimulation phases, take a break and try again in 15–30 minutes. Try breast massage (ask your lactation consultant or health care practitioners) and then pump using the Expression Phase.

| The program stores the last vacuum setting in the stimulation phase.

4

Switch off the breastpump with

→ Do not fill the bottle above max. volume indicator line.

### 7.3 Preparing for hands-free expression with the Easy Expression Bustier\*

1

Put on the Easy Expression Bustier and leave the zip open slightly. Take your assembled pump sets and take off the breastshields.

5

Use the stand or bottle holder to prevent the bottle from tipping over.

6

Clean in accordance with chapter 5, page 12.

Close the bottle with a lid and disc.

→ Follow the instructions in chapter 8 „Storing and thawing breastmilk“.

2

Place the breastshields on your breasts under the Bustier such that your nipples are **centred** in the tunnel of the breastshields.

3

More information on page 22, chapter 7.2.

Close the zip and attach the assembled pump sets to the breastshields.

## 7. Expression

26

### Caution

#### I for step 2:

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

### Info

#### I for step 2:

Please read the notes on breastshield sizes on page 22 (information column).

I Do not kink the tubing during expression as the membrane cap could come off.

I Double pumping saves time and increases the energy content of the milk. Milk production is increased and maintained long-term.

I If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.

### 7.4 Preparing for double pumping

1

Place the breastshield on your breast so that your nipple is properly **centred** in the tunnel.

4

More information on page 23, chapter 7.2, step 2 and 3

Place the second breastshield on the breast so that the nipple is properly **centred** in the tunnel.

2

Hold the breastshield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

3

Switch on the breastpump with .  
→ Stimulation phase begins.  
The stimulation phase will automatically change to the expression phase after 2 minutes.

## 8. Storing and thawing breastmilk

28

### 8.1 Storage

#### Storage guidelines for freshly expressed breastmilk

(For healthy term babies)

Room temperature	Refrigerator	Freezer	Thawed breastmilk
4–6 hours at 19 to 26 °C (66 to 78 °F)	3–8 days at 4 °C (39 °F) or lower	6–12 months –18 to –20 °C (0 to 4 °F)	In refrigerator for 10h max. Do not refreeze!

I Do not store breastmilk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

### 8.2 Freezing

I You can freeze expressed breastmilk in milk bottles or “Pump & Save”<sup>®</sup> bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.

I Label the bottles or “Pump & Save” bags with the date of expression.

### 8.3 Thawing

#### Caution

Do not thaw frozen breastmilk or do not warm breastmilk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

I In order to preserve the components of the breastmilk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C).

I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

## 9. Feeding breastmilk

### i Info

Medela recommends Calma for feeding breastmilk.

#### With Calma

Your baby can feed, pause and breathe regularly.

Milk will flow when a vacuum is created.

Your baby's natural feeding behaviour will be maintained supporting easy transition back to the breast.

29

Calma – unique feeding solution for breastmilk.

You can find detailed information about Calma on the Medela website ([www.medela.com](http://www.medela.com)).

As a result of its comprehensive research work, Medela is one of the first companies in the world to incorporate babies' breastfeeding behaviour into its 2-Phase Expression and Calma innovations. 2-Phase Expression mimics the baby's natural sucking rhythm to express more milk in less time. Short, stimulating patterns are followed by longer ones, providing optimal milk flow. The unique Calma breastmilk feeding device allows the baby to pause and breathe during feeding whilst maintaining the suckling behaviour learned at the breast. Developed for a great breastfeeding experience.

## 10. Troubleshooting

30

Problem	Solution
<b>If the motor is not running</b>	<p>Check whether mains power/battery is available.</p> <p>Check that the device is switched on.</p> <p>Check whether program card is inserted correctly in the card compartment on the back of the breastpump.</p>
<b>If there is low or no suction</b>	<p>Assemble the pump set in accordance with the instructions in chapter 6.4.</p> <p>Check whether all connections of the pump set and breastpump are secure.</p> <p>Express in accordance with chapter 7. Make sure that the breastshield forms a complete seal around the breast.</p> <p>The white valve membrane must lie flat against the yellow valve head.</p> <p>Check that the valve head and membrane are clean and not damaged. See chapter 6.4 "Assembling the pump set".</p> <p>! The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap.</p> <p>! The protective membrane and membrane cap should be correctly assembled and the membrane cap is firmly fitted to the breastpump membrane to form a seal.</p> <p>! The protective membrane must be intact.</p> <p>! Do not kink or trap the tubing during expression.</p> <p>! All components must be completely dry.</p>
<b>If there is an overflow</b>	<ol style="list-style-type: none"><li>1. Place the pump set into the bottle holder.</li><li>2. Turn off the pump and unplug the power cord from the socket.</li><li>3. Use a damp (not wet) cloth to wipe the pump and membrane area.</li><li>4. <b>If there is milk in the tubing:</b> Remove the pump set with tubing, protective membrane and membrane cap and clean in accordance with chapter 5.2 "Before first use and once a day". Shake out any drops of water from the tubing and hang up the tubing to dry or let the pump run with tubing attached for 1–2 minutes or until dry. <b>If there is condensation in the tubing:</b> Remove the pump set with tubing and the membrane cap. Shake water droplets out of the tubing and hang it up to dry or let the pump run with tubing attached for 1–2 minutes or until dry.</li><li>5. When components are completely dry, reassemble breastpump and accessories set.</li></ol>

# 11. Warranty and maintenance/disposal

## Warranty and maintenance

3-year warranty for the unit from the date of purchase, excluding pump set and accessories. 6 months warranty for the batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorised persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorised personnel.

The engineering of the Medela Symphony Breastpump has been verified by independent test institutes for compliance with the EN/IEC 60601-1 standard. Copies of such certificates may be obtained on request.

Because of the method of construction used, Medela does not expect electrical safety to be affected at any time during the life of the product – provided that it is used properly in accordance with the intended use. There are no prescribed or recommended safety checks to be carried out.

## Disposal

The unit is made of various metals and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health. The accessories (pump set/bottles) are made of plastic and may only be disposed of in accordance with local regulations.

## 12. Complete care during breastfeeding time

32

The following accessories for the Symphony are available from Medela distribution partners by specifying the product name or number.

Article number	Product
008.0327*	PersonalFit Breastshield S (21 mm): DE/FR/IT
008.0337*	PersonalFit Breastshield S (21 mm): EN/NL/FR
008.0328*	PersonalFit Breastshield M (24 mm): DE/FR/IT
008.0338*	PersonalFit Breastshield M (24 mm): EN/NL/FR
008.0329*	PersonalFit Breastshield L (27 mm): DE/FR/IT
008.0339*	PersonalFit Breastshield L (27 mm): EN/NL/FR
008.0330*	PersonalFit Breastshield XL (30 mm): DE/FR/IT
008.0340*	PersonalFit Breastshield XL (30 mm): EN/NL/FR
008.0331*	PersonalFit Breastshield XXL (36 mm): DE/FR/IT
008.0341*	PersonalFit Breastshield XXL (36 mm): EN/NL/FR
008.0139	Calma Solitaire
008.0124	Calma with 150 ml milk bottle
008.0188	Calma with 250 ml milk bottle
800.0796	150 ml milk bottle (pack of 3): EN/DE/FR/IT/NL
200.1659	250 ml milk bottle (pack of 2): EN/DE/FR/IT/NL
008.0315	Pump & Save breastmilk bags EN/DE/FR/IT/NL
008.0217	Easy Expression Bustier white S
008.0220	Easy Expression Bustier white M
008.0224	Easy Expression Bustier white L
008.0225	Easy Expression Bustier white XL
008.0040	QuickClean bag

Articles may not be available in all countries. Information about all Medela products is available on our web site **[www.medela.com](http://www.medela.com)**.

\* If the result of expression is unsatisfactory or expression is painful, contact your lactation consultant or health care practitioner.

A different size of PersonalFit Breastshield may help make expression more comfortable and successful.

Please note the information on breastshield sizes on page 22 (information column), visit [www.medela.com/personalfit](http://www.medela.com/personalfit) or scan the QR code.

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Medela UK Ltd.
Medela Inc.
Medela Canada Inc.