



# ILS Seat Walker Holder Assembly Guide

1



Place L shaped bar in rear receiver of scooter and place securing rod or screw bolt in position.

2



Place the small T shaped bar through the L shape bar

3



Slide the small T shaped bar to the very bottom of the L shaped bar and bolt into position

4



Place small extension bars at both ends of the T shaped bar and fasten with bolts

5



Install Long extension tube onto the top of the L shaped bar and fix into position with bolt and metal pin

6



Place dual adapter at the top of the L shape bar with the extension piece situated on the left



7



Secure with bolt and metal pin

8



Place the large T shaped bar on the left-hand extension tube

9



Bolt this T shaped tube into position

10



Bolt the right-side hanging clip into position facing outwards at the end of the T piece bar

11



Bolt the left side hanging clip on the other end facing outwards at the end of the T piece bar

12



See fully assembled walker holder

13



Lift seat and place seat walker on the holder via the backrest (Models may vary)

14



Secure tightly to the holder with Velcro provided