

# SmartDrive MX2 + PushTracker

## Quickstart Guide


### IMPORTANT!

THE DEFAULT CONTROL MODE OF THE SMARTDRIVE MX2+ HAS CHANGED. IF THIS MODE IS NOT A PROPER FIT, ANOTHER MODE CAN BE SELECTED.

PLEASE READ THE USER'S MANUAL FOR BOTH THE SMARTDRIVE AND PUSHTRACKER BEFORE OPERATING EITHER DEVICE.






ALL WARNINGS AND SAFETY PRECAUTIONS MUST BE HEEDED.

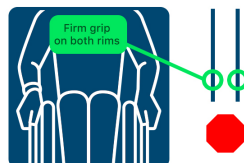

With your SmartDrive powered on, press the left button of the PushTracker to turn power assist ON. You will get a blue light and a vibration.

In MX2+ mode, a DOUBLE tap will START UP the motor and ramp up its speed, and a SINGLE tap will coast at the current speed. Pushes do not power.

TO STOP, perform a DOUBLE tap to deactivate the motor and then grip both handrims to slow down the wheelchair. Other modes will be different.

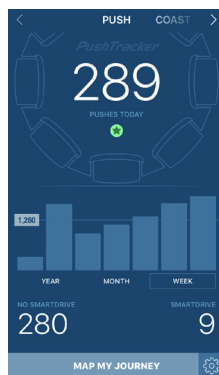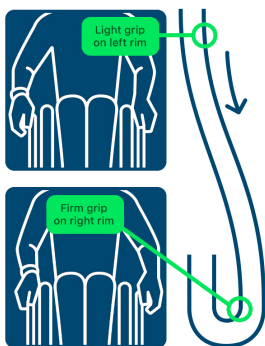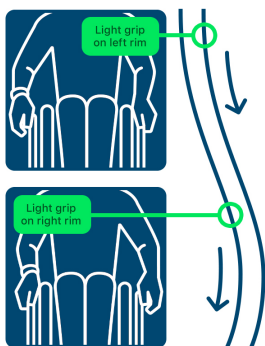

To steer while the motor is driving, lightly grip either handrim to turn in the direction you would like to go. Same as you would without SmartDrive.

To turn while the motor is driving, grip either handrim firmly to make a sharp turn or to turn around completely. Same as you would without SmartDrive.

Be sure to download the PushTracker App from the Apple or Google Play Store on your smartphone.

[www.max-mobility.com](http://www.max-mobility.com)  
[www.pushtracker.com](http://www.pushtracker.com)

