



# SOLAX TRAVEL CASE

## CHANGING PERSONAL CODES

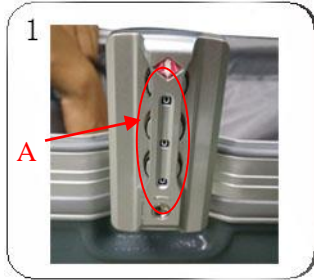

Default code is “000”  
(A)

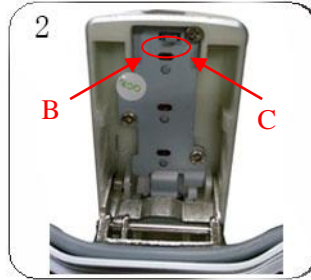


Pull the red switch to “SET” (B) position. Adjust the 3 codes to new settings then flick back to OFF” (C) position. Take note of your new code and keep it safe.



Keyhole (D) is for customs only as no key is supplied


## PACKING CASE

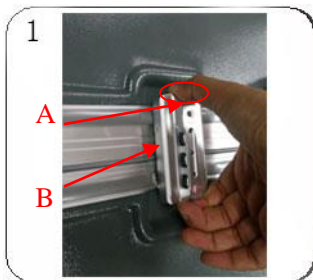


To open press button (A), (near keyhole) then pull bottom of the latch (B)



Place folded scooter (C) horizontally into the case

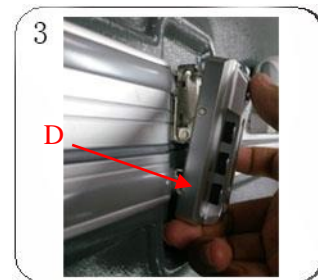

Hook the latch at point (D) first

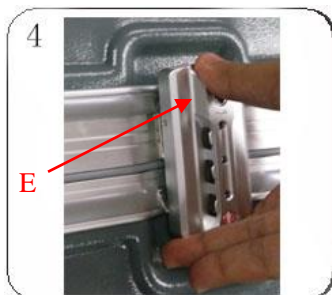

After latch is hooked under clip press top of latch (E) to lock into place



Check the latches are secure and make sure you have your code in a secure place