

Mid-wheel Powerchair USER MANUAL

Models:

COUGAR 10 COUGAR 14





Table of Contents

3	INTRODUCTION
4	EMI WARNING
S 4	ELECTROMAGNETIC INTERFERENCE FROM RADIO WAVES.
MI) 5	POWERED CHAIR ELECTROMAGNETIC INTERFERENCE (EMI
6-7	SAFETY INSTRUCTIONS
8-9	FEATURE GUIDE
10-11	GETTING TO KNOW YOUR SHOPRIDER
12	THE JOYSTICK CONTROLLER (VSI)
	LOCKING / UNLOCKING THE JOY STICK CONTROLLER
14	DRIVE/FREEWHEEL MECHANISM
15	DRIVING YOUR POWERCHAIR
15-17	BATTERIES AND BATTERY CHARGING
18	CARING FOR YOUR POWERCHAIR
	TROUBLESHOOTING
20	SPECIFICATIONS
2 ²	DISCLAIMER
22	WARRANTY TERMS & CONDITIONS
23	NOTES

INTRODUCTION

Congratulations on insisting on the best. Shoprider Powerchairs are the ultimate combination of style and comfort. The COUGAR features support easy maneuverability and smooth control for today's active lifestyle.

The COUGAR powerchair provides the user with outstanding flexability and reliability Combine this with the programmable joystick controller, fully adjustable footplate, armrests,flat-free tyres,deluxe reclining captain seat and easy access batteries.

For a complete powerchair package we know that your Cougar Powerchair will provide many years of mobility and freedom.

This manual contains important information regarding the safe use of your Shoprider Powerchair. Please read this manual carefully and thoroughly before using your Powerchair and make sure you understand all the instructions.





EMI WARNING

Electromagnetic Interference (EMI) from Radio Wave Sources

Powered electric chairs may be susceptible to electromagnetic interference, which is a kind of interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios and cellular phones. The interference (from radio wave sources) can cause the power chairs to release its brakes, move by itself or move in unintended directions. It can also permanently damage the powered chair's control system.

The sources of radiated EMI can be broadly classified into three types:

1. Hand-held portable transceivers (transmitters-receivers) with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkies", security, fire and police transceivers, cellular telephones and other personal communication devices

NOTE: Some cellular telephones and similar devices transmit signals while they are ON, even when not being used.

- 2. Medium-Range mobile transceivers, such as those used in police cars, fire trucks, ambulances and taxis. These usually have the antenna mounted on the outside of the vehicle.
- 3. Long-Range transmitters and transceivers, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.

NOTE: Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, cassette players and small appliances such as electric shavers and hair dryers, so far, as we know, are not likely to cause problems to your Powerchair.



Powered Chair Electromagnetic Interference (EMI)

Because EM energy rapidly becomes more intense the closer one moves to the transmitting antenna (source), the EM fields from handheld radio sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the Powerchair's control system while using these devices. This can affect the Powerchair's movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of the Powerchair.

WARNINGS

The following warnings listed below should reduce the chance of unintended brake release or Powerchair movement, which could result in serious injury.

- 1. Do not operate hand-held transceivers (transmitter-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the Powerchair is turned ON.
- 2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
- 3. If unintended movement or brake release occurs, turn the Powerchair OFF as soon as it is safe.
- 4. Be aware that adding accessories or components, or modifying the Powerchair may make it more susceptible to EMI.

NOTE: There is no easy way to evaluate the overall immunity of the powered chair.

Report all incidents of unintended movement or brake to your Powerchair provider, and note whether there are sources of EMI nearby.

NOTE: The VSI controller EMI rating are Regulated and conforms to federal regulations.



Safety Instructions

Please use your Powerchair to enhance your active lifestyle. With increased mobility, please observe a few rules to ensure safe operation of your Powerchair. So please...

- **1. Do not** drive the Powerchair without reading this manual.
- 2. Do not exceed the safe climbing maximum angle (Fig. 6-7).
- 3. **Do not** use the joystick in an erratic manner when going up or down an incline.
- **4. Do not** carry passengers or exceed the maximum weight limit (Table 2).
- **5. Do not** turn off the joystick controller by switching the On/Off Button when moving at any speed. This will bring the electromagnetic brakes on immediately and could cause damage to the joystick controller.
- **6. Do not** drive over deep and soft terrain (soft dirt, loose gravel, deep grass).
- **7. Do not** attempt to mount a curb height above 2 inches.
- 8. **Do not** mount or dismount the Powerchair unless the electromagnetic brakes are engaged and the joystick controller is off.
- **9. Do not** operate the Powerchair if the unit is in freewheel mode.
- **10. Do not** use on the road, except when crossing between sidewalks.
- **11. Do not** sit on the Powerchair when in a vehicle, but transfer to a vehicle seat.
- **12. Do not** exceed any grade over 6 degrees (10%).
- **13. Always** stop fully before changing forward or reverse direction.
- **14. Always** engage a slow speed when going down gradients (move the joystick slowly towards center position to reduce the speed).
- **15. Always** approach and climb over curbs at slow speed.
- **16. Always** approach curbs and gradients at 90 degrees.
- **17.** Always use the safety belt.



- **18. Always** keep the feet on the leg rest while driving.
- **19. Always** make sure the batteries are fully charged before setting out on a journey.
- **20. Always** charge the Powerchair in a well ventilated area to prevent any possible risk.
- **21. Always** check that the drive wheels are engaged. Someone may have left the Powerchair in freewheel mode.
- **22. Always** reduce your speed when turning sharply.
- 23. Avoid turning sharply on a slope or gradient.
- **24.** To brake in an emergency simply release the joystick.
- **25. Always** try to find a "dropped curb" or "curb cutout" whenever possible.
- **26. Always** keep your Powerchair properly maintained.
- **27. Never** try to use your Powerchair beyond it's limitations as described in this manual.
- **28. Always** put seat in most upright position whenever driving up an incline.

Important Notes

The Shoprider Powerchair is designed to assist in your individual mobility needs. Any usage outside of the guidelines in this manual may result in damage to the chair or injury to the user or third party.

Please do not lift the wheelchair by the armrests.

First Time Use

Prior to using your Cougar Powerchair for the first time, you must charge the batteries fully (refer to Battery Charging Procedures). Charging may take up to 12 hours. Charging the batteries completely prior to first time use will benefit battery efficiency and life of batteries.




Feature Guide Cougar 14



- (1) Removable Armrests with Width, Height & Angle Adjustment (Flips Up for Easy Sideways Transfer)
- (2) Programmable VSI Joystick Controller with Adjustable speed & Battery Life Indicator.
- (3) Charging Port
- (4) Deluxe Reclining Captain Seat
- (5) Adjustable Length, Height, Angle Foot Plate
- (6) Front Castor Assy (flat free tire)
- (7) Frame Serial Number (13 alphanumeric characters)
- (8) Rear Castor Assembly (flat free tire)
- (9) Transport Tie Down Points



Feature Guide Cougar 10

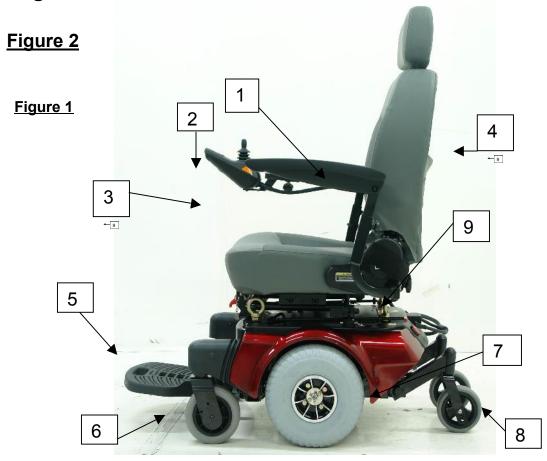

- (1) Removable Armrests with Width, Height & Angle Adjustment (Flips Up for Easy Sideways Transfer)
- (2) VSI Joystick Controller with Adjustable speed & Battery Life Indicator
- (3) Charging Port
- (4) Deluxe Reclining Captain Seat
- (5) Standard Angle Adjustable Foot Plate
- (6) Front Castor Assy (flat free tire)
- (7) Frame Serial Number (13 alphanumeric characters)
- (8) Rear Castor Assembly (flat free tire)
- (9) Transport Tie Down Points



Getting to Know Your Shoprider Powerchair The Seat and Arm Assemblies

The seat is built for comfort, style, and stability. Your seat comes with a reclining back for comfort. The flip-up arm-rests are especially built for easy side access for getting in and out of your Cougar , with the greatest of ease. The arm-rests have a built in height adjustment knob, that adjusts the arm pad to a comfortable level for the driver. The seat is covered in a durable long lasting black/grey vinyl for comfort and style.

Seat recline lever



Flip up armrest

Armrest height adjustment lever

Adjustable footplate

The footplate adjusts to fit to the users size and comfort. It adjusts length and height wise to help in comfort and stability for the user.

Adjustment pins are easily accessible for the removal and adjustment of the length and height of the footplate.






Seat Height Adjustment And Easy Battery Access

- 1. Release levers that allows for the seat to be removed if needed, if not, there is a pivot point that allows for the seat to move up and away from the base.
- 2. Adjustment pins that can be used to adjust the height of the seat to fit the user.
- 3. Rear seat height adjustment can be completed by removing pins and making the height adjustment
- 4. Battery cover is easily popped off, to allow for battery access.
- 5. Batteries are sitting on the base



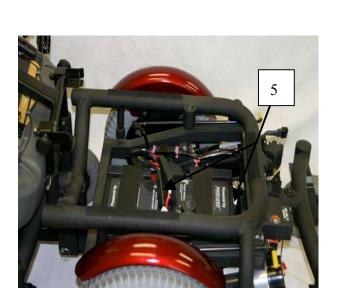




The Joystick Controller (VSI)

- 1. **On-Off Button**: This button turns the joystick controller (hereinafter referred to as VSI) on and off. Do not use this button to stop the power chair, except in an emergency.
- 2. **Battery Gauge**: This is a 10-segment display, which indicates if the VSI is switched on and gives the state of charge of the battery. Additionally, any faults in the Powerchair electrical system are also indicated by this display. Refer to Table 1 for more details.
- 3. **Maximum Speed Indicator**: This is a 5-segment display, which indicates the maximum speed setting selected.
- 4. **Horn Button**: This button operates the Powerchair's horn.
- 5. **Speed Decrease Button**: This button decreases the maximum speed.
- 6. **Speed Increase Button**: This Button Increases the maximum speed.
- 7. Joystick: This controls the speed and direction of the Powerchair. Push the joystick in the direction you want to go. The further you push it, the faster the speed will be. Releasing the joystick will automatically engage the brakes and stop the Powerchair.
- 8. **Charging Port**: This is the primary charging port for the 6Runner. When using this charging port only plug into this port with the qualified Shoprider charger certified by the original manufacturer of the Powerchair. This port should not be used as a power supply for any other electrical devices. Doing either of these as stated above will void the warranty of the Powerchair.

Note: The controller can be fitted to suit either right or left-handed users. Please contact your provider for further assistance.

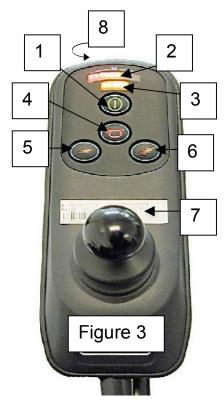



Locking/Unlocking the VSI Joystick

****PLEASE NOTE****

LOCK MODE: The VSI controller may be programmed to have a lock mode function enabled to prevent unauthorized use. The locking method is achieved through a sequence of key presses and joystick movements, as detailed below.

To lock Powerchair controller:

- While the controller is switched on, depress and hold the on/off button.
- After 1 second the controller will bleep. Now release the on/off button.
- Push the joystick forward until the controller bleeps.
- Push the joystick in reverse until the controller bleeps.
- Release the joystick, there will be a long bleep.
- The Powerchair controller is now locked.

To unlock the Powerchair controller:

- Use the on/off button to switch the controller on. The maximum speed/profile indicator will be rippling up and down.
- Push the joystick forward until the controller bleeps.
- Push the joystick in reverse until the controller bleeps.
- Release the joystick, there will be a long bleep.
- The Powerchair is now unlocked.

SLEEP MODE: If the controller is left on and not used for more than ten minutes, the controller will automatically "go to sleep". This is recognized by a slow intermittent flash of the battery indicator lights. Simply turn the controller off and back on to reset.

Always leave the chair in drive mode. The freewheel option is there only to allow the chair to be pushed manually when the need arises (i.e., to store or push unit out of a tight space). Following the sticker picture put each lever in the direction of the person sitting down driving the chair.



The Powerchair will not function while the drive mechanism is disengaged (in freewheel).

Both left and right sides must be in drive mode for power chair to operate. Otherwise the Powerchair motors will spin, but the unit will not move, until both freewheel levers are engaged. Unit will spin in circles if only one side is in drive mode.

*** DO NOT PUT INTO DRIVE MODE WHILE MOTORS ARE SPINNING ****

Cougar 10 – Cougar 14

The drive/freewheel levers are found toward the rear of the Powerchair, both have yellow colored handles. These levers will allow you to disengage the drive mechanism (When the levers are **pulled out** away from the unit) the Powerchair is in freewheel mode. *****NOTE***** the VSI will not work in Freewheel mode (See fig. on bottom) When the levers are **pushed in** (towards the inside of the unit) the motors are engaged and the powerchair is in drive mode.

FIGURE 4



Sticker will also help indicate in which position the brakes are locked or unlocked.

Free wheel lever **NOTE** There is a lever on opposite side



Driving Your Shoprider Powerchair Ramps, Slopes and Cambers


The indoor/outdoor Powerchair has the ability to negotiate ramps, slopes and cambers of a low to medium gradient. It is of the utmost importance that the following points are observed:

- Never attempt to go up or down an incline which has a rough, wet or slippery surface (loose gravel, tree roots, wet grass, polished floor, etc.)
- 2. Always approach an incline head on, not at an angle
- 3. **Always** travel at a speed that you are comfortable with and that you feel in complete control
- 4. Avoid sudden or erratic action with the joystick
- 5. **Always** have the reclining seat in the most upright position when driving up an incline
- 6. When riding along in a straight line you may experience "veering to one side". This is due to the natural camber in the construction of the pavement. To correct this you simply have to steer slightly against it, as you continue to move forward

Figure 5

12° (20), 75 (165)
9° (15), 90 (198)
6° (10), 113 (250)
0° (0), 136 (300)

Figure 6

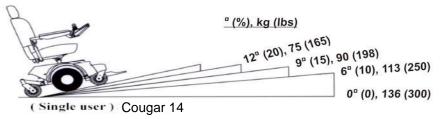



Caring for Your Shoprider Cougar Powerchair First Time Use

Prior to using your Powerchair for the first time, you must charge the batteries fully (refer to Battery Charging Procedures). Charging may take up to 12 hours. Charging the batteries completely prior to first time use will benefit battery efficiency and battery life.

Tires

Since your tires are flat free tires, there is no need to ever worry about tire air-pressure. When tread is worn, it is highly recommended to replace tires.

General Maintenance and Lubrication

- Inspection for loose wheel bolts, seat and arm tightening bolts, castor nuts and other nuts and bolts is recommended on a regular basis.
- Most lubricated components on the Powerchair are designed to be maintenance free and are sealed. Therefore, lubrication <u>is not</u> required for these components. However, the driving wheel axles and seat post should always have a <u>very</u> light coat of multi-purpose grease.**NOTE** <u>Never put grease or oil in the breather cap located on the top of each worm gear.</u>

DO NOT USE TOO MUCH GREASE!

Drips may stain or damage carpets, furnishings, etc.

- It is recommended that the driving axles (inside the drive wheel assembly) and castor assemblies be checked and cleaned occasionally for hair and debris build-up, which will hinder the performance of the unit.
- Check the 6Runner's batteries on a regular basis. For extended battery life, it is highly recommended that the batteries be fully charged after each use. If the batteries are weak, severely discharged or old, a variety of problems may occur with the unit including charging problems, reduced speed, loss of range, loss of charge capacity and others. Please contact your provider for battery replacement and proper disposal. Note: Charging your batteries daily will enhance the life of your batteries. If the batteries are left unattended for extended periods of time, it is recommended that they be left in a fully charged state. Leaving your batteries in a discharged state, or not charging them regularly will diminish the life of your batteries and cause irreversible damage to your batteries.



Batteries & Battery Charging

Batteries

The Shoprider Cougar Powerchairs are designed to operate on two 12 volt sealed lead acid aka SLA (AGM or gel-cell type) maintenance free batteries. The life of the batteries may be affected by temperature, terrain, weight of user, driving methods, and care of batteries. The active user will need to charge the batteries after using approximately 30% of battery capacity. As an inactive user, only using from once a week to every other day, the recharge point would be at 50% of discharge (Refer to Figure 8). For extended battery life, it is highly recommended that these guidelines be followed. You cannot over charge the batteries.

Keeping the batteries charged extends the life of the batteries

SLA batteries should be kept fully charged at all times. They <u>must not</u> be left in a discharged state if the unit is not being used for some time or the batteries will be permanently damaged and will no longer hold a charge. The SLA batteries should be checked once a month and kept charged as needed. Typically SLA batteries have up to about 400 charging cycles.

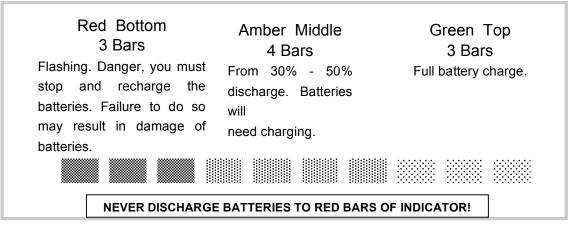


Figure 7



How to Charge Your Unit Cougar 10/14

1) Plug the A/C power cord into the wall. Plug the male three-pin charger plug into the female 3-pin receiver located on the bottom of the VSI. Turn the switch on if applicable (8Amp charger only)

The battery charger's light indicator is as follows:

One light charger (4Amp)

Yellow/Amber- Charging

Green- Charging Complete

Or

Two light charger (8Amp)

Red (solid)- Power ON Yellow/Amber- Charging Charged

NOTE: It is very important to check the charger light color. If there is no light or a different color light on than should be, refer to the charging trouble shooting section in this manual.

Depending on the amount of battery discharge, the charge time can take up to 12 hours. The battery charger has an electronic switch that will terminate the charge when the batteries are fully charged. Do not remove the battery charger until the charging cycle is complete. Be sure to unplug charger from the wall first before unplugging charger from the Powerchair.

WARNING! Only use the original manufacturer approved charger!

This charger is for indoor use only. Do not expose it to rain or water spray.

Do not leave the charger plugged into the Charging Port if the charger is not charging.



Troubleshooting

The Self-Help Guide (Table 1) is intended to assist in the location of a fault that may occur in a certain part of the Powerchair.

If after checking the fault from the table below and the fault is still showing, <u>do not</u> use the Powerchair. Turn off the power and consult your provider immediately.

Table 1

BATTERY GAUGE FAULT CODE	POSSIBLE VSI-50 FAULT
10 Bars flash	Battery voltage is too high, check the battery connections. Call your Shoprider Provider.
9 Bars flash	Brake Fault. Call your provider.
8 Bars flash	Possible controller fault. Check controller connector. Call your Shoprider Provider.
7 Bars flash	Possible joystick fault. Be sure joystick is at center when controller is turned on. Call your Shoprider Provider.
6 Bars flash	Battery charger connected. <i>Unplug charger</i> .
5 Bars flash	Right-hand motor wiring fault. Check connections. Call your Shoprider Provider.
4 Bars flash	Right-hand motor disconnected. Check connections. Call your Shoprider Provider.
3 Bars flash	Left-hand motor wiring fault. Check connections. Call your Shoprider Provider.
2 Bars flash	Left-hand motor disconnected. Check connections. Call your Shoprider Provider.
1 Bar flash	Low battery voltage. Charge batteries immediately. Do not drive Powerchair until batteries are completely charged!

Battery/ Charging Troubleshooting

If the charger indicator light goes to green within a few minutes after being plug in and doesn't charge the batteries the batteries may be over discharged and may require additional service or replacement. Contact your provider for assistance.

If the charger indicator light goes immediately to green when plugged in to charge, the problem is likely a blown fuse, popped circuit breaker (push reset button) or similar. Contact your provider for assistance. If the charger does not come on at all when plugged in, check the charger fuse if applicable, then call your provider.

Specifications

Table 2

	Cougar 10	Cougar 14
Overall Dimensions: L¹ x W x H	920 x 620 x 1120	990 x 690 x 1170
Total Weight of Unit With Seat and Batteries	100kg(w/)	126kg · (w/ 75Ah batt)
Maximum User Weight	136kg	136kg
Battery Capacity	12v – 50Ah x 2	12v-50Ah x 2
Charger	Off Board, 4 amp	Off Board, 8amp
Maximum Speed	8 kmph	8 kmph
Safe Climbing Angle, Maximum Single User Weight	Refer to figure 5	Refer to figure 6
Range ²	up to 25km	up to 25km
Turning Radius	514mm	572mm
Seat to floor height (Adjustable)	19",20",21"	24",25",26"
Ground Clearance	75mm	80mm
Output Power of Motor x2 (Reference Only)	.8 hpx2	1.34 hpx2

As a part of our ongoing product improvement initiative, Shoprider reserves the right to change specifications and design without prior notice. All specifications and dimensions are approximate. Includes the anti-tip wheel.

- The weight of the user
 Ground surface condition
- 3. Battery condition
- 4. Type of charger
- 5. Ambient temperature
- 6. Driving style
- 7. Terrain
- 8. After battery and mechanical moving parts have broken in

²The actual driving range varies with the factors shown below:

^{*}Proceed up any incline with seat in upright position*



Disclaimer

Congratulations on your purchase of the Shoprider Cougar Powerchair.

This Powerchair is not intended to be used by individuals with physical limitations that could prevent the user from operating the Powerchair safely. Shoprider disclaims all responsibility for any personal injury or property damage, which may occur as a result of improper or unsafe use of its products.

Warranty is only valid when genuine Shoprider parts are used. All modifications on the Powerchair, unless approved and authorized by Shoprider will automatically invalidate the warranties. Standard warranty does not extend to consumable items and parties other than the original user.

The preceding guidance is intended to assist you in the safe operation of this Powerchair. If you should have any questions about the correct operation of the Powerchair, please contact your authorized Shoprider provider.

WARRANTY TERMS AND CONDITIONS

<u>Structural Frame – 3 Year Warranty:</u>

Shoprider Australia warrants that the frame will be free from defects under normal use for a period of 3 years from the date of purchase by the original customer.

Shoprider Australia will repair or replace, at our option, free of charge **any part** found upon examination by Shoprider Australia to be defective in material and/or workmanship within the warranty period.

All parts claimed under warranty must be returned to Shoprider Australia in their present condition via the original place of purchase (SHOPRIDER AUTHORISED DEALER) listed on this form. Only an authorised WARRANTY DEALER of Shoprider Australia, following approval by Shoprider Australia, can undertake warranty replacement or repair.

<u>Drive Train Assembly – 18 Months Warranty:</u>

Shoprider Australia warrants that the drive train assembly will be free from defects under normal use, for a period of 18 months from the date of purchase by the original customer.

Shoprider Australia will repair or replace, at our option, free of charge **any part** found upon examination by Shoprider Australia to be defective in material and/or workmanship within the warranty period.

All parts claimed under warranty must be returned to Shoprider Australia in their present condition via the original place of purchase (SHOPRIDER AUTHORISED DEALER) listed on this form. Only an authorised WARRANTY DEALER of Shoprider Australia, following approval by Shoprider Australia, can undertake warranty replacement or repair.

Drive train components include:

Transaxle, Motor and Magnetic brake.

Other Parts – 12 Months Warranty:

Shoprider Australia will repair or replace, at our option, free of charge any part found upon examination by Shoprider Australia to be defective in material and/or workmanship for a period of twelve (12) months from the of purchase by the original customer.

Shoprider Australia will repair or replace, at our option, free of charge **any part** found upon examination by Shoprider Australia to be defective in material and/or workmanship within the warranty period.

All parts claimed under warranty must be returned to Shoprider Australia in their present condition via the original place of purchase (SHOPRIDER AUTHORISED DEALER) listed on this form. Only an authorised WARRANTY DEALER of Shoprider Australia, following approval by Shoprider Australia, can undertake warranty replacement or repair.

Exclusions:

- 1. All transport costs associated with the submission and return of parts for repair or replacement under warranty (as determined by Shoprider Australia) are the responsibility of the original customer.
- 2. Normal wear items are not covered under this warranty. These include such items as brushes, tyres, brake pads, upholstery, seating and arm rests.
- 3. Damaged caused by:
- Excessive, abusive, misuse, negligence or accident.
- Improper maintenance, maintenance carried out by non-approved WARRANTY DEALER
- Commercial use,

Not covered under this warranty:

- 4. Where an authorised dealer has not lodged all required documentation at the appropriate time the warranty is void
- 5. All warranties provided do not extend to purchasers of used or second hand equipment.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

THIS SCOOTER WAS PURCHASED FROM AND IS WARRANTED BY: DEALER NAME: ADDRESS: CONTACT PHONE: EMAIL:



Model Name-		
Provider Stamp		
	Serial No.:	
Shoprider Mobility		www.shoprider.com.au