

## **INSTRUCTIONS FOR SET UP/USE OF PORTABLE SHOWER TRAY**

- Unfold the shower tray, and position flat in the area to be used (dimensions when open 1100mm x1100mm x75mm)
- Locate two outlets options on shower tray, one on the base of the tray and one of the side.
- Where a floor drain/waste outlet exists; remove drain grid, position the shower tray base outlet over the floor drainage and remove plug for use.
- Ensure the floor outlet is pointing outwards, by unscrewing and refitting on the underside of the tray.
- Where there is no floor drain/waste outlet, a hose (optional) can be connected to the side outlet and directed to a suitable drainage point.
- Use the pump supplied, to inflate each of the four sides of the shower tray to trap water during showering/bathing.