



ILS

Rear Bag and Cane Holder Assembly Guide



1 Take side adapter and attach to the bottom of the L piece




2 Move side piece to the corner of the L piece and secure with screw



3 Attach the L piece to the rear receiver on the scooter



4 Secure with screw or metal pin

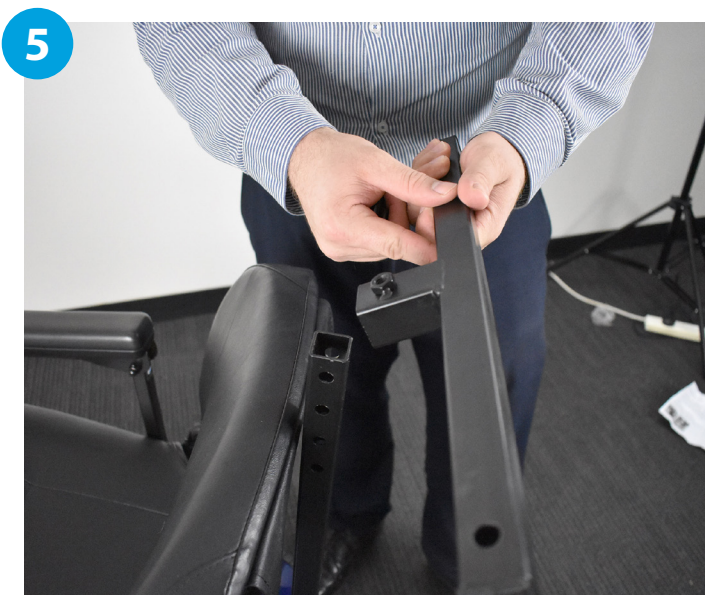


5 Attach T piece to the top of the L piece



6 Secure with screw

7

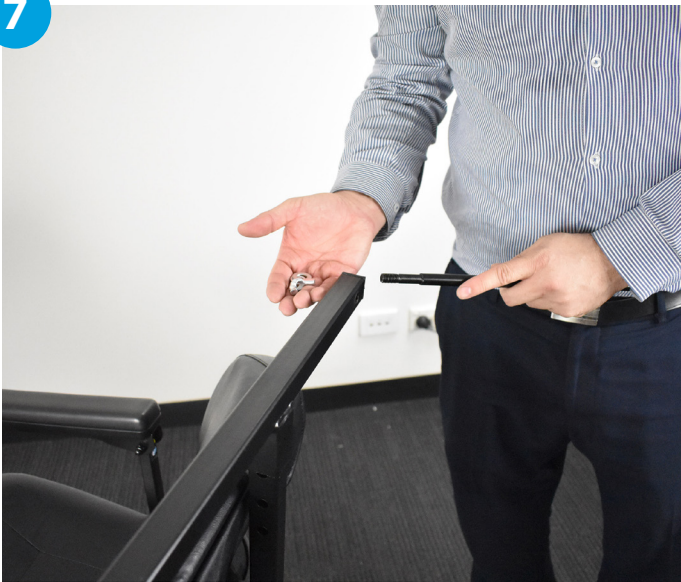

Using 2 washers and a wing nut install the bag pole through the holes in the T piece and secure with wing nut

8



Repeat the last step for the other side

9



Take the bag and align the poles into the rounded out material at the top ends of the bag

10



Position bag toward the frame as far as it can

11



Using Velcro straps attach the bag to the T piece

12



Insert the cane holder to the side adapter attached to the L piece

13



Secure the cane holder with a screw or metal pin

14



Please see completed rear bag and cane holder