



HERO+



User Manual - S13823

Tutor Plus
Forearm Walker
200kg



Forearm Walker - Hero Medical Tutor Plus
Hero Medical 200kg (Folding/Collapsible Portable)

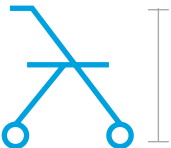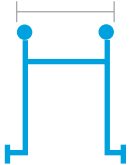

HEIGHT	WIDTH	HANDLE LENGTH	FOREARM GUTTER LENGTH	FOREARM GUTTER WIDTH	MAX S.W. L.
					
99.5 - 123.5cm	64cm	37.5 - 47.5cm	19cm	14cm	200kg

Assembly Instructions

1. Please remove contents from packaging
2. Un-fold the walker and secure into the open position with side frame locks
3. Place gutter adjustment poles in the opening at the top on both sides of the walking frame
4. Using the bolt secure into position until tight
5. Place handle pole in hole underneath the forearm gutter and tighten with screw provided

Maintenance Instructions

The unit should be inspected prior to daily use

1. Check castors rotate and spin without any hindrance
2. Check frame and supports for any sign of fracture or damage
3. Ensure the adjusters are secured and undamaged
4. Ensure forearm gutter foam are intact and the stitching is not compromised

Cleaning Instructions

1. Please wipe over with a warm damp cloth
2. Do not spray or submerge in water
3. Wipe fabric with warm damp cloth without detergent

A warning

Please take caution of pinching when folding and unfolding the walker



SWL : 200kg

- This product is designed to be used daily in the home or community setting