


Instruction on Assemble

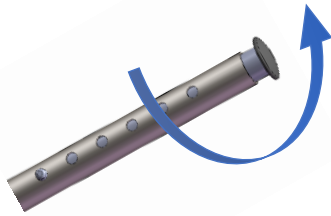

STEP 1: Extend the four adjustable legs from fully retracted position until locating button 'clicks' into first adjustment hole.

NOTE: THE INSERT TUBE MAY NEED TO BE ROTATED SLIGHTLY IN ORDER TO ALIGN BUTTON WITH ADJUSTMENT HOLE.

STEP 2: Working on an elevated table, position upholstered seat upside-down (squared corner of the seat facing you, round corners to the front).

STEP 3: Align one cross-member beam with the right-hand frame and the two nut fixtures incorporated into the seat (larger diameter holes facing up). Secure with two long bolts (**finger-tight only**). Repeat procedure for the second cross-member beam.

NOTE: THE 'TRIANGULAR' LUGS WELDED TO THE INSIDE OF EACH FRAME SLIDE INTO THE RECTANGULAR CROSS-MEMBER SECTION.

STEP 4: Repeat STEP 2 (above), this time aligning the two cross-member beam with the left-hand frame, again utilising the long bolts to secure the two cross-members to the frame and upholstered seat (**finger-tight only**).

STEP 5: Position Chair into 'normal' upright position. Align upholstered back with mounting position plates on the rear of the left/right hand frames. Use smaller round head bolts to affix back to frame.

STEP 6: Position the seat up side down to Tighten up bolts on the seats.

STEP 7: Position the chair into 'Normal' upright position. If the chair feels unlevelled use the adjustable feet on the bottom of the legs to fix the level up.

STEP 8: Utilise supplied tools to secure all bolts.

