JAY® JAY Foam Cushion

EN

Important Consumer Information

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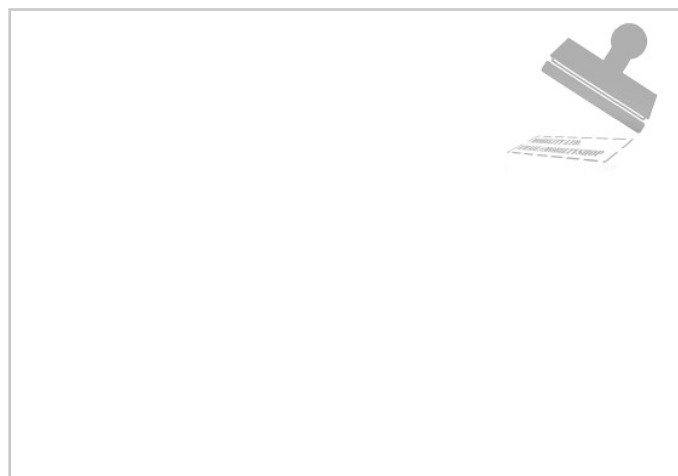
NOTE: Check all parts from shipping damage. In case of damage DO NOT use. Contact Carrier/Sunrise for further instructions.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Foam Cushion is appropriate. Cushions should only be installed by an authorized Sunrise Medical supplier.

⚠ GENERAL WARNINGS!

- DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Hook and Loop velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the velcro.
- Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- The JAY Foam Cushion is designed sitting comfort and postural support, remember no cushion can completely eliminate sitting pressure or prevent pressure sores.
- If you need more pressure distribution for skin protection, ask your authorized supplier for a free demonstration of any of the other JAY cushion products.
- The JAY Foam Cushion is not a substitute for good skin care including; proper diet, cleanliness, and regular pressure relief.
- Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- Avoid sharp objects or exposure to excessive heat or open flame.
- **OBSTRUCTIONS:** Do not place any obstructions between the user and the cushion because it will reduce product effectiveness.
- **CUSHION AND COVER ORIENTATION:** Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

INTRODUCTION

The JAY Foam Cushion is designed for sitting comfort and postural support.

The intended lifetime of this product is 5 years.

Maximum user weight: 150 kg

FITTING THE JAY FOAM CUSHION

- Select a JAY Foam cushion that matches the width of your wheelchair.
- The cushion length should extend to within 1" - 2" (2.5 - 5cm) from the back of the knees.
- Place the cushion to the back of the chair with the material identification tags to the back and the bump towards the front.
- The product label should be visible on the right side of the cushion.
- Adjust the footrests so your legs rest comfortably on the cushion.
- Lowering the footrests on your wheelchair may help reduce the pressure on your ischials, (seat bones) and coccyx, (tailbone).

NOTE:

- Proper footrest adjustment may enhance sitting comfort.
- Avoid using the cushion on overly stretched seat upholstery. Replace stretched seat upholstery or purchase a JAY solid seat insert for your cushion.
- Contact an authorized supplier for a free demonstration.

FITTING A LUMBAR SUPPORT

If your JAY foam cushion includes a lumbar support, its proper use may help to reduce lower back discomfort and sitting fatigue.

1. Adjust lumbar support height so that maximum thickness of lumbar support is at your waist or belt line.
2. Attach the lumbar support by looping one Velcro® strap attachment over the back of the chair, under the push handle and back around the front of the chair. The hook Velcro on the back of the strap should secure to the loop Velcro on the front of the strap.
3. Repeat the attachment process on the opposite side, making sure that the lumbar support is centered on the wheelchair.
4. If additional support is required, a 1/2" (1cm) thick foam insert can be placed inside the cover behind the lumbar support. This acts as a shim to increase the thickness of the lumbar support.

NOTE:

- Junior size lumbar supports attach with an elastic strap and buckle instead of Velcro® straps.
- Varying the height of the lumbar support during the course of the day may help relieve sitting fatigue.
- Placing the cushion about 1", (2.5cm), further forward in the wheelchair makes it more comfortable to use with the lumbar support.
- If your cushion did not include a lumbar support but you would like its benefits, contact an authorized supplier for a free demonstration.

EASY MAINTENANCE

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear.

TO CLEAN COVER

- Remove the cover from the base.
- Turn the cover inside out and machine wash in warm water and drip or tumble dry on low heat.
- No ironing required.

⚠ CAUTION!

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.

After washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE:

- If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.
- Use only JAY-designed covers in conjunction with JAY foam cushions.

TO CLEAN FOAM BASE

- Remove the cover from the foam base.
- Wipe with warm water and soap.
- Wipe off with a clean cloth.
- Avoid complete submersion of the foam base in water.

NOTE: See Cleaning Instructions overleaf.

HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

CONDITIONS FOR SAFE STORAGE

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the "this side up" arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the "do not stack" mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

WARRANTY

Each JAY cushion is carefully inspected and tested to provide peak performance. Every JAY cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There is no implied warranty beyond what is contained herein. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

Type:	Product Name/ SKU Number	XXXX-XXX-XX	Date of Manufacture
Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
 XXX mm	Seat width	MD	This symbol means Medical Device
 XXX mm	Seat depth		Manufacturer's address
 XXX kg	Max user weight	EC REP	European Authorised Representative
CE	CE Mark	UK RP	UK Responsible Person
UK CA	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

UK	RP	Sunrise Medical Limited Thorns Road, Brierley Hill West Midlands, DY5 2LD UNITED KINGDOM
-----------	-----------	---------------------------------------------------------------------------------------------------

SAMPLE

↑ FRONT ↑

JAY Part Code:

Date:

cm cm kg

Max

Sunrise Medical (US) LLC
2842 Business Park Avenue
Fresno, California, 93727 USA

Sunrise Medical GmbH
Kahlbachring 2-4
69254 Malsch / HD
Germany

Assembled in Mexico

EC REP **UK CA** **MD**

* WPC III X CE A i

Cover Materials:
Polyester
Urethane
Vinyl

THIS SIDE DOWN

↓ BACK ↓

248676 Rev. B

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

The varieties of cushions fitting variants, as well as the different positioning components to deliver higher positioning and skin protection benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning and skin protection needs and indications as follows[R7-R14] :

- Spinal Cord Injury - MS
- Neurological disease pattern (hemiplegia, M.Parkinson)
- Extremity defect deformity - Joint contracture
- Amputation - Tetraplegia

The cushion for a wheelchair shall not be used in case of [R3, R4]:

- Redness developed in the skin contact area
- Bottoming out present in the JAY Fluid Insert
- Decubitus

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

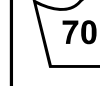
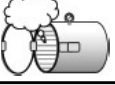
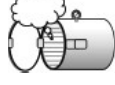
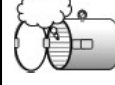
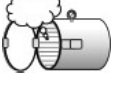
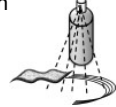
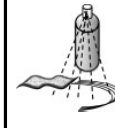
B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

Approved Cleaning Procedures for JAY Cushions

Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
Outer Cover (Microclimatic and Stretch)									
(1). Wet Wash 70°C 									
Outer Cover (Incontinent)									
(2). Wet Wash 60°C 									
Inner Cover									
(3). Spray and Wipe disinfection  or Wet Wash 60°C 	NA	NA	NA	NA	NA		NA	NA	
Cushion core (foam base)									
(4). Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
(5). Immersion disinfection 									
Cushion elements									
Air element	NA		NA	NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		

1. 70°C is the maximum recommended temperature.
2. 60°C is the maximum recommended temperature.
3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.
4. An autoclave is a self locking machine that sterilizes with steam under pressure.
5. At low temperature (< 30°C)

The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute: http://www.rki.de/EN/Home/homepage_node.html

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S