



novis molift[®]
by Etac

Molift

Hoists, Slings & Transfer Aids Catalogue



molift®

Molift by Etac - offers ergonomic hoists, slings and transfer assistive devices ensuring safe and comfortable transfers in various care settings with both the caregiver and patient in mind.

Sit-to-stand solutions

Molift Raiser Pro	7
Molift Quick Raiser 1 and Molift Quick Raiser 2	8
Molift Quick Raiser 205	10
Molift RgoSling Active	11

Mobile Hoists

Molift Smart 150	13
Molift Partner 255	14
Molift Mover	15
Sling bars for Molift mobile hoists	17
Molift Service Tool	18

Ceiling Hoists

Molift Nomad	20
Molift Air	21
Sling bars for Molift ceiling hoists	23
Molift Rail System	24
Molift Quattro	26
Molift Duo	27

Stretchers

Molift RgoSling Fabric Stretcher	29
Molift Stretcher	30

Molift RgoSling 4-point

Molift RgoSling HighBack	33
Molift RgoSling MediumBack	33
Molift RgoSling Toilet	34
Molift RgoSling Amputee	34
Molift RgoSling Shadow	35
Molift RgoSling Comfort HighBack	35
Molift RgoSling Ambulating Vest	36
Molift RgoSling Fabric Stretcher	36
Molift RgoSling MediumBack Plus	37
Molift RgoSling Repositioning sheet	37

Molift EvoSling 2-point

Molift EvoSling HighBack	40
Molift EvoSling MediumBack	40
Molift EvoSling LowBack	41
Molift EvoSling Hygiene	41
Molift EvoSling Ampu MediumBack	42
Molift EvoSling Comfort MediumBack	43
Molift EvoSling FlexiStrap	43

Molift UnoSling

Facts	46
Molift Slings accessories	47

Molift products are designed for life

Molift products are designed to give users a better quality of life while providing caregivers with the best support and optimum work environment. Our objective is to make solutions that improve the quality of life each day for users and give caregivers a safer working environment, enabling them to focus more of their time and energy on the users.

Best in class - Lasting durability

Molift products give disabled people the opportunity to make the most of life - go to school, work and travel – a better life through safe, efficient and comfortable transfer solutions. Molift transfer solutions are designed to be the best in class when it comes to product quality, reliability and durability.

The Molift story

Molift became a part of Etac 2010. The brand was founded more than 30 years ago in Norway by Hans Kasper Andresen. A Norwegian inventor, engineer and designer who, through his own experience (he was hospitalised due to a heart attack), saw the need for lifting and moving users in a secure way.



Molift Air

Molift Air is a small, flexible and strong ceiling hoist with an emphasis on high safety and long cost-effective lifetime. It is easy to handle, easy to move and easy to install and together with Molift Rail System it is a perfect match. Read more about Molift Air and Rail System in the chapter “Ceiling hoists”

Stable lifting and increased dignity

Molift hoists are equipped as standard with 4-point sling bar. The 4-point sling bar provides a very stable and safe lifting experience. Users enjoy a free, open sitting position with plenty of room for the head and upper body, they can feel comfortable and secure when sitting. No straps or parts of the hoist are inconveniently close. Combined with the Molift RgoSling, they provide extraordinary comfort and correct positioning from lying to sitting and vice versa as the sling loops glide automatically into position during hoisting. Most of the Molift hoists are also equally available with 2-point sling bars.



Secure, efficient and outstanding

With the increasing need for efficiency in the healthcare sector and a growing focus on users' and caregivers' needs and safety, top class transfer products are essential.

Cost effective solutions

Studies show that manual lifting and other user-handling tasks are high-risk activities for both carers and users. The aim for Molift products is to give the best and most secure user care, avoid heavy lifting situations which can put caregivers' backs at risk, while at the same time as providing the healthcare sector with the most cost effective solutions. Research and studies over for the past 20 years have shown that the costs of proper equipment are far lower than those associated with carers' work related injuries.

Our standards

Molift is certified according to the ISO standards for our trade. This affects how we drive and safeguard the quality of our products and our processes.

Systems for quality assurance

We are certified according to ISO 9001, which means that all our work processes, from product development and production to the products and their delivery meets the most stringent quality requirements.

Traceability

Molift meets all the requirements of ISO 13485 with regard to traceability of our products. All our hoists and slings have bar codes and unique serial numbers to facilitate traceability for our distributors on their market.

Environmental requirements

We work continuously to minimise the impact of our operation on the environment, and we are certified according to ISO 14001. This standard contains legal requirements on raw materials, emissions, recycling and power consumption.



Sit-to-stand solutions

Molift's sit-to-stand aids are based on the same ergonomic principles. The products imitate the natural pattern of movement and fulfill the transfer needs of users with some weight bearing capacity.

Molift Raiser Pro

The ergonomic platform for a safe and active sit-to-stand transfer

Early mobilization and rehabilitation

Molift Raiser Pro allows the user to use their own muscle power during a transfer. The natural movement pattern stimulated and the ergonomic design makes the user feel safe during the transfer. Raiser Pro helps the user to rehabilitate in all health environments during every transfer so that mobilization can begin early, already in the intensive care unit.

The stability and maneuverability make Raiser Pro ideal in confined spaces, such as toilets.

Accessories



Safety strap – Raiser Strap

Provides support around the back or hips when standing up.

S/M Item no.....82508
L/XL Item no.....82509



Safety strap with sliding function – Raiser Strap+

When a user needs to be assisted, Raiser Strap+ can be used by one or two caregivers.

S/M Item no.....82528
L/XL Item no.....82529



Raiser Belt

Raiser Belt facilitates the work of caregivers who assist users up to a standing position. Can be used by one or two carers

S/M Item no.....82964 (82508 + 82963)
L/XL Item no.....82965 (82509 + 82963)

Features

- A versatile handle with a soft warm feeling, with an optimal grip for both the user and the caregiver.
- The handle has two different fixing heights for accessories.
- Soft padded legrest with side support, height adjustable with only one hand.
- A central brake mechanism with a one-foot touch that gives audible feedback when activated.
- Low step-on height of 35 mm.



Product	Item no.	Material			
Molift Raiser Pro	16090204	Powder coated aluminum, plastic, steel, chrome 3			
Dimensions			For more information visit etac.com		
Height	1200 mm				
Width	540 mm				
Depth	630 mm				
Weight					
Total weight	10,7 kg				

Molift Quick Raiser 1 and Quick Raiser 2

Compact design and excellent manoeuvrability even in narrow spaces

Compact and easy

By virtue of its compact design and low weight, Molift Quick Raiser 1 and 2 are stable and easy to manoeuvre. The unique wheel position and near-perfect weight distribution of the base provides a small turning radius with easy manoeuvrability even in narrow spaces.

Quick Raiser 1 features an extra low base of just 7 cm, and with a narrow front of 34 cm, it fits well around toilets and under most beds, chairs and wheelchairs.

Quick Raiser 2 features a low base of just 10 cm, fits under most beds and chairs with the benefit of electrically adjustable legs to enable accessibility almost everywhere.



Natural pattern of movement

Both the Quick Raiser 1 and 2 have inclined columns that imitate the natural pattern of movement and fulfils the hoisting needs of users with some degree of weight-bearing capacity. Molift Quick Raiser 1 and 2 have a safe working load of 160 kg.


Features

- (Quick Raiser 1) Extra low base - The extra low base with narrow front for easy access.
- Adjustable knee pads - The knee pads are height adjustable and can be turned around for better fitting and to give alternative support for the legs.
- (Quick Raiser 2) Electrically adjustable legs - A perfect alternative when the hoist is used together with chairs with no accessibility underneath, such as electric wheelchairs and recliners.
- (Quick Raiser 1 and 2) Battery Indicator - All Molift hoists have an intergrated battery indicator that provides audio and visual notification.
- (Quick Raiser 1 and 2) Unique wheel positioning - the lightweight construction combined with the unique wheel positioning provides exceptional manoeuverability.

Molift Quick Raiser 1		Item no.	Technical	
V-shape arm		M04100	Battery	NiMH 14,4 V - 2,6 Ah
U-shape arm		M04150	Charge time	approx. 3 hours
Molift Quick Raiser 2			Hoisting speed 75 kg	50 mm/sec
V-shape arm		M04200	Rated performance	40 lifts at 75 kg and 500 mmt
U-shape arm		M04400	Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.	
Chassis Weight QR1/QR2	30/36 kg			
Dimensions QR1/QR2				
Height	1280 mm			
Leg height	70/100 mm			
Chassis length	955/1150 mm			
Chassis width	610 mm			
External Width of wheel base	1200 mm			



Accessories

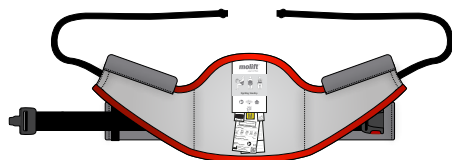

Molift RgoSling StandUp

Molift RgoSling StandUp is designed for a 2-point lifting arm. It is quickly and easily placed around the user. By virtue of the design it is ideal for use in toilet situations.

Features

- Perfect solution for hoisting and moving between different sitting positions.
- Large and firm support.
- Easy application.

Molift RgoSling	Size	Waist (mm)	Item no.
StandUp 2-point	XS	530-710	1720710
	S	640-850	1720720
	M	740-960	1720730
	L	930-1220	1720740
	XL	1150-1430	1720750
StandUp 2-point with support	XS	530-710	1720810
	S	640-850	1720820
	M	740-960	1720830
	L	930-1220	1720840
	XL	1150-1430	1720850

For more information regarding Active and StandUp lifting arms, see previous..
For complete variant matrix, contact your local distributor.

Material
Polyester



For more information visit etac.com

Molift Quick Raiser 205

An ergonomic sit to stand hoist with outstanding lifting capacity

Stable, safe and compact

Molift Quick Raiser 205 is a state of the art sit-to-stand hoist, with an electrically adjustable leg base. The hoist is comfortable, stable and safe with exceptional manoeuvrability. The hoisting of Molift Quick Raiser 205 imitates the natural pattern of movement and fulfils the transfer needs of users with some weight bearing capacity.

The unique wheel position, angled push bar and the extraordinary weight distribution of the base, provides a small turning radius. With the small castor option, it fits under most beds and chairs while the electrically adjustable legs enable accessibility almost everywhere. The fixed-leg option benefits a very small footprint.

Features

- Ergonomic design - the inclined column encourages a natural pattern of movement.
- Active 4-point Lifting arm - provides, together with RgoSling Active, extra support when hoisting from sit to stand.
- Easy to operate in moving and handling situations; toilet, sit-to-stand and stand-up training.
- Automatic Warning for periodic inspection and service.



Molift Quick Raiser 205		Model item no.		Weight	
Model	Battery	Adjustable leg	Fixed leg (FL)	Total weight (FL)	44 (42,5) kg
Active 4-point, castor size Ø75mm	SLA	M29000	M29150	Dimensions	External width of wheelbase 620 mm Internal width of wheelbase 200 mm Height of footboard: 65 mm Castor size Ø 75/50mm Castor size Ø 100/75mm
Active 4-point, castor size Ø100mm		M29100	M29200		
StandUp 2-point, castor size Ø75mm		M29001	M29151		
StandUp 2-point castor size Ø100mm		M29101	M29201		
Active 4-point, castor size Ø75mm	Lithium	M29002	M29152		
Active 4-point, castor size Ø100mm		M29102	M29202		
StandUp 2-point, castor size Ø75mm		M29003	M29153		
StandUp 2-point, castor size Ø100mm		M29103	M29203		
For complete variant matrix, contact your local distributor.					
Accessories		Size	Item no.	Technical	
StandUp 2-point lifting arm			2910061	Battery SLA	SLA 24V 2,9 Ah
StandUp slings		see matrix on next page		Battery Litium	LiFePO4 25,6V 2,4 Ah
Paediatric Footplate			2910065	Charge time	6 hours
Set of inlays for footplate			2920151	Rated performance	100 hoists/75 kg/500 mm
Leg support, bracket and cushions	Small		2910059	Lifting Range	600 mm
	Large		2910056	Turning radius	1150 mm
Set of cushions	Small		2910076	Hoisting speed 75 kg	35 mm/s
	Large		2910077	Hoisting speed 205 kg	30 mm/s
Lithium power pack and charger			2910041		



Molift RgoSling Active

The ideal sling solution for active transfers with Molift Quick Raiser 205


Comfort and easy transfer

Molift Sit to Stand slings are padded and developed for active hoisting together with Molift Quick Raiser 205. The slings provides perfect lumbar support and are the ideal solution for hoisting and moving between different sitting positions. The slings have an efficient anti-slip lining to prevent the sling from sliding up. The waist belt provides additional support and stability.


Molift RgoSling Active is designed for a 4-point lifting arm and has sliding loops for the possibility to an optimal posture during the sit to stand movement. Molift RgoSling Active provides additional grip and support around the hips when standing. The wide back-support distributes weight and pressure evenly over a large surface, which makes it very comfortable for the user.



RgoSling Active and Quick Raiser 205 with 4-point lifting arm

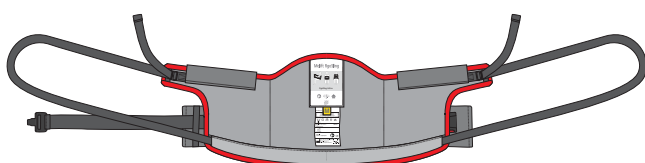

Molift RgoSling Active

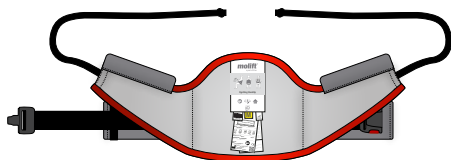

Molift RgoSling StandUp

Molift RgoSling StandUp is designed for a 2-point lifting arm. It is quickly and easily placed around the user. By virtue of the design it is ideal for use in toilet situations.

Features

- Perfect solution for hoisting and moving between different sitting positions.
- Self-adjusting sliding loops (RgoSling Active).
- Large and firm support.
- Easy application.

Molift RgoSling	Size	Waist (mm)	Item no.
Active 4-point	XS	500-720	1720910
	S	590-830	1720920
	M	730-950	1720930
	L	960-1210	1720940
	XL	1200-1390	1720950
StandUp 2-point	XS	530-710	1720710
	S	640-850	1720720
	M	740-960	1720730
StandUp 2-point with support	L	930-1220	1720740
	XL	1150-1430	1720750
	XS	530-710	1720810
StandUp 2-point with support	S	640-850	1720820
	M	740-960	1720830
	L	930-1220	1720840
XL	1150-1430	1720850	

Information

For more information regarding Active and StandUp lifting arms, see previous.. For complete variant matrix, contact your local distributor.

Material

Polyester



For more information visit etac.com





Mobile Hoists

Molift mobile hoists are well known for many innovative and intuitive solutions such as the 4-point sling bar, integrated service software, a compact lightweight design and an ergonomic, user friendly push handles.

Molift Smart 150

Light, foldable and transportable without tools

Compact smartness

Molift Smart 150 is designed to be easily wheeled and transported to where it is required. It is designed for home care, travelling and easy storage. Molift Smart 150 is foldable in one piece without using any tools.

Due to its low weight, the small footprint and the unique construction, Molift Smart 150 is easy to manoeuvre even in narrow spaces. The hoisting range of 270–1680 mm provides an excellent maximum hoisting height and at the same time gets very low, can lift from the floor. The 4-point sling bar ensures a comfortable and spacious hoisting position for the user.

Molift Smart 150 has been awarded for Design Excellence by the Norwegian Design Council.

Features

- Comfortable and spacious hoisting.
- Compact and foldable without tools.
- Low weight and small footprint.
- Lift from floor.
- Easy to manoeuvre.



Molift Smart 150 folded



Soft travel bag



Travel suitcase

Product	Item no.	Hoisting height	210–1665 mm	Rated performance	40 hoists/75 kg/500 mm
Molift Smart 150 (incl. sling bar)	M10200	Hoisting interval	1410 mm	Material	Aluminium, steel
Accessories	Item no.	Width chassis, legs open inner (mm)	925 mm		
Soft travel bag	3049177	Width chassis, outer (mm)	660 mm		
Travel suitcase	0990200	Weight	26 kg		
Dimensions		Technical			
Length	1200 mm	Battery	NiMH 14.4 V – 2.6 Ah		
Leg height	110 mm	Battery charge time	approx. 3 hours		
Dimensions folded	L1160 x W475 x H360 mm	Hoisting speed	60 mm/sec		

Molift Partner 255

The flagship of Molift's mobile hoists - with vertical hoisting pattern for excellent stability

Ideal solution for critical care

Molift Partner 255 provides an ideal solution for hoisting in acute and critical care areas in both hospitals and care facilities. The straight vertical column in combination with the fixed sling bar provides excellent stability. It maintains the user at the same distance from the column throughout the hoisting. With an outstanding hoisting range, Molift Partner 255 allows hoisting from the floor as well as high surfaces.

Molift Partner 255 is one of the lightest in its category and combined with excellent construction, ergonomic design and the caster solution, it becomes very easy to manoeuvre even with a lifting capacity of 255 kg.

Molift Partner 255 is available in several variants for the possibility of optimizing the product according to needs; low base chassis, supporting arms for gait/walk training and weight scale.

Features

- High safe working load - 255 kg
- Fixed sling bar and straight vertical column for safe and stable hoisting
- Low weight and easy to manoeuvre
- Suitable for use with Molift stretcher



Product Molift Partner 255 incl. sling bar M13000 Molift Partner 255 incl. sling bar and support arms M13100 Molift Partner 255 incl. sling bar and low base M13050 For complete variant matrix, contact your local distributor.		Item no.		For more information visit etac.com		Battery NiMh 26,4 V-2,6 Ah approx. 3 hours Battery charge time Hoisting speed 30 mm/sec Rated performance 50 hoists/75 kg/500 mm	
Accessories Casters ø100mm with directional lock 1320140 Weight scale (see page for sling bars) Sling bars (see page for sling bars) Stretchers (see page for stretchers)		Item no.		Dimensions Length 1285 mm Chassis width (outer) 700 mm Chassis width (inner) 550 mm Chassis width, open legs (inner) 1065 mm Hoisting height 390–1680 mm Hoisting interval 1290 mm Leg height 115 mm		Material Aluminium, Steel	
		Weight Total weight 42.5 kg					
		Technical					

Molift Mover

A family of compact, lightweight mobile hoists with great flexibility and large hoisting range. Suitable in homes, hospitals and care environments.

Large hoisting range

Molift Mover has a lightweight aluminium construction that makes the hoist very easy to manoeuvre. The low weight in relation to lifting capacity is the best in class. A large hoisting range allows hoisting from high surfaces as well as hoisting from the floor. The integrated Molift Service Tool ensures correct service and maintenance intervals.

Ergonomic handling

For improved caregiver ergonomics, the Molift Mover hoists are equipped with Molift's unique intuitive push handle. The handle provides multiple and flexible grip possibilities for improved caregiver ergonomics. The easily accessible hand control is very user friendly.

Molift Mover 180

Molift Mover is suitable for transfer situations in home, hospital and care environments. Optional 2- or 4-point sling bar with quick release.



Features

- A lightweight and compact hoist
- Very easy to manoeuvre
- Large hoisting range
- Hoisting from floor

Molift Mover Variants (excl. sling bar)	Item no.		
	Mover 180	Mover 205	Mover 300
Standard chassis, small casters	M18000	M14000	M15050
Standard chassis, large casters	M18003	-	-
Low base chassis	-	M14050	-
Standard chassis + supporting arms	-	M14150	M15150
Medium base chassis + supporting arms	-	-	M15151

For complete variant matrix, contact your local distributor.

Accessories	Item no.
Supporting arms	see etac.com
Weight scale	see page for sling bars
Sling bars	see page for sling bars
Stretchers	see page for stretchers
Casters ø100mm with directional lock for Molift Mover 205 and 300	1320140



For more information visit etac.com



Molift Mover 205

Molift Mover 205 is a workhorse for frequent and everyday use, suitable for transfer situations in nursingcare and hospital settings. A quick release function provides optional 2- and 4-point sling bars in different dimensions. A hoist with great flexibility and many accessories.



Molift Mover 300

Molift Mover 300 has the high hoisting capacity of 300 kg. It is the ideal solution for hospitals and care facilities with bariatric clients who have frequent and everyday moving and handling needs. With its low weight and high capacity, it is one of the lightest in its category.



Molift Mover 300 with supporting arm


Facts <i>for Standard chassis excl. sling bar</i>	Molift Mover 180	Molift Mover 205	Molift Mover 300
Maximum user weight (SWL)	180 kg	205 kg	300 kg
Total weight	32 kg	37.6 kg	40.1 kg
Length	1260 mm	1285 mm	1285 mm
Chassis width (outer)	700 mm	700 mm	700 mm
Chassis width (inner)	550 mm	550 mm	550 mm
Chassis width, legs open (inner)	1025 mm	1065 mm	1065 mm
Leg height	105 mm	115 mm	115 mm
Battery	NiMH 14.4 - 2.6 Ah	NiMH 26.4 V - 2.6 Ah	NiMH 26.4 V - 2.6 Ah
Battery charge time	3 hrs	3 hrs	3 hrs
Hoisting height	270-1630 mm	2093 mm	2100 mm
Hoisting interval	1365 mm	1431 mm	1420 mm
Hoisting speed	40 mm/sec	40 mm/sec	32.6 mm/sec
Rated performance	50 hoists/75 kg /500 mm	50 hoists/75 kg /500 mm	50 hoists/75 kg /500 mm
Service Software	Included	Included	Included
Material	Aluminium, steel, plastic	Aluminium, steel, plastic	Aluminium, steel, plastic

Sling bars for Molift mobile hoists

Lightweight sling bars with quick release coupling for easy change

Molift aluminium sling bars fit new and older hoists. The lightweight sling bar has a quick-release coupling, which makes it very easy for the caregiver to change.

The weight scale is class III approved and complies to all technical standards. It can calculate BMI (Body mass index) at the touch of a button.

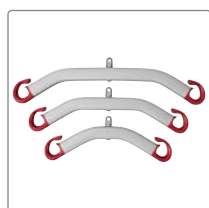

Mover 180 2-point sling bar
Aluminium/zink, SWL 300 kg
Small..... 1830043
Medium..... 1830042
Large..... 1830041



Mover 205 2-point sling bar incl weight scale
Aluminium/zink, SWL 300 kg
Small..... 1430120
Medium..... 1430121
Large..... 1430122




Mover 180 4-point sling bar
Aluminium/zink, SWL 300 kg
Small..... 1830110
Medium..... 1830111
Large..... 1830112
Extra large..... 1830113




Mover 205/300 4-point sling bar incl weight scale
Aluminium/zink, SWL 300 kg
Medium..... 1530110
Large..... 1530111



Weight scale Mover 180
SWL 300 kg
Scale..... 1840000

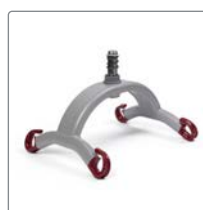

Partner 255 4-point sling bar
Aluminium/zink, SWL 255 kg
Medium..... 1330160

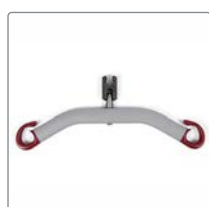

Mover 205 2-point sling bar
Aluminium/zink, SWL 300 kg
Small..... 1430117
Medium..... 1430116
Large..... 1430115



Partner 255 8-point sling bar
Steel, SWL 255 kg
Medium..... 2140002



Mover 205/300 4-point sling bar
Aluminium/zink, SWL 300 kg
Medium..... 1530101
Large..... 1530102
Extra large..... 1530103

Dimensions (mm)	Small	Medium	Large	Extra large
2-point Sling bar	H130 x L350	H130 x L450	H130 x L550	-
4-point Sling bar	H210 x W300 x L350	H240 x W300 x L450	H240 x W350 x L550	H240 x W350 x L650
8-point Sling bar	-	H420 x W920 x L1430	-	-


Molift Service Tool

The software that interacts with the hoist

Ensuring correct service and maintenance

All Molift hoists have an integrated service software that indicates when a service is necessary. This ensures user, caregiver and management that the hoist is safe in relation to applicable laws and regulations.

Molift Service Tool calculates the number of lifts and gives a warning signal when a service is required. Through Molift Service Tool, a technician can connect and interact with the hoist to perform and log service and for technical trouble-shooting. The software also allows reading, checking and adjusting key parameters in the hoist as well as logging, reading and printing the service history.

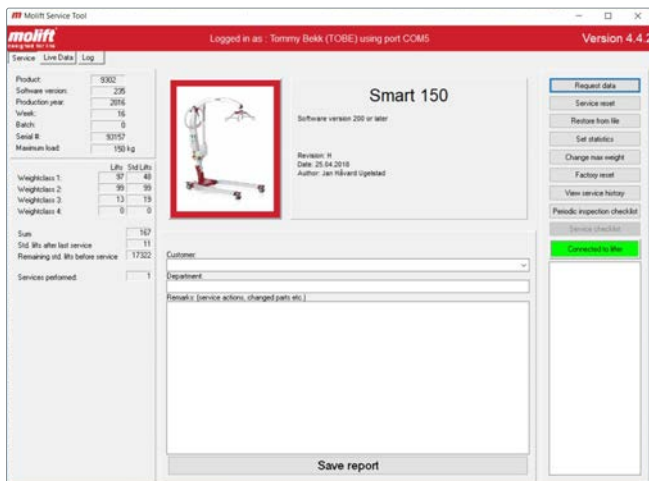

Molift Service Tool enables us to collect data about Molift hoists around the world and to continuously improve our products.

The software provides access to a product specific service checklist.

Record and analyse hoist usage

Molift Service Tool can also be used to retrieve data about hoist usage and hoisting patterns. When the data is linked to work-related injury among the employees, it can be a helpful tool for the employers to reduce injuries. Furthermore Molift Service Tool can improve efficiency as it indicates whether the types of hoists in use are suitable for the patient group and their needs.





Ceiling Hoists

Molift Ceiling Hoists and Molift Rail System are the optimal solutions to assist caregivers. The hoists are intuitive and safe to use and can solve all hoisting needs for disabled people and improve accessibility to the user and increases caregiver and user safety.

Molift Nomad

The portable low weight hoist, suitable for all day-to-day operations.

Low weight - high hoisting capacity

The weight of the Molift Nomad is only 6.8 kg. Despite the low weight it has a hoisting capacity up to 255 kg. Molift Nomad has a unique curved design and the integrated 4-point sling bar, used with any Molift sling, provides the user with a comfortable and spacious position. Molift Nomad is equipped with numerous safety features such as electronic overload protection, electrical emergency lowering and emergency stop.

Easy room-to-room transfer

Molift Nomad has integrated carry handles for easy handling. The connection is easy and no tools are required. In combination with the special designed trolley, Molift Nomad can be used to climb from one rail system to another without a transition rail. This results in easy transfer from room to room. By using Molift Nomad Extension Arm the hoist can easily be connected and disconnected to the trolley from the floor. Molift Nomad Cart is ideal for storing and transporting the hoist.

Features

- Low weight and high hoisting capacity.
- Portable ceiling hoist.
- Climbing room to room.
- Easy connection without tools to trolley.
- Integrated 4-point sling bar.



Description	Item no.	Length	522 mm	Hoisting range	Up to 2000 mm
Molift Nomad 205 (SWL 205 kg)	M22205	Width	200 mm	Material	Reinforced composite and aluminium
Molift Nomad 255 (SWL 255 kg)	M22255	Height	330 mm		
Accessories	Item no.	Weight		For more information visit etac.com	
Weight scale, class III	1100253	Total weight	6.8 kg		
Cart	1200005	Technical			
Extension arm L:900 mm	1200004	Battery	NiMh 26.4 V – 2.6 Ah		
Extension arm L:1050 mm	1200006	Battery charge time	approx. 3 hours		
Dimensions		Hoisting speed	30 mm/sec		
		Rated performance	80 hoists/75 kg/500 mm		

Molift Air

For a better working situation for carers and a safe and dignified quality of care for users

Dignified quality of care

Molift Air is a smart and user-friendly family of ceiling hoists, with outstanding lifting capacity. Molift Air is suitable for sitting and horizontal transfers as well as standing and gait training situations.

Together with Molift Rail System and a wide range of accessories, Air ceiling hoists offers great flexibility and eliminates strenuous manual lifting in a variety of care situations. A complete solution within patient handling, that creates a better working situation for carers and provides a better and more dignified quality of care for users.


Light and flexible

Molift Air is used with fixed ceiling rails or portable free-standing systems. The hoist is easy to remove and replace during installation and servicing, due to very low weight and the clever quick-release function. Together with all safety features, a large hoisting range and the multiple sling bar range, is this the complete solution in all health care settings.

Molift Air can easily be adapted to other manufacturers rail systems on the market.

Features

- Small hoist - great lifting capacity
- Flexible solutions
- Low lifecycle cost
- Silent and soft operation.

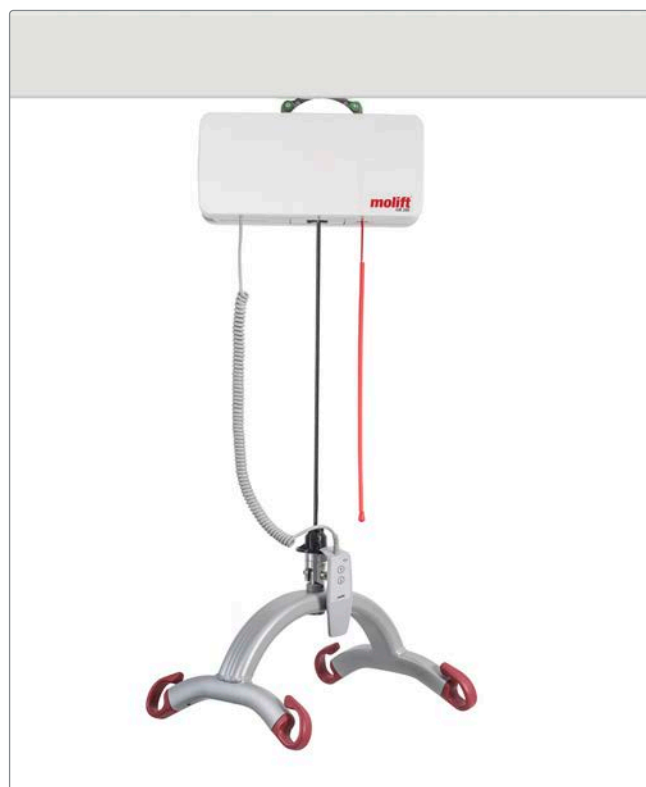

Molift Air 200

Molift Air 200, with its lifting capacity of 200 kg, is optimized for long term care facilities, nursing homes and private properties. The hoist is charged through the handset.

Product description	Battery	Item no.	Accessories	Item no.
Molift Air 200	SLA	M26200	Sling bars	see page for sling bars
Molift Air 200	Litium	M26201	Weight scale	see page for sling bars
Molift Air 350	SLA	M26350	Room to room transfer unit	2510295
Molift Air 350	Litium	M26351		
Molift Air 350 Propulsion	SLA	M26352		
Molift Air 350 IRC and Propulsion	SLA	M26353		
Molift Air 350 IRC	SLA	M26354		
Molift Air 500	SLA	M26500		

For complete variant matrix, contact your local distributor.

Material
Aluminium, plastic

For more information visit etac.com

Molift Air 350

Molift Air 350 has an outstanding lifting capacity and is optimized for long lifetime and low service costs. Molift Air 350 has several variants available; In-Rail charging, propulsion trolley and lithium battery cells.

Molift Air 500

Molift Air 500 is designed for bariatric users and has a lifting capacity of 500 kg. With the specially designed sling bar, the bariatric user can be lifted safely and securely.



Facts	Molift Air 200	Molift Air 350	Molift Air 500
Maximum user weight (SWL)	205 kg	350 kg	500 kg
Total weight (with SLA battery)	8.6 kg	8.6 kg	15kg
Length	360 mm	360 mm	360 mm
Width	190 mm	190 mm	250 mm
Height	195 mm	195 mm	195 mm
Battery Lithium	25.6 V LiFePO4 2.4 Ah	25.6 V LiFePO4 2.4 Ah	-
Battery SLA	26.4 V SLA 2.6 Ah	26.4 V SLA 2.6 Ah	12 V SLA 2.9 Ah
Battery charger	100-240 V AC, 40-60 Hz	100-240 V AC, 40-60 Hz	100-240 V AC, 50-60 Hz
Battery charge time	6 hrs	6 hrs	6 hrs
Motor	24 V DC	24 V DC	24 V DC
Hoisting interval	3000 mm	2600 mm	3000 mm
Hoisting speed 75 kg	60 mm/sec	60 mm/sec	30 mm/sec
Expected lifetime	30 000 lifts SWL/10 years	30 000 liftsw SWL/10 years	30 000 lifts SWL/10 years
Molift Service Software	Included	Included	Included
Emergency lowering	Mechanical and electrical	Mechanical and electrical	Mechanical and electrical
Emergency stop	Reachable from floor with string. Reset possible from floor		
Protection class; hand control & hoist motor	IP24	IP24	IP24

Sling bars for Molift ceiling hoists

Lightweight sling bars with quick release coupling for easy change

Molift aluminium sling bars fit new and older hoists. The lightweight sling bar has a quick-release coupling, which makes it very easy for the caregiver to change.

The weight scale is class III approved and complies to all technical standards. It can calculate BMI (Body mass index) at the touch of a button.



Molift Air 2-point sling bar
Aluminium/zink, SWL 350 kg
Small..... 2530115
Medium..... 2530120
Large..... 2530125



Molift Air 4-point bariatric pigtail sling bar
Steel, SWL 550 kg
Large..... 2510435




Molift Air 2-point pigtail sling bar
Aluminium, SWL 350 kg
Small..... 1830033
Medium..... 1830032
Large..... 1830031




Molift Air 8-point sling bar
Steel, SWL 300 kg
Medium..... 2140003



Molift Air 4-point sling bar
Aluminium/zink, SWL 350 kg
Small..... 2530210
Medium..... 2530211
Large..... 2530212
Extra large..... 2530213

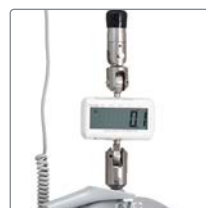

Weight scale Molift Air
(not available for 8-pt sling bar)
Scale SWL 300 kg..... 1840000
Scale SWL 400 kg..... 1431021

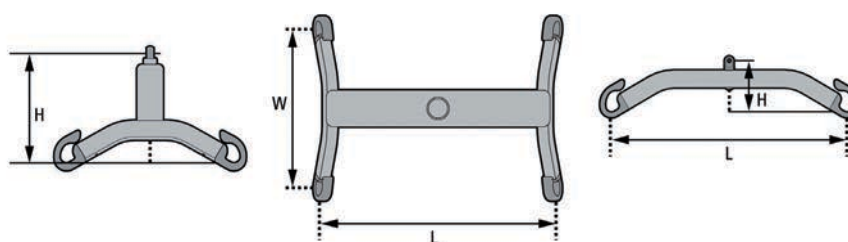

Dimensions (mm)	Small	Medium	Large	Extra large
2-point Sling bar	H130 x L350	H130 x L450	H130 x L550	-
2-point Sling bar pig tail	H150 x L400	H150 x L500	H150 x L600	-
4-point Sling bar	H210 x W300 x L350	H240 x W300 x L450	H240 x W350 x L550	H240 x W350 x L650
4-point Sling bar pig tail	-	-	H200 x W550 x L 550	-
8-point Sling bar	-	H420 x W920 x L1430	-	-

Molift Rail System

A cost-effective and flexible solution for all environments

Efficient installation process

Molift Rail System has rail profiles with innovative solutions for integration into all types of ceilings and walls. The wide range of fixing options results in making it one of the most most efficient installation processes in the market. It is easily integrated and is aesthetically adjusted to the design and architecture of the room. The range of switches and other transition options makes the rail system extremely flexible.

Customised capacity

Depending on the different needs and choice of the hoist motor, the Molift Rail System can be constructed from 160 up to 500 kg.

Cost effective solutions

The efficient installation process, the flexibility and the many features make Molift Rail System a cost-effective, competitive and profitable investment for hospitals, institutions, health care settings and the home. This helps create a practical and efficient workplace for nurses and caregivers, as well as an attractive environment for patients and visitors.

Traverse rail system

One of the most flexible solutions on the market, covering all needs of hoisting, regardless of renewal or equipment changes in the room.

Traverse switch

Enables two different rail systems to be connected, for example between a traverse system and a single rail. Very useful for transfers from one room to another.

Single rail system


Straight rail and/or curve rails gives a simple and effective patient transfer for the most common lifting situations such as bed-to-wheelchair hoisting.

Room to room transfer

When transfer is needed from one room to another or from one rail system to the next system, Molift can provide Room to Room function (Climbing), switch systems or continuing rail systems through doorways etc.

Rail switch

Rail Switch - enables a change of direction of single rails, for example from a straight rail to a curve rail.

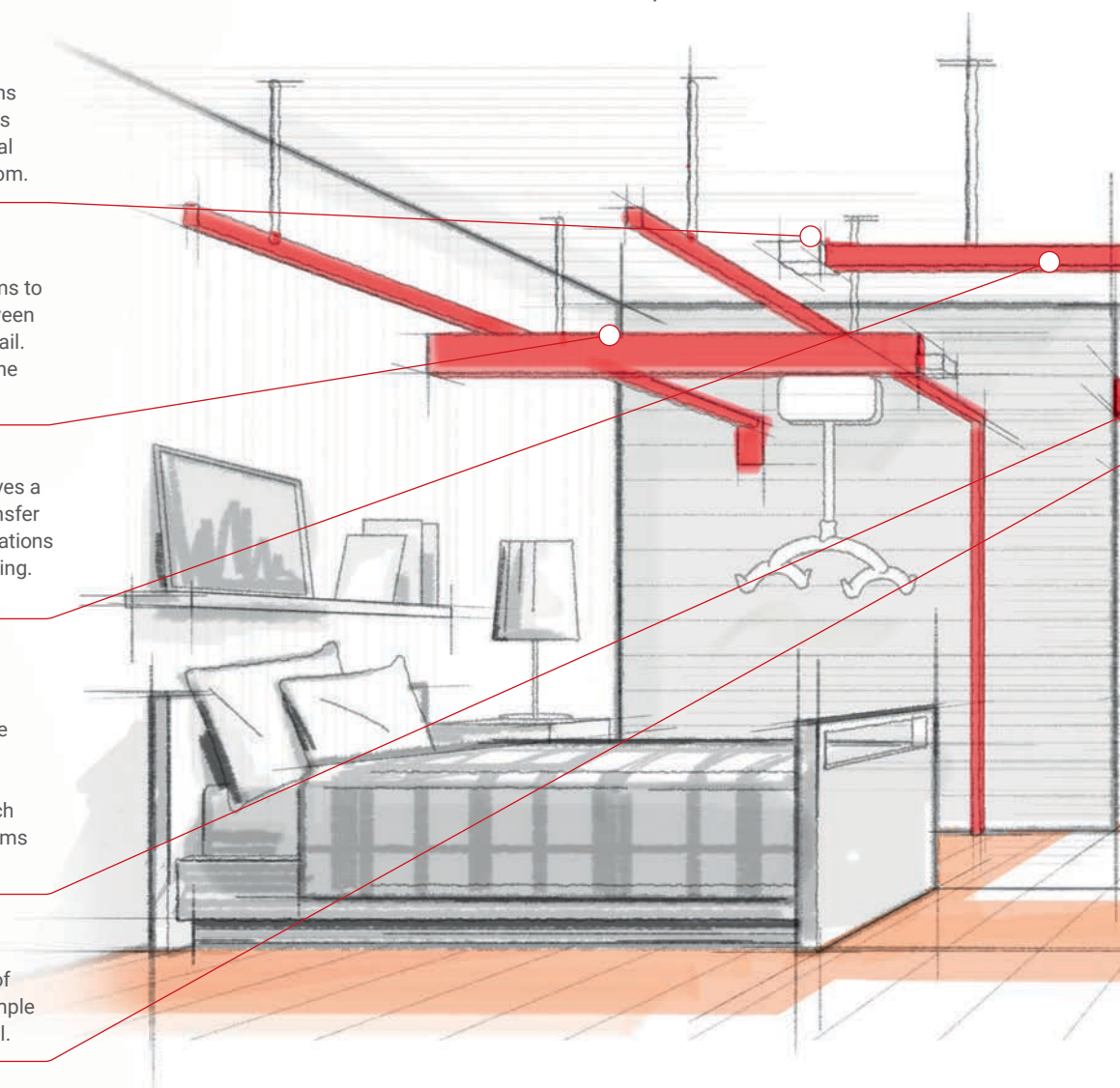

Flexible system

- Easy to install.
- Various weight classes.
- A variety of installation options – integrated in, or under the false ceiling, on the wall, on the floor with upright supports, etc.
- Can be installed in all types of ceilings, walls and floors.

Possibilities

Straight profiles
 Curves - 30, 45, 60 and 90 degrees
 Fixed brackets, telescope brackets
 Flexible traverse options
 Traverse switch
 Rail switch
 In-rail charging
 Curtain pass through

Maximum user weight (SWL)

160-500 kg

Colours

White

Rails


3 dimensions:
 Height: 62, 112, 142 mm

Max distance between fixtures

Up to 6000 mm

Switches

Rail switch
 Traverse switch

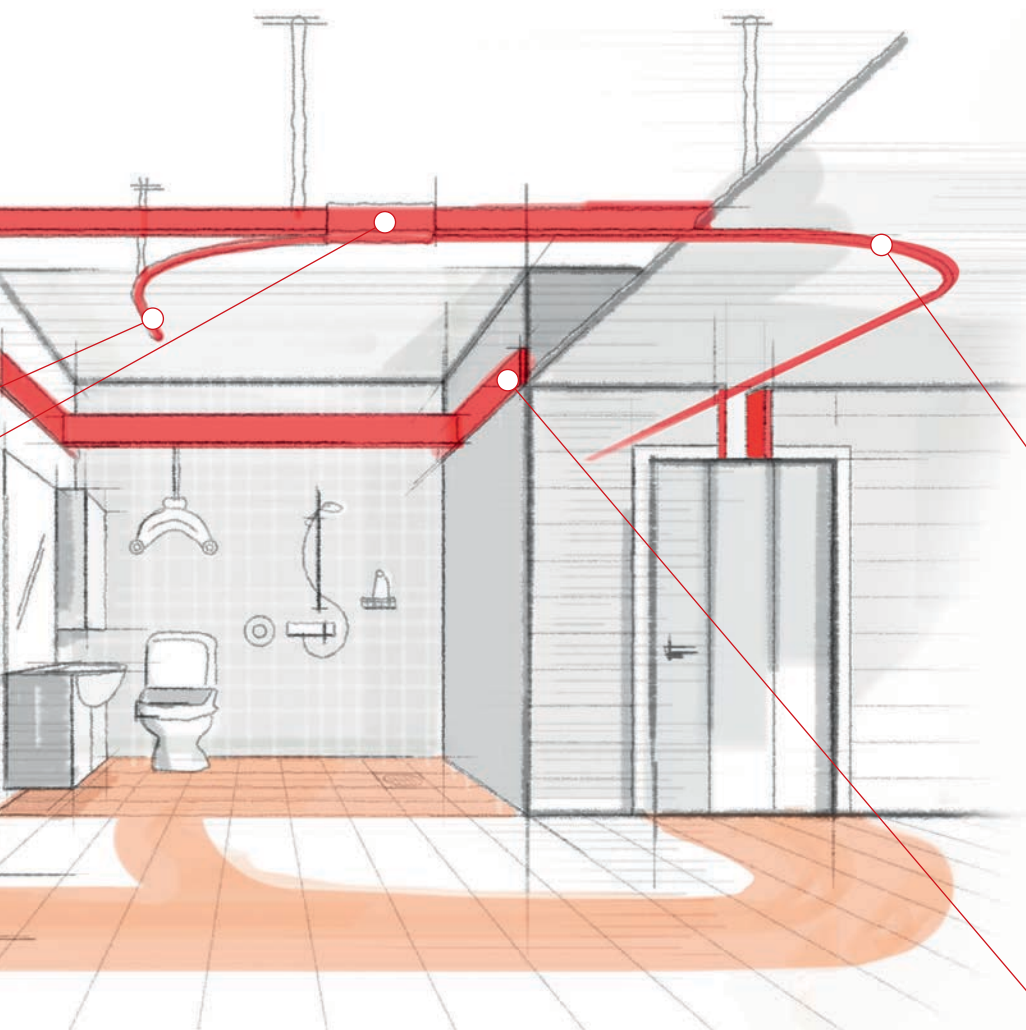

In-rail charging

In rail charging allows the hoist to charge batteries automatically anywhere in the system. This way the hoist is always ready for operation. In-rail charging can be mounted in straight rail and traverse rail systems only.

Integrated rail

Integrated Rail System - a single integrated rail offers a very discrete and clean solution.

Curtain pass through

Curtain pass through allows gaps for curtains mounted in the ceiling of the ward. This means several beds can share one motor, and still maintain privacy when individual nursing is required.

Wall mounted rails

Wall mounted rails - if the ceiling cannot support the system, it is possible to mount the rail on the wall.

Molift Quattro

The temporary hoist solution with excellent stability

A free-standing traverse gantry

Molift Quattro is a free-standing hoist unit that offers a (four post) Traverse System which is the best solution for temporary hoisting needs or when the facility is not built for fixed installations. The gantry is easy to install without the need to mount additional fixtures on walls or ceilings. It is ideally suited for temporary hoist applications in nursing or private homes.

For overhead lifting situations

The traverse has a lightweight aluminium frame and four steel posts that provides excellent stability and covers the surface of the hoisting requirement. Molift Quattro solves all overhead lifting situations together with the Molift Air or Molift Nomad hoist motor and a sling. With a lifting capacity of max 300 kg, it is ideal for lifting and transferring a person from or to bed, floor, chair, wheelchair, toilet etc.

Features

- Easy to install - designed for quick installation by two persons (preferably).
- No wall or ceiling mounting is required.
- Supreme stability and manoeuvrability .
- Easy to charge - the hoist motor can easily be charged through the hand control or through the in-rail charging option.



Description	Item no.	Height with adjustable feet	2360 – 2430 mm	Technical
Molift Quattro 3	1600030	Internal height	2190 mm	Expected Lifetime:
Molift Quattro 4	1600035	Width/Depth	3160 - 4160 mm	30 000 hoisting cycles (SWL) or 10 years
Accessories	Item no.	Maximum internal space	3000/4000 x 2190 mm	Material
Adjustable foot (set of 4)	1610200	The length of the posts are fixed. The lowest allowed distance from floor to ceiling is 2400 mm.		Steel, aluminium, plastic
Dimensions		Weight		
Boom lengths	3000/4000 mm	Total weight Quattro 3 (complete unit)	111 kg	
Height	2330 mm	Total weight Quattro 4 (complete unit)	132 kg	For more information visit etac.com

Molift Duo

A flexible and free standing gantry system for temporary hoist situations

The easy solution for most room sizes

Molift Duo offers a system that is easy to install without being fixed to the ceiling or walls. It is ideally suited to temporary hoist applications in hospitals, nursing homes or private homes. In combination with Molift Air or Molift Nomad hoist and a sling, Molift Duo solves most overhead lifting situations, whatever the environment. With a lifting capacity of 300 kg, it is ideal for lifting and transferring a person from or to; bed, floor, chair, wheelchair, toilet etc.

Lightweight construction


Molift Duo has a lightweight aluminium construction and a functional design which makes the gantry system easy to install, dismantle and move elsewhere.

This 2-post gantry system has a rail length between 2000 mm and 3500 mm and has 4 step adjustable heights up to 2550 mm, making this free standing hoist unit suitable for all bed and room sizes.

Features

- Easy to install - due to the lightweight system, it is easy to install the gantry and move it to a new location.
- Supreme stability - the gantry system has supreme stability and a functional design.
- Adjustable height - height positions can be set at four fixed positions from 225 cm up to 255 cm. Adjust the height with the quick release pin.
- Charging of lift motor - Molift hoist motors can easily be charged through the hand control or optionally through in-rail charging.



Description	Item no.	Dimensions	Material
Molift Duo 2000	1600013	Rail lengths 2000/2500/3000/3500 mm	Anodized aluminium, Plastic  max 300 kg / max 660 lbs For more information visit etac.com
Molift Duo 2500	1600015	Height 2250 - 2550 mm	
Molift Duo 3000	1600017	Depth 1200 mm	
Molift Duo 3500	1600019	Maximum internal space 3500 x 2400 mm	
		Weight Total weight (complete unit) 38 - 44,3 kg	





Stretchers

Molift offers a range of accessories for different needs. One of them is the unique Molift Strecher which can be used directly on a standard Molift 4-point sling bar.

Molift RgoSling Fabric Stretcher

A soft fabric hoisting device for horizontal transfers at hospitals and institutions

The soft stretcher option

Molift RgoSling Fabric Stretcher is a soft fabric hoisting device for hospitals and institutions. It is designed for horizontal transfers. Molift RgoSling Fabric Stretcher is suitable when hoisting in a normal sling would be inappropriate due to medical conditions, for example comatose or sedated clients that require a stable recumbent position. It is a comfortable alternative to a stretcher. Molift RgoSling Fabric Stretcher is compatible with Molift Partner 255 and Molift Air.

8-point sling bar

Hoisting with this product requires an 8-point sling bar. The sling bar is compatible with Molift Partner 255 and Molift Air. Molift Fabric Stretcher has integrated hoisting straps that are hooked directly to the sling bar, with some possibilities to adjust in length by using the loops. The multiple hoisting points provide an open and stable horizontal transfer.

Features

- Molift RgoSling Fabric Stretcher is suitable when hoisting in a normal sling would be inappropriate due to medical conditions.
- The application is easily performed, with an Immedia SlingOn, when the client is in a horizontal position.
- Open and stable horizontal hoisting - the 8-point sling bar with multiple hoisting points provides an open and stable horizontal transfer.



Description		Item no.	Material	
Molift RgoSling Fabric Stretcher M		1721520	Molift Fabric Stretch: Polyester	
Molift RgoSling Fabric Stretcher L		1721530		
Molift RgoSling Fabric Stretcher XL		1721540		
Dimensions				
Length outer	Medium	1750 mm		
	Large	1780 mm		
	Extra large	1830 mm		
Width outer	Medium	450 mm		
	Large	650 mm		
	Extra large	900 mm		
<p>* The safe working load will always be dependent on the hoists safe working load when less than the Fabric Stretcher.</p> <p>For more information visit etac.com</p>				

Molift Stretcher

A scoop stretcher, ideal for horizontal hoisting situations

Easy to use

Molift Stretcher is a scoop stretcher for hospitals and institutions. It is designed for horizontal transfers and when sling hoisting is considered inappropriate due to medical conditions. Molift Stretcher is an optimal solution for transfer of clients who are comatose or sedated, as well as clients with suspected spinal injuries and/or multiple fractures.

Ideal for emergency care

In combination with Molift standard 4-point sling bar, it is ideal for emergency care needs, when time is critical and the patient has limited mobility. Molift Stretcher can be divided in two parts in length and is therefore easy to place under the client.

Molift Stretcher has five handles along each side, which provides a manual hoisting alternative from the floor and allows horizontal and lateral transfer between beds and surfaces.

Features

- A unique patented scoop design, for safe and efficient moving.
- Low total weight and easy to carry.
- A safe locking system - easily opened when unloaded - completely locked when loaded.
- Many possible uses and compatible with X-ray, CT and MRI. Designed for minimal shadow in X-ray.




Description	Item no.	Total weight	14.5 kg		
Molift Stretcher MRI	2150108	Dimensions			
Material		Height	126 mm	<p>* The SWL always dependent on the hoists SWL when less than the stretcher.</p> <p>For more information visit etac.com</p>	
Plastic composite body		Length outer	1980 mm		
Stainless steel locks		Width outer	630 mm		
Weight		Compatibility	Molift Stretcher MRI: Compatibility with MRI. Approved for X-ray and CT-scan		



Molift RgoSling 4-point

Molift RgoSlings offer a wide range of models and sizes ensuring we have a solution, whatever the users requirements. Molift RgoSlings are designed for a 4-point sling bar.

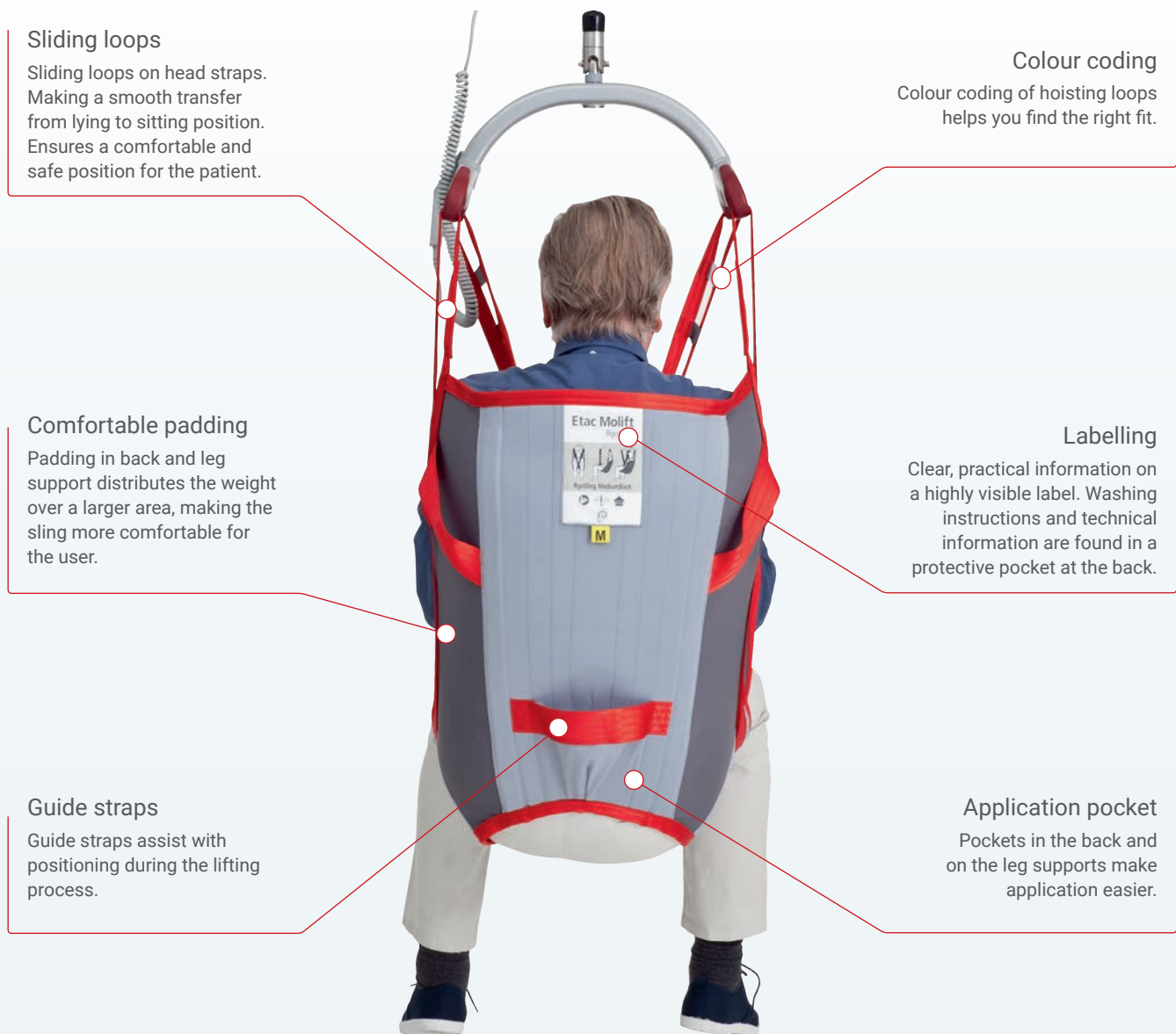

Sliding loops

Sliding loops on head straps. Making a smooth transfer from lying to sitting position. Ensures a comfortable and safe position for the patient.

Colour coding

Colour coding of hoisting loops helps you find the right fit.

Comfortable padding

Padding in back and leg support distributes the weight over a larger area, making the sling more comfortable for the user.

Labelling

Clear, practical information on a highly visible label. Washing instructions and technical information are found in a protective pocket at the back.

Guide straps

Guide straps assist with positioning during the lifting process.

Application pocket

Pockets in the back and on the leg supports make application easier.

Molift RgoSlings

Molift has a long reputation for manufacturing high quality hoists and slings. RgoSling is a portfolio of 4-point slings developed to meet the demands for the everyday needs in healthcare.

Standard models: HighBack and MediumBack

Our standard models are designed for use in the usual transfer situations and are suitable for most users. The standard models are available in two back heights: HighBack and MediumBack.

Models with specific features

We know that users have many different requirements, so we've designed additional models with specific features to meet their needs, including the RgoSling Toilet and RgoSling Amputee.



Molift RgoSling HighBack















Standard leg support opening

- Sliding loops on head straps.
- Support for the entire back and the head.
- Hoisting to and from a lying or a sitting position.
- All-round sling that fits most users.
- Hoisting from floor.

Molift RgoSling MediumBack

Standard leg support opening

- Sliding loops on head straps.
- Suitable for a user with headcontrol.
- All-round sling that fits most users.
- Adjust the sitting angle by the leg straps.

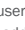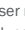Size	HighBack padded	HighBack net	Size	MediumBack padded	MediumBack net
XXS 	1720200	1720400	XXS 	1720100	1720300
XS 	1720210	1720410	XS 	1720110	1720310
S 	1720220	1720420	S 	1720120	1720320
M 	1720230	1720430	M 	1720130	1720330
L 	1720240	1720440	L 	1720140	1720340
XL 	1720250	1720450	XL 	1720150	1720350
XXL 	1720260	1720460	XXL 	1720160	1720360

Maximum user mass: 300 kg
HighBack padded XL - XXL 350 kg

Maximum user mass: 300 kg
MediumBack padded XL - XXL 350 kg



Molift RgoSling Toilet















Wide leg support opening

- Fixed loops.
- Easy access sling, facilitate dressing and undressing.
- For users with good core stability and muscle tonus.
- Comfortable padding.
- Flexible leg support.
- Belt included - for extra security.

Molift RgoSling Amputee


Narrow leg support opening

- Sliding loops on head straps.
- For users with an amputated leg or a user with reduced core stability or muscle tonus.
- Ideal solution for user with a high amputee.
- Comfortable padding.
- Belt included - for extra security.

Size	LowBack	HighBack	Size	MediumBack	HighBack
XXS 	1720500	1720600	XXS 	1721100	1721200
XS 	1720510	1720610	XS 	1721110	1721210
S 	1720520	1720620	S 	1721120	1721220
M 	1720530	1720630	M 	1721130	1721230
L 	1720540	1720640	L 	1721140	1721240
XL 	1720550	1721450	XL 	1721150	1721250
XXL 	1720560	1720660	XXL 	1721160	1721260



HighBack

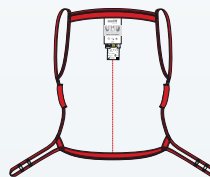

HighBack



Molift RgoSling Shadow













Standard leg support opening


- Sliding loops on head straps.
- Suitable for users in moulded seats.
- No straps, seams, pockets or labels in contact with the user.
- Polyester net with thin webbing: no straps, seams, pockets or labels in the contact surface to the user.

Molift RgoSling Comfort High-Back

Full body support sling without split leg

- Full body and head support.
- Can be left in the wheelchair underneath the user.
- Polyester net which provides good ventilation.
- Red seam indicating the center line of the back, to facilitate correct application.

Size	HighBack	Size	HighBack
XXS 	1722900	XS 	1721410
XS 	1722910	S 	1721420
S 	1722920	M 	1721430
M 	1722930	L 	1721440
L 	1722940	XL 	1721450
XL 	1722950	XXL 	1721460

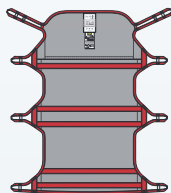

Molift RgoSling Ambulating Vest







For gait training with assistance



- 4 straps with fixed loops.
- Self tightening strap in the waist.
- Standing or balance exercise.
- Double buckles for extra security.
- Standard polyester fabric with padding.
- Padded groin straps is ordered separately.

Molift RgoSling Fabric Stretcher

A soft fabric hoisting device for horizontal transfers

- Molift RgoSling Fabric Stretcher is suitable when hoisting in a normal sling would be inappropriate due to medical conditions.
- Application easily performed.
- Standard polyester fabric with padding of the head part.
- Used with an 8-point sling bar.

Size	Item no	Size	Groin straps	Item no
XXS 	1721300	XXS-XS	single pack (order two for a pair)	1721700
XS 	1721310	S-M		1721720
S 	1721320	L-XL		1721740
M 	1721330	Size	Width x Length	Item no
L 	1721340	M	450 x 1750 mm	1721520
XL 	1721350	L	650 x 1780 mm	1721530
		XL	900 x 1830 mm	1721540

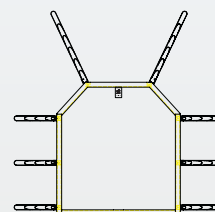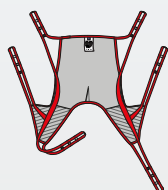

Molift RgoSling MediumBack Plus

Full body support for users weighing from 160 kg


- Provides full body support
- Straps with fixed loops and multiple height options
- Comfortable padded leg-support with dual straps
- Colour coded straps
- Split leg opening
- Can be used with a separate headrest

Molift RgoSling Repositioning sheet

An in-bed lift and positioning sheet

- Turning and transferring patients higher in the bed.
- Lifting from bed to another supine surface, stretcher etc.
- Stays in bed - for comfortable and dignified care.
- Facilitates care situations - saves time for caregivers.
- Colour coded straps facilitate communication when attaching the straps equally on both sides.

Size	MediumBack Plus	Size	Quantity	Repositioning sheet
L ■	1721940	L 1340 x W 1300 mm	1 pc	1721570
XL ■	1721950	L 1340 x W 1300 mm	10 pcs	1721570-C
Maximum user mass: 500 kg		Maximum user mass: 500 kg		

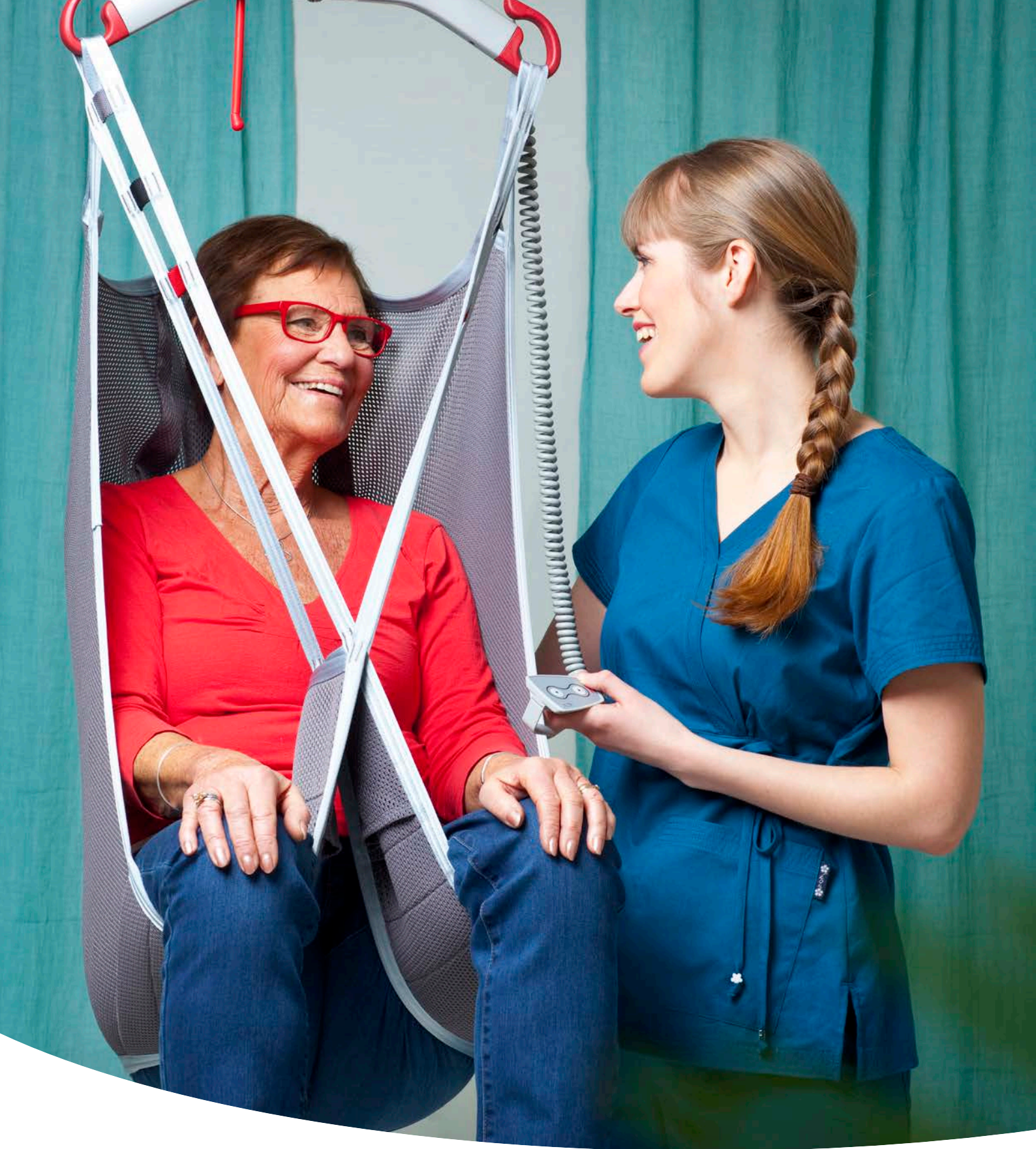

Molift EvoSling 2-point

Molift RgoSling offer a wide range of sling models and sizes ensuring we have a solution, whatever the users requirements. Molift EvoSlings are designed for a 2-point sling bar.



Adjust with fixed loops
 Hoisting loops on both head and leg supports allow the distance to the sling bar and the user's sitting angle to be adjusted.

Colour coding
 Colour coding of hoisting loops helps you find the right fit.

Comfortable padding
 Padding in back and leg support distributes the weight over a larger area, giving a much more comfortable lift for the user.

Labelling
 Clear, practical information on a highly visible label. Washing instructions and technical information are found in a protective pocket at the back.

Guide strap
 Guide straps assist with positioning during the lifting process.

Application pocket
 Pockets in the back and on the leg supports make application easier.

Molift EvoSlings

Molift has a long reputation for manufacturing high quality hoists and slings. EvoSling is a portfolio of 2-point slings developed to meet the demands for the everyday needs in healthcare.

Standard models: HighBack, MediumBack & LowBack
 Our standard models are designed for use in the usual transfer situations and are suitable for most users. They are available in three different back heights: HighBack, MediumBack and LowBack. The padded version has polyester fabric with padded back and legs and net version has polyester net with padded legrests.

Models with specific features
 We know that users have many different requirements, so we've designed additional models with specific features to meet their needs, including the EvoSling Hygiene and EvoSling Amputee.



Molift EvoSling HighBack

Standard leg support opening

- Support for the entire back and the head.
- Hoisting to and from lying or sitting position.
- Hoisting from the floor.
- All-round sling that fits most users.
- Net version has padded legs for extra comfort.

Molift EvoSling MediumBack

Standard leg support opening

- Suitable for a user with headcontrol.
- All-round sling that fits most users.
- Net version has padded legs for extra comfort.

Size		HighBack padded	HighBack net padded	Size		MediumBack padded	MediumBack net padded
XS	<input type="checkbox"/>	1722810	1722710	XS	<input type="checkbox"/>	1723010	1723110
S	<input checked="" type="checkbox"/>	1722820	1722720	S	<input checked="" type="checkbox"/>	1723020	1723120
M	<input checked="" type="checkbox"/>	1722830	1722730	M	<input checked="" type="checkbox"/>	1723030	1723130
L	<input checked="" type="checkbox"/>	1722840	1722740	L	<input checked="" type="checkbox"/>	1723040	1723140
XL	<input checked="" type="checkbox"/>	1722850	1722750	XL	<input checked="" type="checkbox"/>	1723050	1723150
XXL	<input type="checkbox"/>	1722860	1722760	XXL	<input type="checkbox"/>	1723060	1723160



Molift EvoSling LowBack

Standard leg support opening


- Suitable for users with head control.
- Comfortable reclined sitting position.
- All-round sling that fits most users.
- Net version has padded legs for extra comfort.

Molift EvoSling Hygiene

Wide leg support opening

- Comfortable padding.
- Belt for extra security.
- For users with good core stability and muscle tonus.
- Easy access sling, facilitate dressing and undressing.

Size	LowBack padded	LowBack net padded	Size	LowBack padded
XS <input type="checkbox"/>	1722310	1722510	XS <input type="checkbox"/>	1722210
S <input type="checkbox"/>	1722320	1722520	S <input type="checkbox"/>	1722220
M <input type="checkbox"/>	1722330	1722530	M <input type="checkbox"/>	1722230
L <input type="checkbox"/>	1722340	1722540	L <input type="checkbox"/>	1722240
XL <input type="checkbox"/>	1722350	1722550	XL <input type="checkbox"/>	1722250
XXL <input type="checkbox"/>	1722360	1722560	XXL <input type="checkbox"/>	1722260

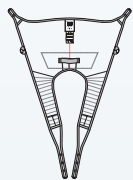

MediumBack net padded



HighBack net



Molift EvoSling Ampu Medium-Back

Narrow leg support opening

- For users with an amputated leg or a user with reduced core stability or muscle tonus.
- For user with headcontrol.
- Anti-slip material on part of the inside.
- Reclined sitting position.
- Comfortable leg padding.
- Polyester net.

Molift EvoSling Shadow HighBack

Standard leg support opening

- For users who need extra support.
- Suitable for user in moulded seats.
- Six straps gives more support, security and flexibility.
- Arms can be placed inside or outside the sling.
- Polyester net with thin webbing: no straps, seams, pockets or labels in the contact surface to the user.

Size	MediumBack net padded	Size	HighBack
XS <input type="checkbox"/>	1723210	XXS <input type="checkbox"/>	1723900
S <input type="checkbox"/>	1723220	XS <input type="checkbox"/>	1723910
M <input type="checkbox"/>	1723230	S <input type="checkbox"/>	1723920
L <input type="checkbox"/>	1723240	M <input type="checkbox"/>	1723930
XL <input type="checkbox"/>	1723250	L <input type="checkbox"/>	1723940
XXL <input type="checkbox"/>	1723260	XL <input type="checkbox"/>	1723950



MediumBack



Molift EvoSling Comfort Medium-Back

Full body support sling without split leg

- Ideal for users with joint problems or sensitivity to pain.
- Can be left in the wheelchair underneath the user.
- Polyester net, which provides good ventilation.
- Red seam indicating the center line of the back, to facilitate correct application.

Molift EvoSling FlexiStrap

Avoids heavy lifting of a leg or an arm

- Lifts a leg or arm to avoid heavy lifting.
- The FlexiStrap available in two widths.
- Padded polyester for comfortable lifting.
- Pocket on one side to facilitate application.

Size	MediumBack net	Version	Width x Length	FlexiStrap
XS <input type="checkbox"/>	1722610	Molift EvoSling FlexiStrap, standard	15 x 95 cm	1723310
S <input type="checkbox"/>	1722620	Molift EvoSling FlexiStrap, wide	25 x 95 cm	1723330
M <input type="checkbox"/>	1722630			
L <input type="checkbox"/>	1722640			
XL <input type="checkbox"/>	1722650			
XXL <input type="checkbox"/>	1722660			



Molift UnoSling

Slings for short-term use are specific to a single user, a patient specific sling. An effective solution for preventing the transmission of infection.

Molift UnoSling are designed to fit a 2-point or a 4-point sling bar.




Straps with 2 heights
Straps with 2 different height settings.

2-point or 4-point
The Molift UnoSling is suitable for use with either 2-point or 4-point sling bar.

Comfortable support
Support for the entire back and the head.

Label with user's name
At the back: large, clear label with practical information. The size is shown clearly and it is easy to add the user's name, which ensures safe use.

Disposable product
All-round sling that fits most users.

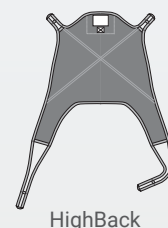

HighBack




Molift UnoSling HighBack

Standard leg support opening

- A personal disposable sling for short-term use.
- Hoisting to and from lying or sitting position.
- Support for the entire back and the head.
- For environments with a short term need for a sling,. Can be used several times by the same user.




Disposable product supplied in packaging which makes it easy to store and take out a new sling when needed.

Size	Number in package	Art. no.	
S	20	3060020	the user no longer needs it, or if it becomes damaged or dirty.   max 300 kg / max 660 lbs 
M	20	3060030	
L	20	3060040	
XL	20	3060050	
XXL	10	3060060	
Disposable product			
Do not wash. The UnoSling should be discarded when			


Facts

Materials

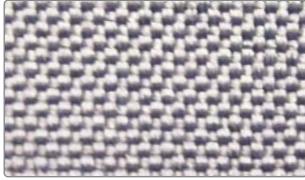

Polyester

The standard polyester fabric is durable and hard wearing. The material is easy to maintain and has a dirt repellent surface.

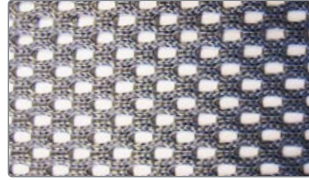

Polyester net

Hard wearing and durable soft net fabric, breathes well and dries quickly. Suitable for shower and bathing situations.




Non-woven

Polypropylene non-woven. Patient specific slings when the sling is for short term use.



Durable polyester

Extra strong durable polyester fabric for products that has a weight limit up to 500 kg. The material is easy to maintain and dries quickly.

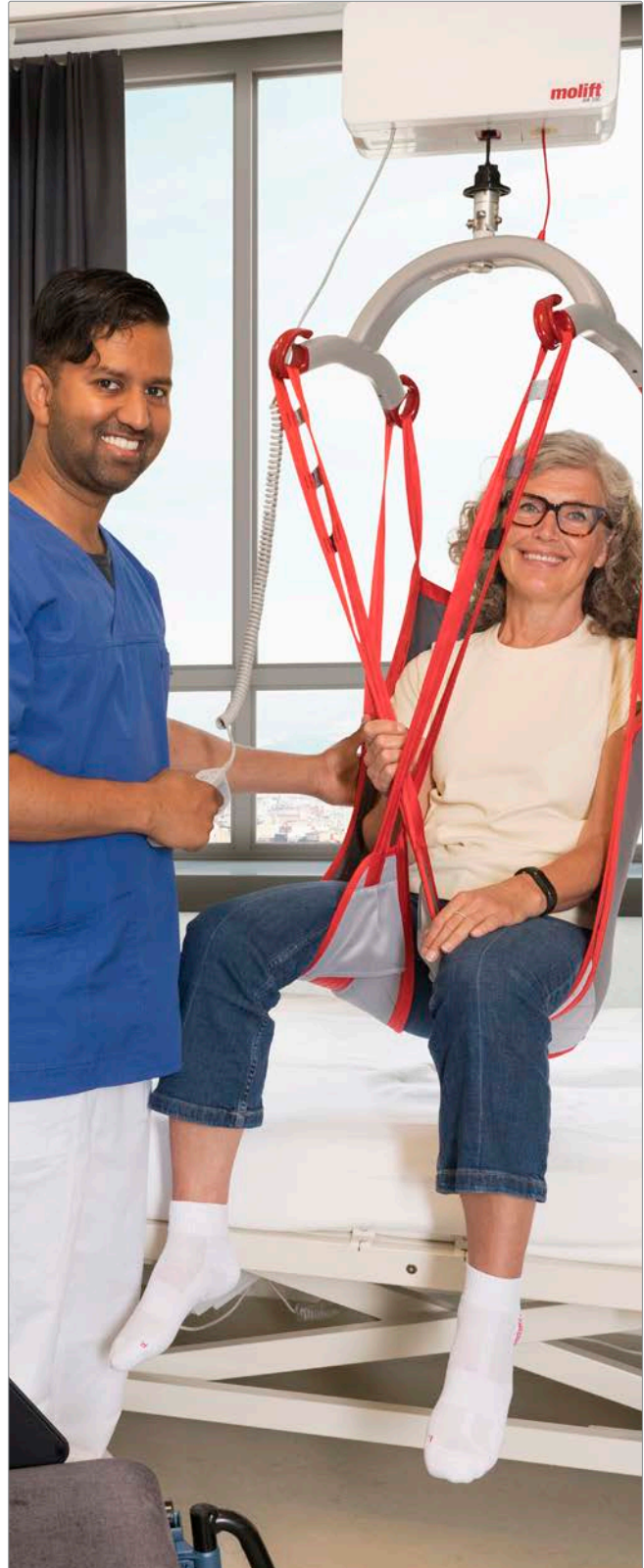

Detailed information

Size	Recommended user weight
XXS	12 - 17kg
XS	17 - 25 kg
S	25 - 50 kg
M	45 - 95 kg
L	90 - 160 kg
XL	160 - 240 kg
XXL	230 - 300 kg

Washing instructions



Washing temperature 60-85°C / 140-185°F
Washing at higher temperatures wears out the material faster.



All slings have a maximum user mass of 300 kg.
Some exceptions exist which are specified under each product.


Molift Slings accessories



Molift EvoSling Support

Accessories for slings without head support

The head support is attached to the sling and its length can be adjusted as required using the plastic buckle. It gives the user support for the head and is a useful addition when the user requires more support.

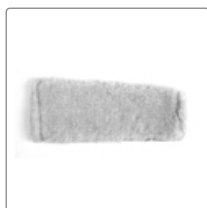

Molift EvoSling Soft Padding

For pressure-sensitive users

EvoSling Soft Padding consists of pressure-relieving padded sleeves which are slipped over the leg parts of slings with split-leg support. Washing 40°C / 104°F.



Molift RgoSling Extension Loops

Straps for easier adjustment

RgoSling: Used to extend the hoisting straps, flexible for 3 different length. N.B: Order 2 for a pair.



Molift EvoSling Extension Loops

Straps for easier adjustment

Used to extend the hoisting straps or to keep the leg supports together.

N.B: Order 2 for a pair.



Molift Strap Marker

For marking a selected hoisting loop for head or leg support hoisting straps. Mark the hoisting loops selected during individual testing of the sling.



Immedia SlingOn

When the user can't be turned onto their side

Immedia SlingOn used with a folding technique makes it possible to apply the sling without first turning the user over to the side. Washing in 80°C or 176°F.

Product		Item no.
Molift EvoSling Support		1723400
Product (sold in pairs)	Size	Item no.
Molift EvoSling Soft Padding	XS - M	1723710
Molift EvoSling Soft Padding	L - XXL	1723740
Molift EvoSling Hygiene Soft Padding	S - XXL	1723810
Product	Extension	Item no.
Molift RgoSling Extension loops (flexible)	+60, 180 or 260 mm	1721600

Product	Extension	Item no.
Molift EvoSling Extension loop	+90 mm	1723510
Molift EvoSling Extension loop	+180 mm	1723520
Product (sold in pairs)		Item no.
Molift Strap Marker		1723600
Product (sold in pairs)	Size	Item no.
Immedia SlingOn - Small	800 x 1000 mm	IM437/S
Immedia SlingOn - Medium	800 x 1700 mm	IM438
Immedia SlingOn - Large	1200 x 1500 mm	IM437



Proud distributor of Molift products in Australia.

novis ***molift***[®]
by Etac

1300 738 885 / novis@novis.com.au
12/12 Mars Rd, Lane Cove West, NSW 2066

Scan QR code for more information
novis.com.au/molift

