



HERO 

User Manual - S11198/S13165

Toilet Surround
205kg



Toilet Surround
- Hero Medical (Aluminium Rust Free)



Instructions for Assembly

1. Remove contents from packaging
2. Attach rear legs to the frame connectors
3. attach the arm rests to the top tubes
4. attach rubber feet to leg sections

How to use the Toilet Surround

Place the Toilet Surround around the toilet with the open side toward the back of the toilet. When sitting, move back until the seat is behind your legs, firmly place each hand on the armrest, if appropriate, to ensure your body is supported and the chair is stable. Then slowly lower yourself down.

When getting off the aid, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Place your hand on the armrests, if appropriate, to push yourself into standing leading with your head and shoulders.

Adjusting the Height

To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height. One leg has extra holes, which can be adjusted to help stabilize the shower chair for uneven surfaces.



- This product is designed to be used daily in the home or community setting