



**ROYALE**

## **User Manual**

**S15306/ S9705/ S10187/S7267**

---

**High Back Chair  
205kg**



Chair - High Back Chair - 205kg - Royale Medical



S15306

**Instructions for Assembly**

1. Remove contents from packaging
2. Insert adjustment legs into the side frames by securing with push pin
3. Attach side frames to seat section
4. Using provided screws please secure side supports to seat section
5. Insert Backrest onto rear poles to secure

## How to use the Chair

When sitting on the chair, move back until the seat is behind your legs, firmly place each hand on the armrest, if appropriate, to ensure your body is supported and the chair is stable. Then slowly lower yourself down.

When getting off the chair, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Place your hand on the armrests, if appropriate, to push yourself into standing leading with your head and shoulders.

## Adjusting the Height

To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height.



S15306



**SWL : 205kg**

- This product is designed to be used daily in the home or community setting