



Hero Medical Standard Seat Walker
USER MANUAL

Contents

SECTION	TITLE	PAGE
1	Introduction	2
2	Safety & Precautions	3
3	Features	4
4	Specifications	4
5	Instructions For Use	5
6	Labelling Information	8
7	Product Care & Maintenance	9
8	Recycle & Disposal	9
9	Warranty	10

No part of this user manual may be copied, distributed, reproduced, duplicated, scanned or stored whether in whole or part in any form or by any means without the expressed written consent from the officers of Independent Living Specialists Pty Ltd. Independent Living Specialists does not accept responsibility for any changes, errors or omissions. All information is subject to change without prior notice and products may vary from that of the actual images depicted.

1 Introduction



S11167 - Seat Walker - 6 inch Std Burgundy (with Bag) - Hero Medical 140kg
 S11168 - Seat Walker - 6 inch Std Blue (with Bag) - Hero Medical 140kg
 S11169 - Seat Walker - 6 inch Std Purple (with Bag) - Hero Medical 140kg
 S11990 - Seat Walker - 6 inch Std Champagne (with Bag) - Hero Medical 140kg
 S11991 - Seat Walker - 6 inch Std Black (with Bag) - Hero Medical 140kg
 S11172 - Seat Walker - 8 inch Std Burgundy (with Bag) - Hero Medical 140kg
 S11173 - Seat Walker - 8 inch Std Blue (with Bag) - Hero Medical 140kg

Safety & Precautions 2

Weight Capacity

The maximum safety weight limit for the seat walker is 140kg and must not be exceeded. If the maximum safety weight limit is exceeded, this can increase the risk of damage to the seat walker and/or risk of serious injury to the user and others.

User Safety

To ensure user safety, it is highly recommended that the below safety checks are completed prior to each use of the seat walker:

- Brakes are functional.
- Castors do not show signs of wear and tear.
- Backrest, seat, hand grip, storage bag and other removable parts are properly attached to the seat walker.
- Ensure backrest is securely locked into position before use - make sure pushbuttons are firmly protruding before leaning against backrest.

Note: Failure to complete the above safety checks may result in seat walker malfunction and has the potential to cause injury to the user and others.

Exposure to Weather

Avoid exposure to hot or cold temperatures for extended periods of time, as it can cause parts of the seat walker to become very hot or cold. This can result in an increased risk of damage to the seat walker and/or risk of injury to the user and others.

Uneven Terrain

Operating the seat walker on uneven terrain increases the risk of the seat walker tipping over. Only operate on even and solid surfaces.

Intended Use

The seat walker is designed to be used as a walking aid. Before sitting on the seat walker, please apply brakes and ensure it is on even ground. Breakages and injuries may occur as a result of attendant pushing an occupant on a seat walker is extremely dangerous.


3 Features

- ▶ Lightweight aluminium frame
- ▶ Folds easily for transportation and compact storage
- ▶ Swivel front wheels for easy maneuvering.
- ▶ Height adjustable handles
- ▶ Foldable padded backrest

4 Specifications




Name	Standard 6"	Standard 8"
SKU	S11167 Burgundy S11168 Blue S11169 Purple S11990 Champagne S11991 Black	S11172 Burgundy S11173 Blue
Frame material	Aluminium	Aluminium
Frame weight	5kg	5kg
Product weight	6.7kg	8.4kg
Overall height (floor to handles)	82cm - 94cm	82cm - 97cm
Seat height (seat to floor)	55cm	56cm
Seating width	44cm	44cm
Outer width (including back castors)	56.5cm	58cm
Front Castors	6"	8"
Rear Castors	6"	8"
Safe work load (SWL)	140kg	140kg

Instructions for Use 5

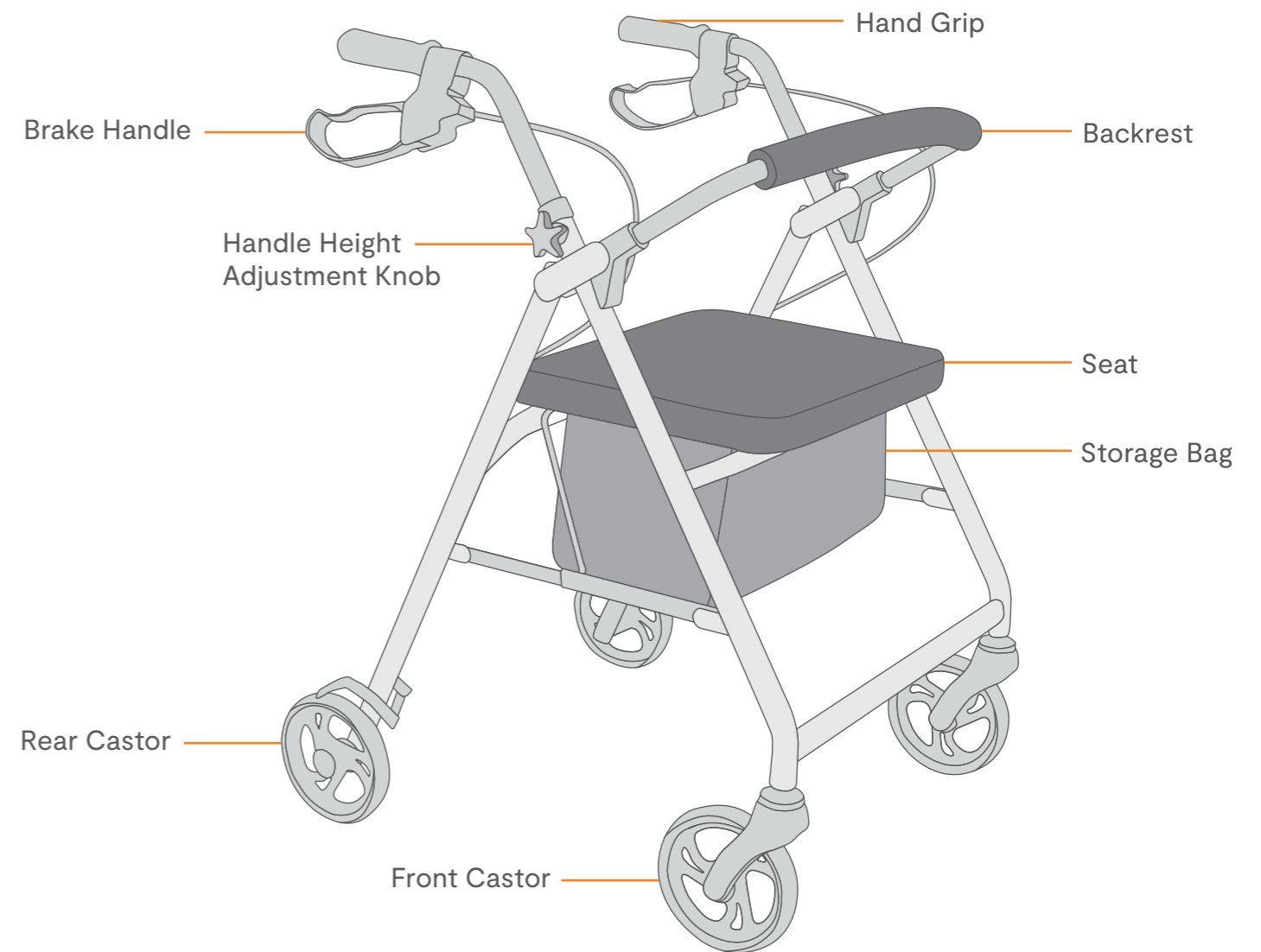

Product Assembly

- Unboxing the seat walker
 - Comes with two main components, the frame and the backrest bar
 - Place seat walker into standing position by pressing down on centre aluminium bar
 - Remove items from storage bag
- Attaching backrest
 - Inset backrest into designated slots so that push buttons are facing outwards
 - Push holes in and continue to insert backrest until buttons securely lock into holes
- Attaching handles
 - Insert handle into frame receiver with handles facing inwards for palm comfort
 - Select desired handle height and thread bolt from the inside hole to meet plastic washer and plastic hand screw on the outside
- Securing brake cables
 - Use plastic clip that is attached to the brake cable and clip onto aluminium frame

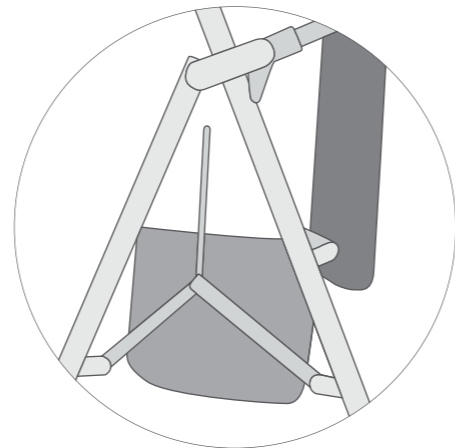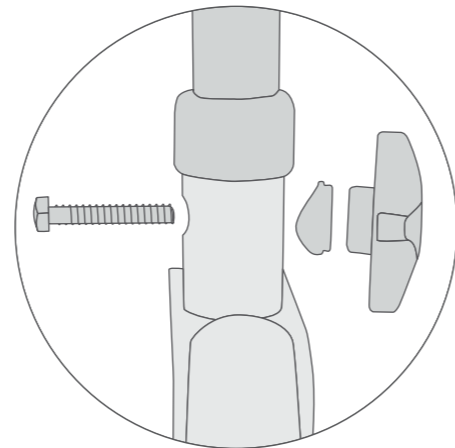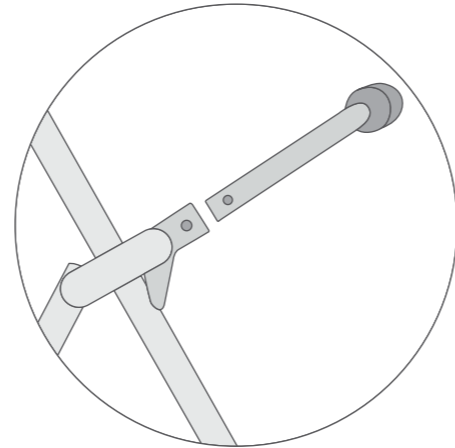


Folding the Seat Walker

- Pull metal bar situated under the seat
- This will bring up the aluminium bar connecting front and rear wheels
- Reverse for unfolding, ensuring frame has been fully extended before using

Using the Brakes

The seat walker features handbrakes with an integrated locking function to provide optimal control and safety for the user.

- To apply the brakes when in motion, pull the handbrake lever towards the handle and release to disengage brakes.
- To lock the brakes when the seat walker is parked, push the handbrake lever down until it locks. To release the brakes, pull the handbrake lever towards the handle.
- To adjust the tension of the brakes, tighten or loosen the tension adjustment nut (located at the end of the brake cable above the rear wheels). Adjusting the tensions of the brakes ensures the safe operation of the brakes when in use.

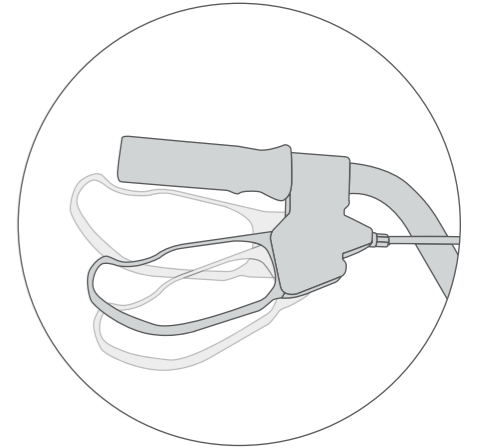


Sitting on the Seat Walker

- Visually inspect walker to ensure there is no visible damage before sitting on walker.
- Please ensure that the hand brakes are in the locked position by following the steps listed in applying brakes.
- Please ensure you are on a stable surface.
- Turn facing backwards the walker and hold both handles firmly, move back until your legs touch the walker and slowly lower yourself onto the walker.
- Please ensure that your weight is centered and not to sway your body weight from side to side.
- **WARNING:** Do not use seat walker as a wheelchair aid.

6 Labelling Information

a. Product Label


The seat walker has a product label on the front of the frame. The product label includes the following information:

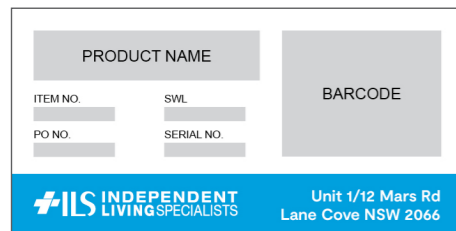

- Name and address of the company
- Name and code for the product
- Safe Working Load (SWL)
- Serial number of the product (which includes the date of manufacture and the batch serial number)


b. Usage Label

The seat walker has a usage label on the front of the frame. The usage label includes the following information:

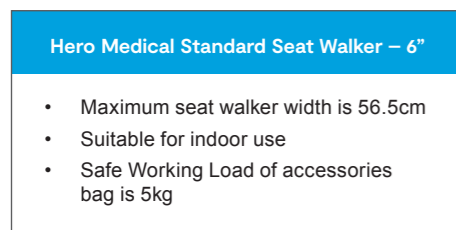

Label for product variations with 6" castors:
S11167, S11168, S11169, S11990 & S11991.

- Maximum seat walker width: 565mm
- Suitable for indoor use
- Safe Working Load of accessories bag is 5kg

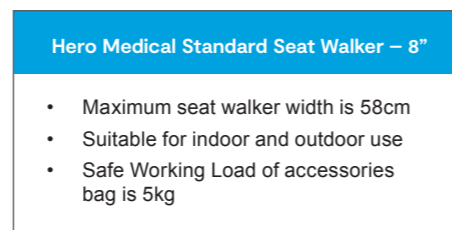

Label for product variations with 8" castors:
S11172 & S11173

- Maximum seat walker width: 580mm
- Suitable for indoor and outdoor use
- Safe Working Load of accessories bag is 5kg

Product Care & Maintenance 7

- It is recommended that the seat walker is inspected regularly for signs of wear and tear of the castors, seat or backrest.
- If there are visible signs of wear and tear, please contact your authorised dealer for replacement parts.
- Please ensure the brakes are tested prior to each use.
- If exposed to water or moisture, promptly dry the seat walker to prevent corrosion of the frame.
- The seat walker should be wiped down regularly using warm water or a mild detergent.
- Only clean your seat walker with soft cleaning implements such as soft sponges, dish clothes and soft-bristled brushes. Warning, using sharp cleaning tools or similarly abrasive cleaning products could cause damage to your seat walker.
- Clean wheels with a mild detergent and warm water, removing hair and other dirt from axles.
- Lubricate with multi-purpose oil annually or if wheels begin to squeak.
- All replacement parts as well as after-sales support and repair services are available through Hero Medical dealers.

Recycle & Disposal 8

Please follow the local government rules regarding proper recycling and disposal. It should always be done through an authorised agent or licensed place of disposal. Please review the list of seat walker parts and packaging materials to ensure they are recycled or disposed of correctly.

- Frame: tubular aluminium
- Seat & backrest: vinyl & nylon
- Castors and handgrips: polyurethane
- Packaging: cardboard box and plastic

9 Warranty

The Hero Medical Standard Seat Walker comes with 12 months warranty from the date of purchase on structural integrity.

- During the warranty period any parts that have become defective due to faulty workmanship or material will be repaired or replaced without charge by a Hero Medical dealer of where the product was purchased.
- The warranty is for normal domestic use only and excludes any defect or injury caused by or resulting from misuse, abuse, neglect, accidental damage, or any other alteration that affects the reliability or performance of the unit, not attributable to faulty manufacture, parts or labour.
- The warranty excludes castors, components and parts that are subject to reasonable wear and tear.
- The warranty excludes faults that have been identified as misuse or user error.
- Unauthorized changes or modifications will forfeit your warranty.
- If a defect or fault is discovered, the Hero Medical dealer of where the seat walker was purchased should be contacted immediately.



Independent Living Specialists
Unit 1, 12 Mars Road
Lane Cove West NSW 2066
1300 008 267 | ilsau.com.au

©2024 HERO MEDICAL. INDEPENDENT LIVING SPECIALISTS