


## User Manual

### Royale Medical Premium Shower Chair

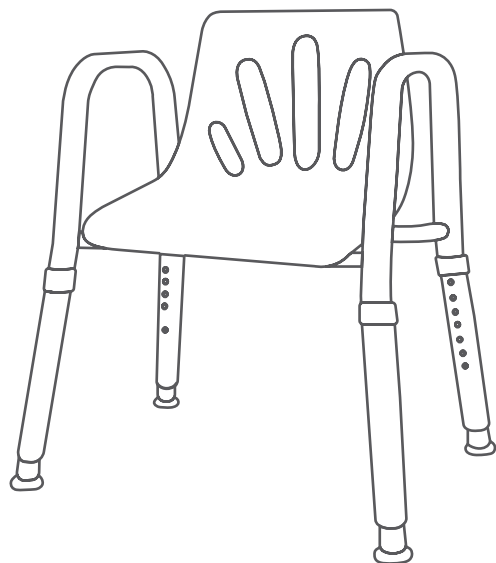

SKU	S11188 Standard	S11964 Extra Wide
Overall Width	68cm	77cm
Overall Depth	52cm	52cm
Seat Width	44cm	44cm
Arm Width	50cm	56cm
Seat Height	40-55cm	40-55cm
Armrest Height	29cm	29cm
Weight	4.7kg	4.7kg
Safe Work Load (SWL)	280kg	280kg

## Safety & Precautions

**User Safety:** It is recommended that the below safety checks are completed prior to each use:

- All locking pins must be fully protruding.
- Adjusted to appropriate height for user.
- Legs are in a secure position with anti-slip rubber tips attached.
- Weight on chair does not exceed safe work load.

**Weight Capacity:** the maximum safety weight limit for the product is 280kg. If the maximum safety weight limit is exceeded, this can increase the risk of damage to the product and/or risk of serious injury to the user and others.

## Instructions for Use

### Product Assembly

1. Remove contents from packaging.
2. Place seat on top of main frame.
3. Insert screw through main frame and line up with seat hole.
4. Half tighten screws into seat, starting with front right and rear left.
5. Tighten all four screws, ensuring seat is stable and secure.
6. Attach all four legs by sliding leg component into main frame receiver until the locking pin is secured and protruding at the appropriate height for the user.
7. Ensure legs are stable and secure.

### Adjusting the Height

1. Push locking pin in and slide internal leg up or down to the appropriate height. Ensure all locking pins are fully protruding prior to use.
2. Please note that one leg has additional holes, to assist with stabilising product on uneven floors.

## Product Care & Maintenance

- It is recommended that the product is inspected regularly for damage, corrosion or rust.
- If there are visible signs of damage, please contact your authorised dealer for replacement parts.
- The product should be cleaned regularly using warm water or a mild detergent. Use a soft cleaning implement, such as a soft sponge or dish cloth, as abrasive cleaning tools could damage the product.
- All replacement parts as well as after-sales support and repair services are available through Hero Medical dealers.

## Recycling & Disposing

- Please follow the local government rules regarding proper recycling and disposal. It should always be done through an authorised agent or licensed place of disposal. Please review list of product part and packaging materials to ensure they are recycled or disposed of correctly.
- Frame: Aluminium
- Seat: Plastic
- Anti-slip tips: Rubber
- Packaging: Cardboard box and plastic

## Labelling Information

Product label is located on the front of the frame. Information on the label includes: company name, company address, product code, product description, safe work load, serial number (which includes the date of manufacture and the batch serial number).

## Warranty

Product comes with 12 months warranty from the date of purchase on structural integrity:

During the warranty period any parts that have become defective due to faulty workmanship or material will be repaired or replaced without charge by a Hero Medical dealer of where the product was purchased.

The warranty is for normal domestic use only and excludes any defect or injury caused by or resulting from misuse, abuse, neglect, accidental damage, or any other alteration that affects the reliability or performance of the unit, not attributable to faulty manufacture, parts or labour.

The warranty excludes castors, components and parts that are subject to reasonable wear and tear.

The warranty excludes faults that have been identified as misuse or user error.

Unauthorised changes or modifications will forfeit your warranty.

If a defect or fault is discovered, the Hero Medical dealer of where the product was purchased should be contacted immediately.

Independent Living Specialists  
 Unit 1, 12 Mars Road, Lane Cove West NSW 2066  
 1300 008 267 | ilsau.com.au