


Assembly Guide

Seat Walker Holder – Scooter Accessory

S1839 – Scooter Accessory – Seat Walker Holder for Scooters (ILS ADL)

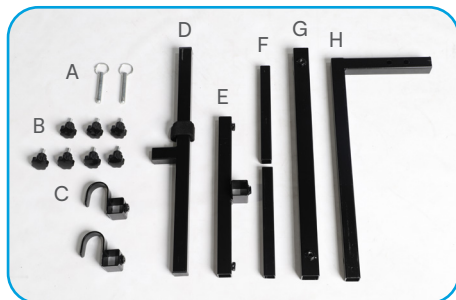

Included in the Box

- A. Metal pins
- B. Screws
- C. Hanging clips
- D. Large T shaped bar
- E. Small T shaped bar
- F. Small extension bars
- G. Large extension bar
- H. L shaped bar



- 1 Place L shaped bar in rear receiver of scooter and place metal pin and/or screw in position.



- 2 Place the small T shaped bar through the L shaped bar.



- 3 Slide the small T shaped bar to the very bottom of the L shaped bar and tighten screw into position.



- 4 Place small extension bars at both ends of the T shaped bar and tighten with screws.

Assembly Guide

Seat Walker Holder – Scooter Accessory

S1839 – Scooter Accessory – Seat Walker Holder for Scooters (ILS ADL)




- 5 Install long extension tube onto the top of the L shaped bar and fix into position with screw and metal pin.



- 6 Place this T bar on top of the long extension tube and insert screw.



- 7 Screw a hanging clip into position on the left and right side, facing outwards at the end of the small T shaped bar.

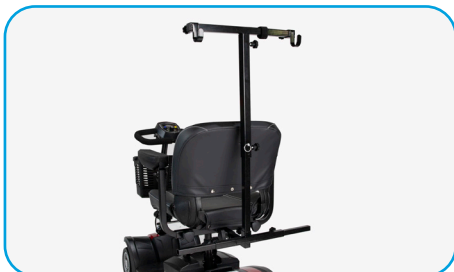

- 8 See fully assembled seat walker holder.



- 9 Lift and place seat walker on the holder via the backrest (models may vary, best to check prior to purchasing).



- 10 Secure tightly to the holder with Velcro straps provided.