




Hero Medical Bedside Commode  
**User Manual**

## Contents

Section	Title	Page
1	Introduction	2
2	Safety & Precautions	3
3	Features	4
4	Specifications	4
5	Instructions For Use	5
6	Labelling Information	5
7	Product Care & Maintenance	6
8	Recycle & Disposal	6
9	Warranty	7

No part of this user manual may be copied, distributed, reproduced, duplicated, scanned or stored whether in whole or part in any form or by any means without the expressed written consent from the officers of Independent Living Specialists Pty Ltd. Independent Living Specialists does not accept responsibility for any changes, errors or omissions. All information is subject to change without prior notice and products may vary from that of the actual images depicted.

# 1 Introduction

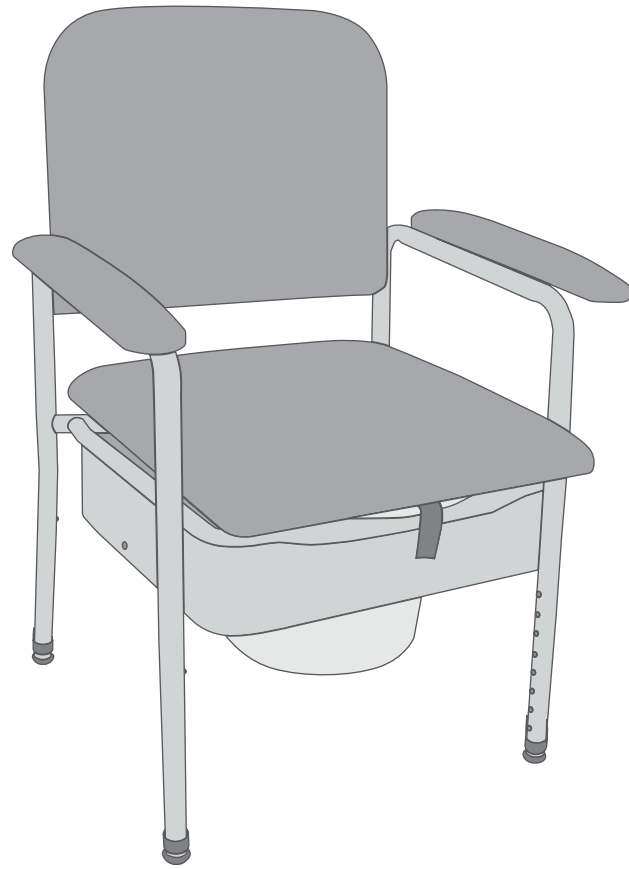

S10313 - Bedside Commode - with Pan Deluxe Padded Champagne Standard 44cm 150kg (Hero Medical)

S10315 - Bedside Commode - with Pan Deluxe Padded Slate Standard 44cm 150kg (Hero Medical)

S307429 - Bedside Commode - with Pan Deluxe Padded Champagne 52cm 180kg (Hero Medical)

S307430 - Bedside Commode - with Pan Deluxe Padded Champagne 60cm 205kg (Hero Medical)

Please follow all user safety guidelines and take additional care while using the bathroom. To ensure user safety, it is highly recommended that the below safety checks are completed prior to each use:

- All push buttons must be fully protruding and checked
- The bedside commode must be setup to the correct height for the user
- Heavy and/or uneven user weight distribution may cause stress to the frame
- Ensure you are seated in a central, even position
- Ensure lid and seat are securely attached to bedside commode
- Ensure that toilet paper is easily reached and other necessary items are conveniently placed

# 2 Safety & Precautions

## Weight Capacity

If the maximum safety weight limit is exceeded, this can increase the risk of damage to the bedside commode and/or risk of serious injury to the user and others.

## Stability

Stability is very important for the users safety whilst using the bedside commode. Ensure all four legs are in a secure, stable position. Ensure all four legs have there anti-slip rubber tips attached. Ensure height of the bedside commode is suitable for the user and that all height adjusting buttons are securely locked into position.

## Intended Use

The bedside commode is a toilet with no plumbing. Designed to be kept near a bed for people with limited mobility or incontinence to provide convenience. Individuals use the pan/bowl and waste is emptied into the toilet.

## 3 Features

- ▶ Padded and comfortable seat, backrest and armrests
- ▶ Sliding padded seat
- ▶ Easy access to removable bowl and lid
- ▶ Height adjustable seat

## 4 Specifications

Name	Champagne 44cm	Slate 44cm	Champagne 52cm	Champagne 60cm
SKU	S10313	S10315	S307429	S307430
Overall Height (min-max)	86-103cm	86-103cm	86-103cm	86-103cm
Overall Width	51cm	51cm	59cm	67cm
Overall Depth	47cm	47cm	47cm	47cm
Width Between Arms	44cm	44cm	52cm	60cm
Seat Height (min-max)	47-64cm	47-64cm	47-64cm	47-64cm
Net Weight	13kg	13kg	13kg	13kg
Safe Working Load (SWL)	150kg	150kg	180kg	205kg

## Instructions for Use 5

### Adjusting the Height

- To adjust the leg length, push the button in and slide the internal leg piece up or down to the appropriate notch.
- All push buttons must be fully protruding and checked before use.
- Your Occupational Therapist can advise you regarding the appropriate height.


### Attaching the Backrest

1. Line up the backrest with the brackets.
2. Insert provided screws into the backrest.

## Labelling Information 6

### a. Product Label

The bedside commode has a product label on the front of the frame. The product label includes the following information:

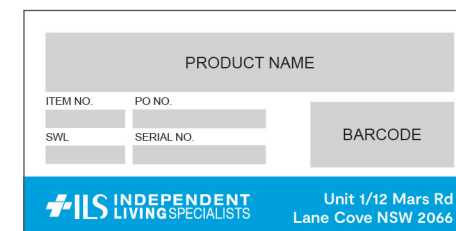

- Name and address of the company
- Name and code for the product
- Safe Working Load (SWL)
- Serial number of the product (which includes the date of manufacture and the batch serial number)

## 7 Product Care & Maintenance

- It is recommended that the Bedside Commode is inspected regularly for signs of wear and tear.
- Damaged items should be withdrawn from use until rectified.
- Should the equipment appear damaged, corroded or rusted – do not use. It may require replacement.
- Recommended to be cleaned after every use, ensure you wear protective gloves while emptying the pan.
- Recommended using a mild detergent and dry clean cloth to avoid damaging components.
- Only clean your bedside commode with soft cleaning implements such as soft sponges, dish clothes and soft-bristled brushes. Warning, using sharp cleaning tools or similarly abrasive cleaning products could cause damage to your bedside commode.
- All replacement parts as well as after-sales support and repair services are available through Hero Medical dealers.

## 8 Recycle & Disposal

Please follow the local government rules regarding proper recycling and disposal. It should always be done through an authorised agent or licensed place of disposal. Please review the list of bedside commode parts and packaging materials to ensure they are recycled or disposed of correctly.

- Frame: Steel
- Seat, lid and splashguard: Plastic
- Padded seat, backrest and armrests: Fabric and vinyl upholstery
- Anti-slip tips: Rubber
- Packaging: Cardboard box and plastic

## Warranty 9

The bedside commode comes with 12 months warranty from the date of purchase on structural integrity.

- During the warranty period any parts that have become defective due to faulty workmanship or material will be repaired or replaced without charge by a Hero Medical dealer of where the product was purchased.
- The warranty is for indoor use only and excludes any defect or injury caused by or resulting from misuse, abuse, neglect, accidental damage, or any other alteration that affects the reliability or performance of the unit, not attributable to manufacturing faults, parts or labour.
- The warranty excludes faults that have been identified as misuse or user error.
- Unauthorised changes or modifications will forfeit your warranty.
- If a defect or fault is discovered, the Hero Medical dealer of where the bedside commode was purchased should be contacted immediately.



Independent Living Specialists  
Unit 1, 12 Mars Road  
Lane Cove West NSW 2066

1300 008 267 | [ilsau.com.au](http://ilsau.com.au)

©2024 HERO MEDICAL. INDEPENDENT LIVING SPECIALISTS

PDT038-0824