

# HERO

## User Manual

### Hero Medical Bariatric 3 in 1 Commode



SKU	S8270	S9826
Overall Height (min-max)	77cm-89cm	76cm-86cm
Overall Width	69cm	71cm
Overall Depth	56cm	52cm
Width Between Arms	60cm	65cm
Seat Height (min-max)	43cm-55cm	50cm-60cm
Unit Weight	9kg	11.5kg
Weight Capacity/Max SWL	225kg	225kg

## Safety & Precautions

**User Safety:** Please follow all user safety guidelines and take additional care while using the bathroom. To ensure user safety, it is highly recommended that the below safety checks are completed prior to each use:

- All push buttons must be fully protruding and checked.
- The 3 in 1 commode must be setup to the correct height for the user
- Heavy and/or uneven user weight distribution may cause stress to the frame.
- Ensure you are seated in a central, even position.
- Ensure seat is securely attached to the 3 in 1 commode.
- Ensure that toilet paper is easily reached and other necessary items are conveniently placed.


**Weight Capacity:** The maximum safety weight limit for the Hero 3 in 1 Commode is 225Kg. If the maximum safety weight limit is exceeded, this can increase the risk of damage to the bedside commode and/or risk of serious injury to the user and others.

**Stability:** Stability is very important for the users safety whilst using the 3 in 1 commode. Ensure all four legs are in a secure, stable position. Ensure all four legs have there anti-slip rubber tips attached. Ensure height of the 3 in 1 commode is suitable for the user and that all height adjusting buttons are securely locked into position.

**Intended Use:** The 3 in 1 commode can be used in various ways; as a shower stool, over toilet aid and as a portable commode. Individuals can use it as a versatile unit besides the bed, using the removable bucket and lid.

## Labelling Information

The 3-in-1 commode has a product label on the front of the frame. The product label includes the following information:

PRODUCT NAME	
ITEM NO.	PO NO.
SWL	SERIAL NO.
BARCODE	
	
Unit 1/12 Mars Rd Lane Cove NSW 2066	

- Name and address of the company
- Name and code for the product
- Safe Working Load (SWL)
- Serial number of the product (which includes the date of manufacture and the batch serial number)

## Features

- Heavy-duty/extra wide
- 3-in 1 functions
- Highly durable
- Height adjustable seat

## Instructions for Use

### Product Assembly

1. Remove contents from packaging.
2. Clip the toilet seat to the frame
3. Attach all four legs by sliding leg component into main frame receiver until the locking pin is secured and protruding at the appropriate height for the user.
4. Ensure legs are stable and secure.
5. Attach the backrest using provided screw and wingnut (*specific for S8270*).
6. Attach the backrest by sliding into main frame receiver until the locking pin is secured and protruding (*specific for S9826*).

### Adjusting the Height

1. To adjust the leg length, push the button in and slide the internal leg piece up or down to the appropriate notch.
2. All push buttons must be fully protruding and checked before use.
3. Your Occupational Therapist can advise you regarding the appropriate height.

## Product Care & Maintenance

- It is recommended that the 3 in 1 Commode is inspected regularly for signs of wear and tear.
- Damaged items should be withdrawn from use until rectified.
- Should the equipment appear damaged, corroded or rusted – do not use. It may require replacement.
- Recommended to be cleaned after every use, ensure you wear protective gloves while emptying the pan
- Recommended using a mild detergent and dry clean cloth to avoid damaging components
- Only clean your bedside commode with soft cleaning implements such as soft sponges, dish clothes and soft-bristled brushes. Warning, using sharp cleaning tools or similarly abrasive cleaning products could cause damage to your bedside commode.
- All replacement parts as well as after-sales support and repair services are available through Hero Medical dealers

## Recycling & Disposing

Please follow the local government rules regarding proper recycling and disposal. It should always be done through an authorised agent or licensed place of disposal. Please review the list of bedside commode parts and packaging materials to ensure they are recycled or disposed of correctly.

- Frame: Steel
- Seat, backrest, armrests, bowl and lid: Plastic
- Anti-slip tips: Rubber
- Packaging: Cardboard box and plastic

## Warranty

The 3 in 1 commode comes with 12 months warranty from the date of purchase on structural integrity.

- During the warranty period any parts that have become defective due to faulty workmanship or material will be repaired or replaced without charge by a Hero Medical dealer of where the product was purchased.
- The warranty is for normal domestic use only and excludes any defect or injury caused by or resulting from misuse, abuse, neglect, accidental damage, or any other alteration that affects the reliability or performance of the unit, not attributable to faulty manufacture, parts or labour.
- The warranty excludes faults that have been identified as misuse or user error.
- Unauthorized changes or modifications will forfeit your warranty.
- If a defect or fault is discovered, the Hero Medical dealer of where the 3 in 1 commode was purchased should be contacted immediately.

Independent Living Specialists  
Unit 1, 12 Mars Road, Lane Cove West NSW 2066  
1300 008 267 | ilsau.com.au