



HERO+

## User Manual - S10360

---

Toilet Seat Raiser  
190kg - 10cm



## Toilet Seat Raiser - Savannah 10cm With Lid (ILS) - Hero Medical



### Instructions for Assembly

1. Remove contents from packaging
2. Screw in clamps to the toilet seat raiser but don't tighten
3. Depending on the toilet you may need to remove the seat and lid of the toilet
4. Place the raiser over the bowl and tighten screws until raiser is secure

## How to use the Toilet Seat Raiser

When sitting on the stool, move back until the seat is behind your legs, then slowly lower yourself on the seat.

When getting off the raiser, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Stand leading with your head and shoulders.



**SWL : 190kg**

- This product is designed to be used daily in the home or community setting