



## **User Manual - S11189**

---

Premium Aluminium  
Shower Stool  
250kg



**Shower Stool - Premium Aluminium Standard  
Width 250kg (No Back) - Royale Medical**



**Instructions for Assembly**

1. Remove contents from packaging
2. Attach all four legs onto the side supports by using the leg pin to clip into position

## **How to use the Shower Stool**

When sitting on the stool, move back until the seat is behind your legs, firmly place each hand on the armrest, if appropriate, to ensure your body is supported and the stool is stable. Then slowly lower yourself down.

When getting off the stool, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Place your hand on the armrests, if appropriate, to push yourself into standing leading with your head and shoulders.

## **Adjusting the Height**

To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height.



**SWL : 250kg**

- This product is designed to be used daily in the home or community setting