



## **User Manual - S11965**

---

Premium Bariatric  
Over Toilet Aid  
250kg



**Over Toilet Aid - Premium Bariatric 56cm  
280kg (Includes Splash Guard) - Royale Medical**



**Instructions for Assembly**

1. Remove contents from packaging
2. Attach all four legs onto the side supports by using the leg pin to clip into position
3. Insert splash guard in centre frame, underneath the seat

## **How to use the Over Toilet Aid**

Place the Over Toilet Aid over the toilet and ensure that the legs are adjusted so that there is no movement of the legs.

When sitting on the aid, move back until the seat is behind your legs, firmly place each hand on the armrest, if appropriate, to ensure your body is supported and the aid is stable. Then slowly lower yourself down.

When getting off the aid, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Place your hand on the armrests, if appropriate, to push yourself into standing leading with your head and shoulders.

## **Adjusting the Height**

To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height.



**SWL : 280kg**

- This product is designed to be used daily in the home or community setting