




HERO+

User Manual - S12066

Bed Rail with
Adjustable Height
145kg

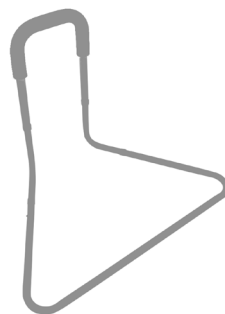

Bed Rail with Adjustable Height Hero Medical 130kg



Instructions for Assembly

1. Remove contents from packaging
2. Place receiver poles into base hole as shown in photos



How to use the Adjustable Bed Rail

This product is designed to be used in conjunction with a bed for as a stability support and the aid in the safety from bed falls

Adjusting the Height

Using the pin, adjust the height of the rail



SWL : 145kg

- This product is designed to be used daily in the home or community setting