



HERO+

User Manual - S13166

Extra Wide
Over Toilet Aid
205kg



Over Toilet Aid - Extra Wide 53cm 205kg
- Hero Medical Aluminium (with Lid)
(Includes Splash Guard)



Instructions for Assembly

1. Remove contents from packaging
2. Attach all four legs onto the side supports by using the leg pin to clip into position
3. Insert splash guard in centre frame, underneath the seat

How to use the Over Toilet Aid

Place the Over Toilet Aid over the toilet and ensure that the legs are adjusted so that there is no movement of the legs.

When sitting on the aid, move back until the seat is behind your legs, firmly place each hand on the armrest, if appropriate, to ensure your body is supported and the aid is stable. Then slowly lower yourself down.

When getting off the aid, bring your bottom to the front of the seat ensuring your feet are firmly placed on the floor. Place your hand on the armrests, if appropriate, to push yourself into standing leading with your head and shoulders.

Adjusting the Height

To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height. One leg has extra holes, which can be adjusted to help stabilize the Over Toilet Aid for uneven surfaces.



SWL : 205kg

- This product is designed to be used daily in the home or community setting