




ROYALE

User Manual - S13279

Transfer Bench –
Sliding + Swivel Bench
100kg

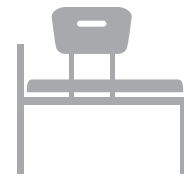

**Transfer Bench – Sliding + Swivel Bench
100kg - Royale Medical**



Instructions for Assembly

1. Remove contents from packaging
2. Attach 2 L brackets to the main frame
3. attach legs into the L bracket
4. Attach rubber feet onto the bench
5. Place backrest in the two holes on the rear of the bench and use pin to secure into the bench (Ensure the arm rest is on the right-hand side)

How to use the Transfer Bench

Place the two legs of the bench on the handled side into the bath.

When transferring onto the bench, stand with the back of your legs against the side of the bench as close to the bath as possible, and firmly place each hand on the arm rests, if appropriate, to ensure your body is supported and the chair is stable. Then slowly lower yourself down to sit on the edge of the seat.

Using the support of the transfer bench handle, turn your body and slowly lift your legs over the bath walls until both legs are in the bath, this can be done by lifting one leg at a time. Use the transfer bench handle and the front of the seat to slide yourself across using the front lever until you are seated in the middle of the bath.

When getting off the bench, using the arm rest and front of seat, slide across away from the bath. Slowly lift legs over the bath wall and position yourself on the edge of the bench with your legs close to the bath wall.

Bring the seat to an outer position

Bring your bottom to the front of the seat ensuring your feet are firmly placed flat on the floor. Place your hands on the seat of the transfer bench, if appropriate, to push yourself into standing leading with your head and shoulders.

Adjusting the Height


To adjust the leg length, press the button in and slide the leg piece up or down to the appropriate notch. Your Occupational Therapist can best advise regarding the appropriate height. One leg has extra holes, which can be adjusted to help stabilize the shower chair for uneven surfaces.

Press the button to turn



Adjusting the Height



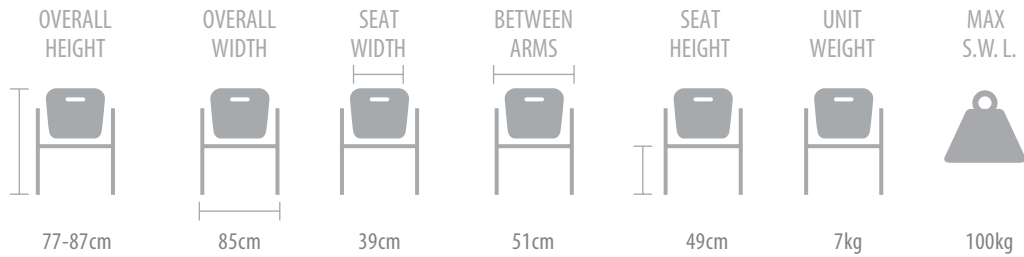

- This product is designed to be used daily in the home or community setting

Warranty Information

- This product comes with a 12-month comprehensive warranty

Service Life of Product

- With proper care and maintenance this product has a life expectancy of 8 years