


USER'S MANUAL

Overbed Table

S14681

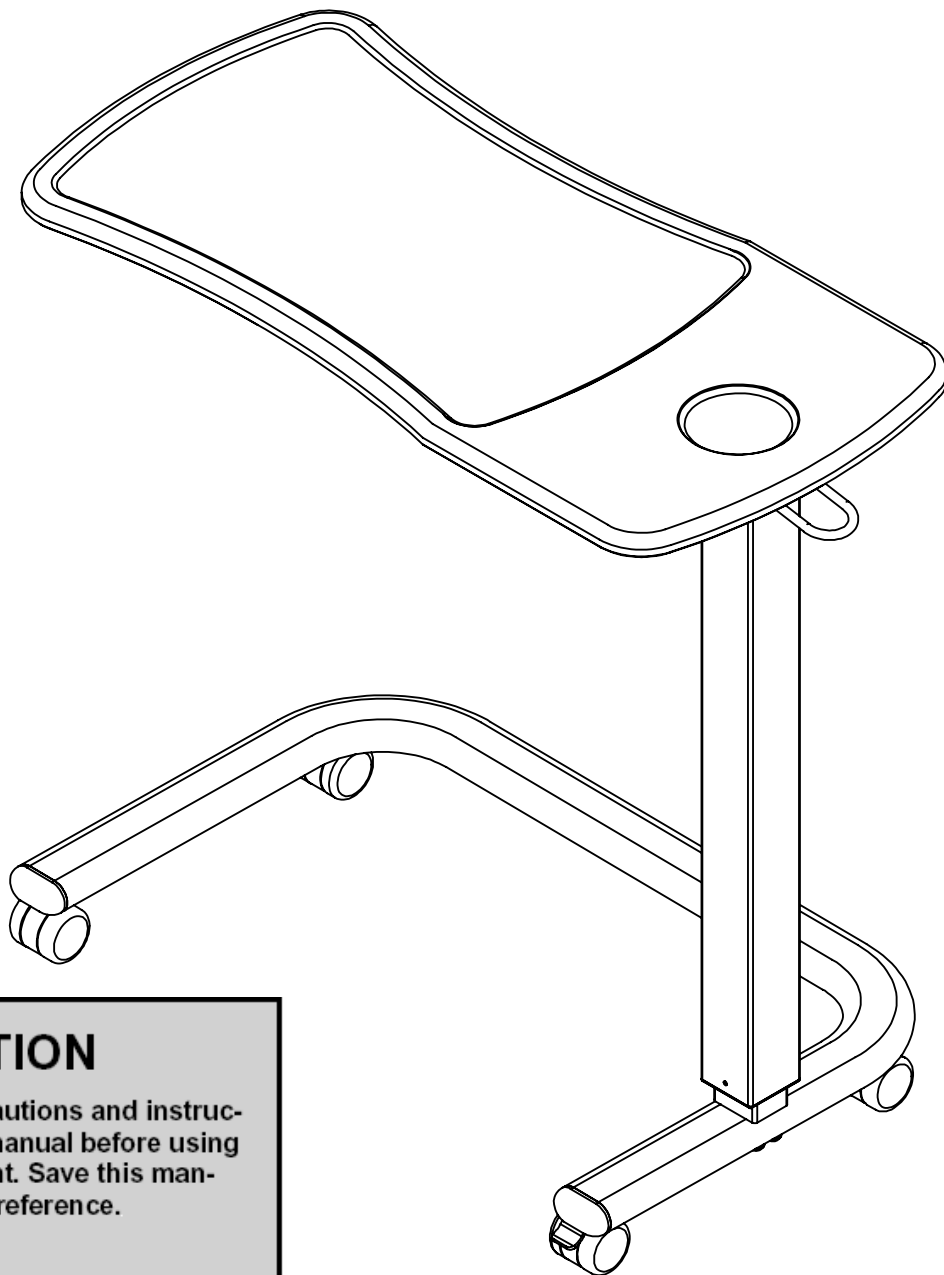

⚠ CAUTION

Read all precautions and instructions in this manual before using this equipment. Save this manual for future reference.

IMPORTANT PRECAUTIONS

WARNING: To reduce the risk of burns, fire, electric shock, or injury to persons, read the following important precautions and information before operating the Overbed Table.

1. It is the responsibility of the owner to ensure that all users of this Overbed Table are adequately informed of all warnings and precautions.
2. Use the Overbed Table only as described in this manual.
3. Place the Overbed Table on a level surface.
4. Keep the Overbed Table indoors, away from moisture and dust. Do not put the Overbed Table in a garage or covered patio, or near water.
5. This Overbed Table utilizes a high-tension air Pneumatic cylinder loaded to raise the table. Do not remove shipping Block Pin from column assembly prior to assembly. The column assembly may cause injury.
6. Check for any obvious damage to the carton or its contents. If damage is evident, DO NOT USE. Contact carrier / distributor for further instruction.
7. Inspect and tighten all parts of the Overbed Table regularly.
8. Never insert or drop any object into any opening.
9. This Overbed Table is intended for in-home use only. Do not use this Overbed Table in any commercial, rental, or institutional setting.

WARNING: *Ensure that the table is assembled as described, and that all components are securely attached, before use.*


SAVE THESE INSTRUCTIONS

BEFORE YOU BEGIN

Thank you for selecting the revolutionary **HP1034 Overbed Table**. The **HP1034** offers an impressive array of features to make your home life more enjoyable and effective.


For your benefit, read this manual carefully before using the Overbed Table. If you have additional questions, please call our Customer Service Department To

help us assist you, please note the product model number and serial number before calling. The model number of the **Overbed Table** is **HP1034**. The serial number can be found on a decal attached to the Overbed Table. Before reading further, please familiarize yourself with the parts that are labeled in the drawing below.

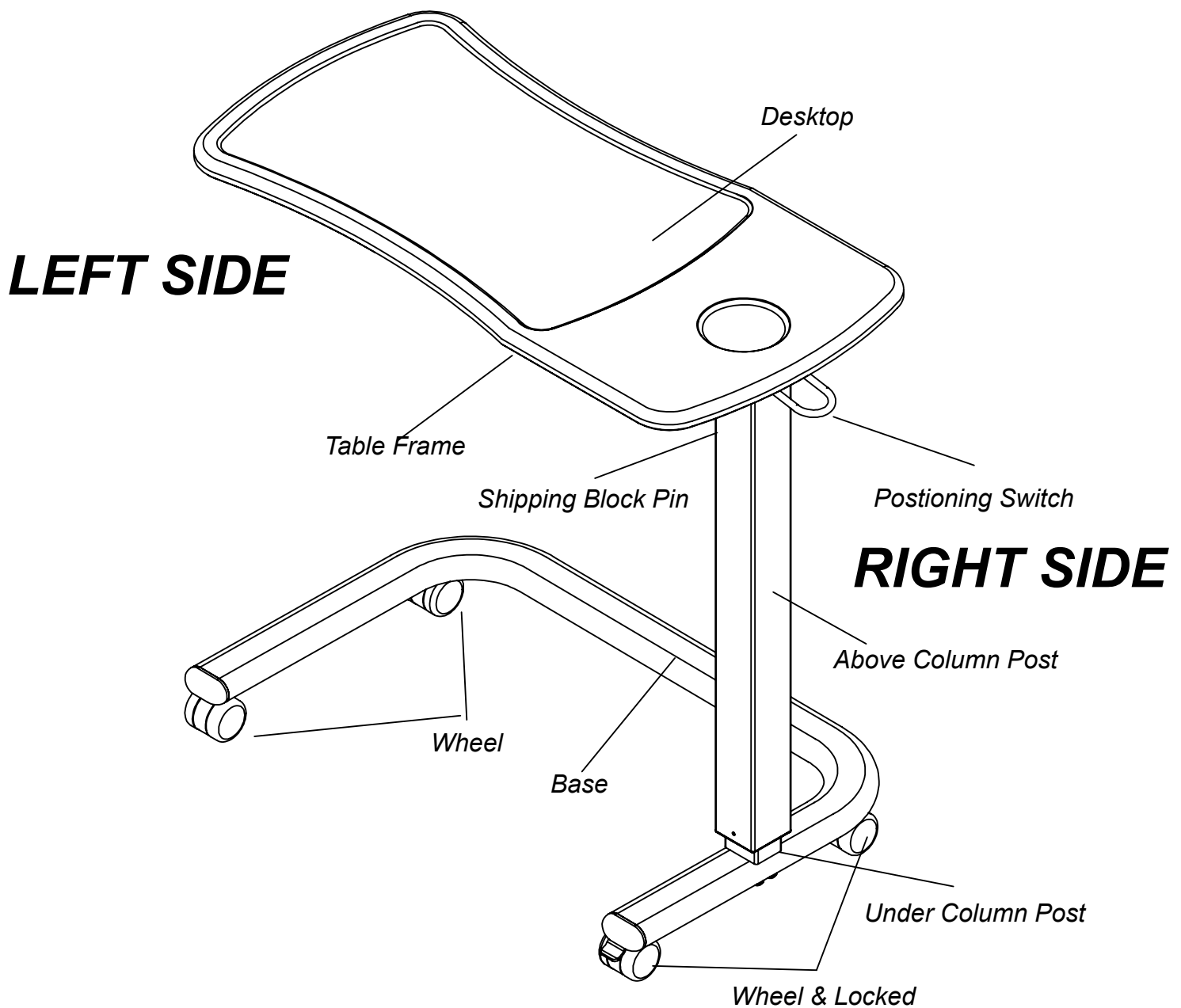

ASSEMBLY


Set the Overbed Table in a cleared area and remove all packing materials. Do not dispose of the packing materials until assembly is completed.

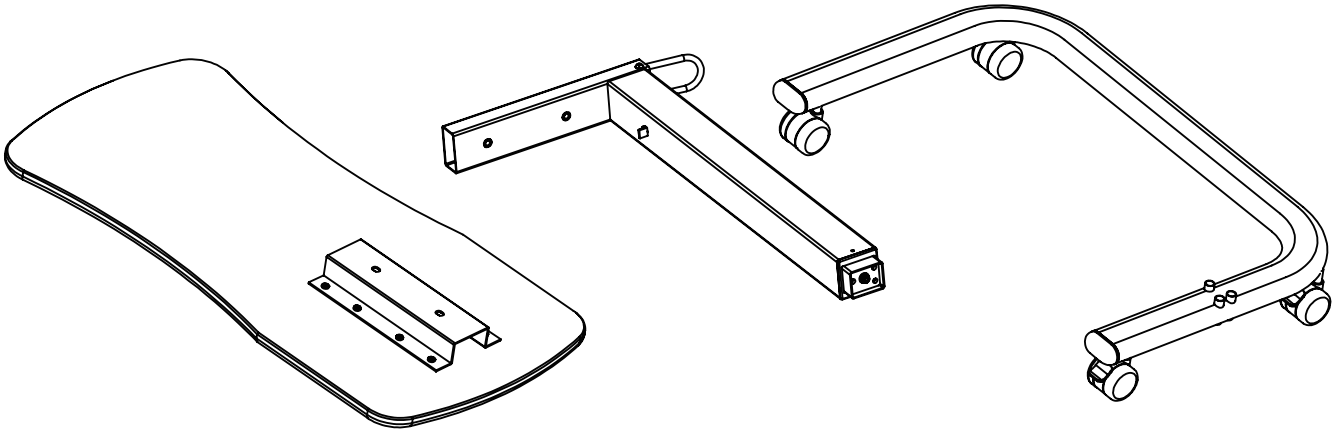

Unpacking your **Overbed Table** where will be using it. Being careful Not to damage . Place your **Overbed Table** on a level flat surface. It is recommended that you place a protective covering on your floor.


STEP 1 Place table top upside down on a level surface as show at right.

Position column bracket above table top bracket as shown at right, with holes matched, and lower column onto table top. Insert socket head cap bolts through column bracket into table top bracket as shown. Use M8 hex key to tighten the bolts.

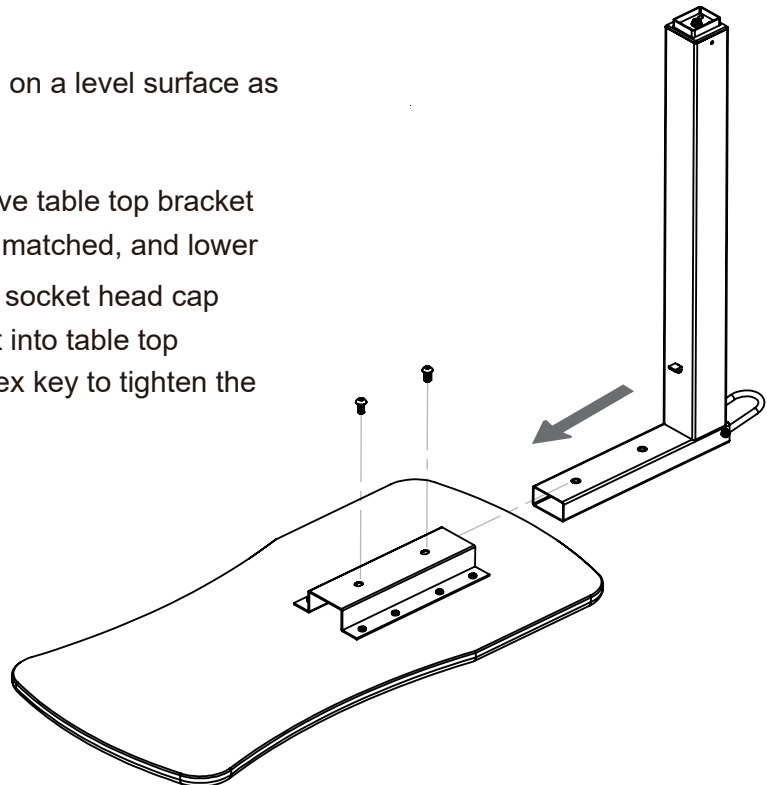


WARNING:

Do not remove shipping Block Pin from column assembly prior to assembly.

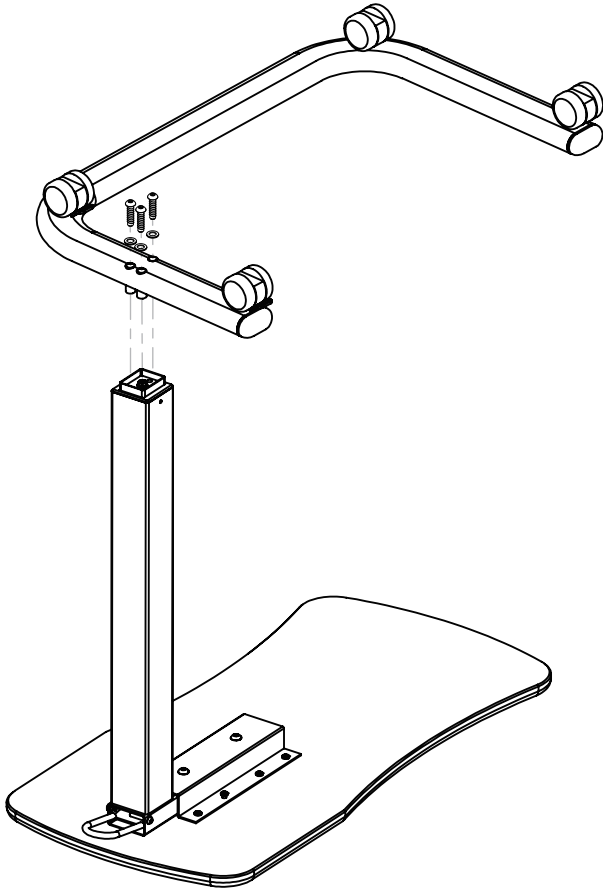


STEP 2 Place the base upside down above the column as shown at left. Install the 3 bolts through the washer, bolt plate, base, and into column end as shown.

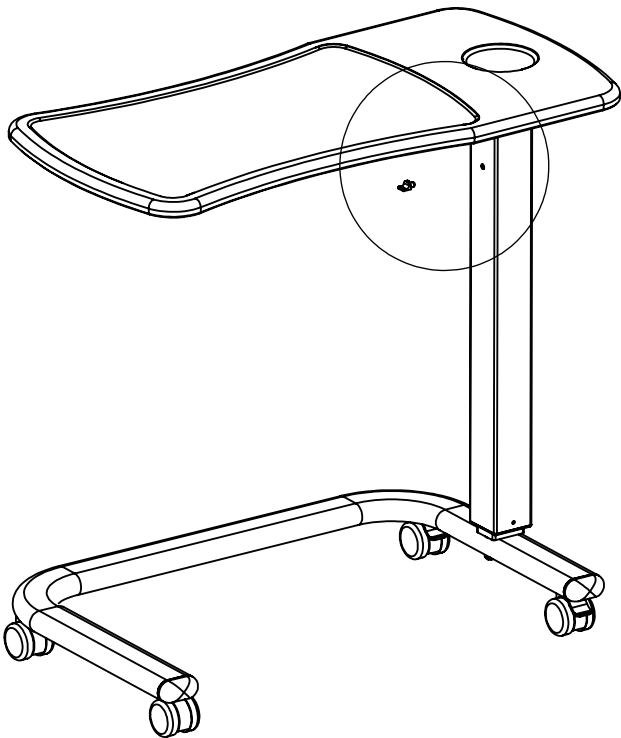

STEP 3 Turn the Table UPRIGHT and place it on the floor.

Remove shipping Block Pin from column assembly .

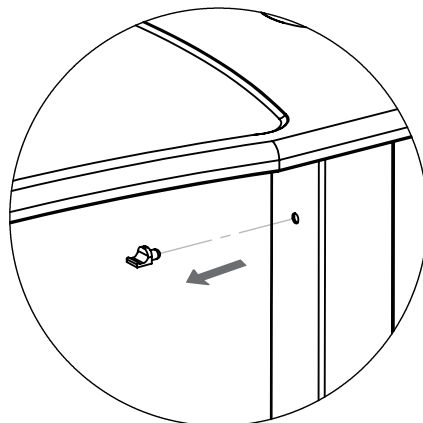

Make sure that all parts are tightened before you use the Overbed Table.

Adjusting the Hi -Low of Overbed Table

Note : The Overbed table MUST be in the upright position before Hi-Low adjusting.

Table top up - Pull the table top up.

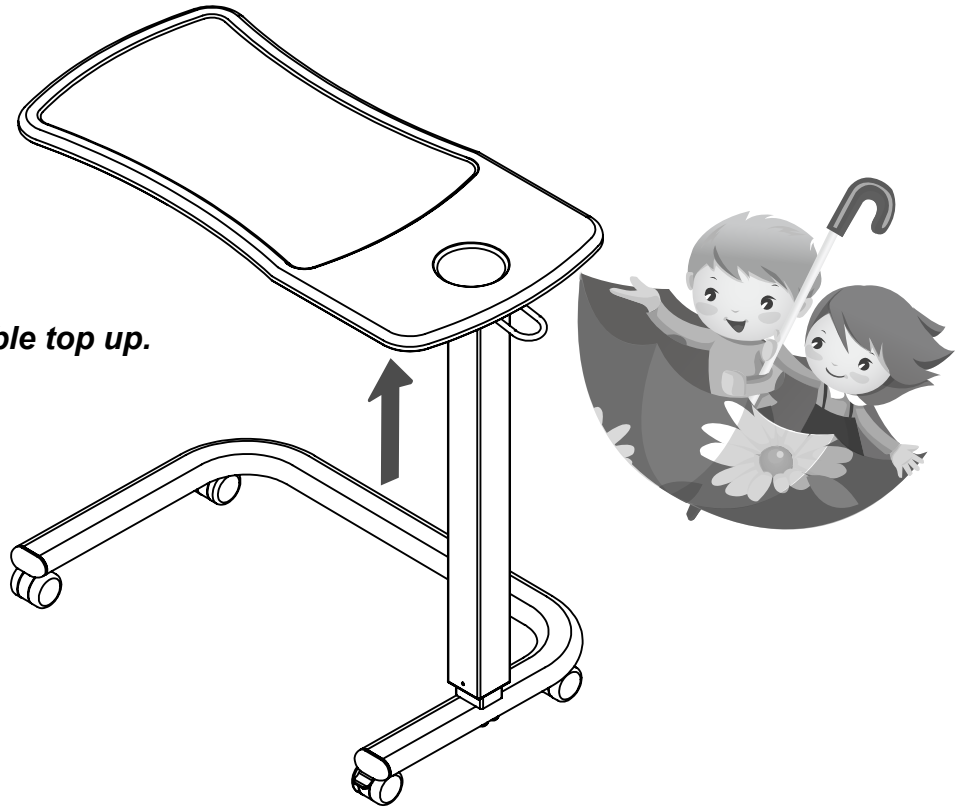


Table top down

-lift up the switch and push the table top down

