



...reducing avoidable harms

# QUATTRO<sup>®</sup> Sentinel Bariatric Mattress System



## User Guidelines





# Explanation of Label Symbols and Statements



Caution



Refer to instructions of use / booklet



Medical Devices Directive  
93/42/EEC



North America ETL listed



Class II Equipment (Double Insulated)



Do not dispose of with the normal household waste (*please refer to [www.talleygroup.com](http://www.talleygroup.com) for further details*)



Manufacturer



Date of Manufacture



Fragile, handle with care



Keep dry



Protect from heat and radioactive sources



Temperature limitation



Humidity limitation



Atmospheric pressure limitation



Suitable for connection to type BF applied parts

## WARNING

This is a statement that alerts the user to the possibility of serious injury or other adverse reactions with the use or misuse of the device

## CAUTION

This is a statement that alerts the user to the possibility of a problem with the system associated with its use or misuse



Operating Instructions

**IP:** Ingress Protection

**2:** Protection against fingers or other object not greater than 80mm in length and 12mm in diameter

**1:** Protection from vertically dripping water

**IP21**



# Contents

	Page
EXPLANATION OF LABELS SYMBOLS AND STATEMENTS	1
INTRODUCTION	2
CAUTIONS AND WARNINGS INFORMATION	3
INSTALLATION AND USER GUIDELINES	5
CARE AND MAINTENANCE	8
FAULT FINDING	11
SPECIFICATIONS	13
EMI / EMC STATEMENT	14
EMC MANUFACTURER'S DECLARATION	BACK COVER

## Introduction

Thank you for choosing to use the Talley QUATTRO® SENTINEL bariatric mattress replacement system. The QUATTRO® SENTINEL is suitable for the prevention and treatment of pressure ulcers within all care environments, and is specifically designed for the care of bariatric patients. Clinically proven to provide pressure relief, lower average interface pressures and enhanced comfort, the QUATTRO® SENTINEL will accommodate users weighing up to 58 stone. Ideal for use with width-retractable bariatric bed frames, the QUATTRO® SENTINEL mattress can be reduced in width by the simple removal of the side bolsters, allowing easy movement of the complete bed system through doorways, lifts, corridors, etc, with the patient in-situ.

The QUATTRO® SENTINEL system will benefit from careful installation and use, providing a long and effective service life. Please read this user manual in order to achieve the best possible results.



# Cautions and Warnings



## CONTRAINDICATIONS FOR USE

Alternating pressure therapy should not be used for patients with unstable fractures, gross oedema, burns or an intolerance to motion.

- There are no special skills required to operate the system.
- The medical professional is responsible for applying his/her best medical judgment when using this system.
- Select correct setting for therapy required. Care should be taken not to accidentally change pressures once set as the efficiency of the therapy may be reduced. This could also be caused by pets, pests or children.
- The electricity supply is of the type indicated on the power unit
- Check the mains lead is free from damage and is positioned so as not to cause an obstruction, or injury, e.g. strangulation.
- Ensure the mains lead or pump cannot become trapped or crushed, e.g. via raising or lowering of bed or bed rails or any other moving object.
- The power unit must only be used with a suitable approved cord and plug set as supplied by Talley.
- The system is not used in the presence of flammable anesthetics
- Suitable for continuous use.
- Not suitable for sterilisation.
- Do not position the power unit to make it difficult to disconnect the power supply, plug or garments.
- Do not place device on or near a heat source.
- Do not use with hot water bottles or electric blankets.
- The materials used in the manufacture of all components of the system comply with the required fire safety regulations.
- Talley advice against smoking whilst the system is in use, to prevent the accidental secondary ignition of associated items which may be flammable, such as bed linen.
- Do not allow sharp objects to penetrate the mattress material.
- Do not modify the mattress or power unit in any way.
- Do not store in damp conditions.
- Not for use in an oxygen enriched environment.
- Not for use in an outdoor environment.
- No part of the medical device should be serviced while it is in use by the patient.



- The medical equipment requires 5 hours to warm from the minimum storage temperature before it is ready for its intended use.
- The medical equipment requires 1 hour to cool from the maximum storage temperature before it is ready for its intended use.
- Intended for home healthcare use and professional healthcare facility environments where operators with medical training are continually available when patients are present.
- The device is intended to be hung over the beds footboard.
- Wireless equipment such as mobile phones should be kept at least 1 foot or 0.3 metres away from the equipment.
- Do not connect to any other medical device or equipment.
- Risk of fire if incorrect fuse used.
- The mattress and pump should be cleaned between patient use, please refer Care and Maintenance section for all warnings and cautions.
- Mattress must be properly set up as directed.
- Check periodically to ensure patient support and comfort, adjusting the Comfort Control setting if desired.
- All hoses must be free of kinks, twists, properly connected and positioned so as not to cause an obstruction or injury.
- In order for alternating air pressure mattress to operate effectively, please avoid placing objects on the surface that may obstruct the movement of air between the cells. For the same reason, please discourage people from sitting on the edge or on the end of the mattress whilst it is in use.
- Do not use abrasive cleaners, phenol disinfectants, solvents or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these may destroy the cover materials.
- The system is used as part of a pressure ulcer prevention program, not solely relied upon for this purpose.
- The above warnings, cautions and any safety considerations should be observed on a routine and regular basis, not only upon installation.



# Installation and User Guidelines

1. Remove any existing mattress from the bed frame, as the QUATTRO® SENTINEL is intended to completely replace the bed mattress.
2. Place the mattress on the bed frame, ensuring the mattress hose exits at the foot end on the right hand side and the coloured cover is uppermost.
3. Secure the mattress to the bed frame using adjustable straps - pass straps beneath or around bed frame mattress platform and secure with the buckles (Fig. 1), allowing for moving components on electric bed frames. This is important as it stops the mattress becoming detached from the bed frame and causing possible injury to the patient. A strap extension kit is available for use with divan beds (part number 11062).

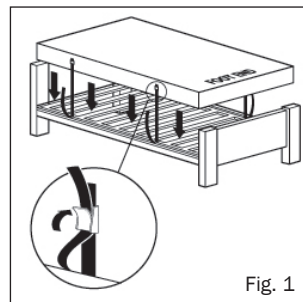


Fig. 1

4. Suspend the power unit from the foot board of the bed having first adjusted the hanger brackets as instructed at the rear of the unit. Alternatively the power unit may be placed on the floor.
5. Attach the air supply hose to the power unit by aligning the black line on the air supply hose connector with the black line on the power unit connector and pushing together. Rotate the mattress hose connector clockwise until the green indicator fills the indicator hole on the power unit connector (Fig. 2).

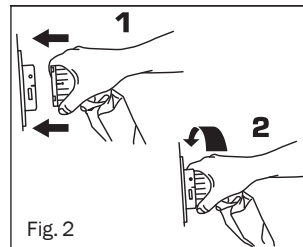


Fig. 2


6. Plug the smaller end of the power lead into the left hand side of the power unit, and the other end into mains outlet in wall. Ensure that the mains lead and tubing cannot become trapped in the bed frame.
7. Switch the power on at the outlet, and at the side of power unit adjacent to the power lead entry.
8. The system will display STARTING, then INITIALISING whilst the mattress inflates (this will take between 15 - 20 minutes).

**Note:** It is important that during the INITIALISING phase the mattress connector is not disconnected from the power unit. If this is done, the power unit must be switched off, MUTE button pressed when the audible alarm is made, the mattress connector re-engaged, and the power unit restarted. If UNCALIBRATED is displayed when switched on, the system will continue to operate but should be recalibrated as soon as possible.

9. When a bottom sheet is added to the mattress, ensure that it is left loose to allow the mattress surface to conform with the patient's body as much as possible.
10. Allow the mattress to inflate before positioning the patient on the mattress.



11. Once inflated the system will automatically default to ACTIVE mode. CONTINUOUS LOW PRESSURE therapy can be selected via the THERAPY MODE button (please see THERAPY MODE on page 6). The comfort setting can be adjusted using the up and down arrow buttons (see COMFORT CONTROL on page 6).

NB. The power unit will automatically lock 2 minutes after last button operation when running to prevent inadvertent operation of button functions (except MUTE), as indicated by  on the display screen. Press and hold the MUTE/UNLOCK button until power unit beeps if further button operation is needed (i.e. change of therapy mode or comfort setting).

12. Place the carry bag and user manual in a safe place for future use.

## OPERATION BUTTONS

The operation buttons on the face of the power unit provide the following functions.



### THERAPY MODE

Pressing the THERAPY MODE button will toggle between ACTIVE (1 in 4 alternating air pressure cycle) and CONTINUOUS LOW PRESSURE therapy modes. The selected therapy mode is shown on the display screen. The default mode is ACTIVE therapy. To switch to CONTINUOUS LOW PRESSURE mode, press THERAPY MODE button for 2 seconds until an audible tone is heard. The system will then display 'PLEASE WAIT' and requires approximately 2 minutes to initialise.



### COMFORT CONTROL



Air pressure is regulated within each of the cells throughout the cycle so that support, posture and therapy are constantly maintained at optimum levels, in response to patient weight, movement and position. Equalisation of cell pressure automatically takes place at each stage of the 1-in-4 cycle, again to ensure precise pressure and therapy is provided. The automatic default comfort setting is medium. However, if the patient prefers a firmer or softer mattress, increase or decrease the comfort control setting accordingly using the UP and DOWN arrow buttons (SOFT/MEDIUM/FIRM). The comfort setting is shown on the display screen. Check periodically to ensure patient support and comfort.




### DATA

Pressing the DATA button at any time switches the display into DATA mode. Use the up and down arrow buttons to scroll through the product data and user information set. Pressing the DATA button again returns the display to the previous mode.

NB. Used for accessing information only, does not affect mode of operation.



### MUTE/UNLOCK

Press to silence the sounder and to clear the message from the display screen. The power unit will automatically lock 2 minutes after the last button operation when running to prevent the inadvertent operation of button functions (except MUTE), as indicated by  on the display screen. Press and hold the MUTE/UNLOCK button until the power unit beeps if further button operation is needed (i.e. comfort setting). The power unit will lock again 2 minutes after the last button operation.

NB. After power failure/switching the power off, pressing MUTE cancels the system's previous settings. When power returns the default setting of ACTIVE mode, MEDIUM comfort setting is invoked. (Note that previous settings are automatically cancelled if the duration between switch off and switch on is greater than 12 seconds. If power returns before a period of 12 seconds has passed and the MUTE button has not been pressed, the system will return to the previous mode of operation.)



### MAX. INFLATE


Necessary for some nursing procedures, the MAX INFLATE mode inflates the mattress to maximum static pressure for a period of 15 minutes. After pressing the MAX INFLATE button to inflate mattress, the system displays 'PLEASE WAIT' followed by 'READY' and a 5 second audible tone when maximum pressure is achieved and 'MAX INFLATE' is shown on the display screen. After 15 minutes the system automatically returns to the Active mode of operation.

## MAXIMUM USER WEIGHT GUIDELINES

370kg (58 stone) max.

### PATIENT TRANSPORT FACILITY / REMOVABLE SIDE BOLSTERS (Fig. 5 & 6)

Press MAX. INFLATE button to fully inflate the mattress. When maximum pressure is achieved, detach the mattress air supply hose from the power unit by rotating the mattress hose connector anti-clockwise until the black lines align, and pull the mattress hose connector away (Fig. 5). The mattress cells will remain inflated, so supporting the patient. Ideal for use with width-retractable bariatric bed frames,

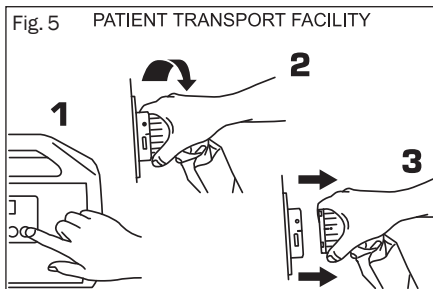








the QUATTRO® SENTINEL mattress can be reduced in width by the simple removal of the side bolsters (Fig. 6). To remove the side bolsters, simply unzip the cover, unfasten the poppers on the side bolster retaining straps either side of the mattress and lift out the side bolsters. This will now allow the width of the bed frame to be retracted, if applicable, allowing easy movement of the complete bed system through doorways, lifts, corridors, etc, with the patient in-situ.

### CPR FACILITY (Fig. 7)

The CPR device is situated at the head end on the right hand side of the mattress (viewed from foot end), as indicated by arrows on the mattress tag. For rapid deflation rotate the dial of the CPR device anti-clockwise to 'click' into the open position . If re-inflating the mattress, make sure the dial of the CPR device is rotated clockwise until it 'clicks' into the closed position .

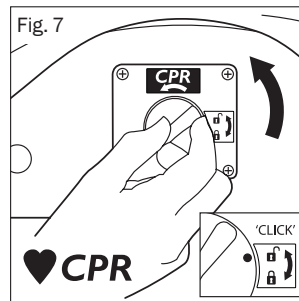

All alternating air pressure mattresses can be used on profiling bed frames, slatted frames, in-filled frames and divans.

Back rests or pillows for support should be placed beneath the mattress to allow uninterrupted body contact with the mattress surface.

Place the bottom sheet loosely on the mattress to allow the mattress surface greater contact with the patient's body. Avoid using fitted sheets. The use of incontinence sheets / excessive bedding beneath the patient may reduce the pressure relieving effect of the mattress.

To remove air from the mattress when dismantling the system, use the CPR facility as described above.

Care should be taken when raising and lowering bed safety side rails in order to avoid possible interference with the CPR mechanism.

A gap of 2.5cm on either side of the mattress should not be exceeded when side rails are deployed.

## Care and Maintenance



### COVER


Always keep the mattress cover as clean as is practicable. The material is waterproof and vapour permeable.

- Inspect top cover for signs of damage or wear which could result in the contamination of the interior, e.g. tears, holes, damage to seams or zips, underside staining, etc. The frequency of these checks should be at each decontamination process, i.e. between patients or patient occupancy (or weekly for longer term patients).



- Care should be taken to avoid puncturing cover with objects such as needles, scalpels, pat slides, acrylic nails, etc.
- The cover may be removed and cleaned in accordance with The Revised Healthcare Cleaning Manual June 2009 subject to the following action: Following the use of a detergent and or disinfectant solution the mattress cover should be rinsed with clean water using a clean cloth and allowed to dry.
- Frequent or prolonged exposure to high concentrations of aggressive disinfectant solutions will reduce the useful life of the mattress cover.
- Where high concentration disinfectants e.g.  $\geq 10,000$ ppm chlorine releasing agent (e.g. Haztab or bleach) or combined cleaning/chlorine releasing agent (e.g. Chlorcleam, Actichlor) and detergent solutions are used to remove blood or other body fluids, the mattress should be thoroughly rinsed with clean water to remove any residues. This will help prevent any possible long term compatibility issues associated with disinfectant residues.
- Alternatively disinfection may be achieved by laundering at temperatures not exceeding 65°C for 10 minutes or 73°C for 3 minutes which may include a chlorine rinse.
- Do not use abrasive cleaners, phenol disinfectants, solvents or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these may destroy the cover materials.
- Do not iron.
- Ensure that the mattress is thoroughly dried before remaking the bed or placing in storage.

## INTERIOR COMPONENTS

- Check air cells and mattress interior for signs of damage or contamination, e.g. staining or evidence of fluid ingress. The frequency of these checks should be at each decontamination process, i.e. between patients or patient occupancy (or weekly for longer term patients)
- Care should be taken to avoid puncturing air cells with objects such as needles, scalpels, pat slides, acrylic nails, etc.
- The individual cells can also be wiped clean with a mild antiseptic solution.
- All cells are replaceable and can be obtained easily from Talley.
- Do not immerse the foam side bolsters and base in water.
- With the exception of the foam side bolsters and base, the QUATTRO® SENTINEL mattress can be laundered, but ensure the CPR device is fully closed (i.e. the dial of the CPR device is rotated clockwise until it 'clicks' into the closed position ) , prior to laundering to prevent water entering the air cells/tubing.

## POWER UNIT

Always disconnect the power unit from the electricity supply before carrying out maintenance, repairs, servicing or cleaning. Check all electrical connections and power lead for signs of excessive wear. The power unit can be wiped down with detergent or disinfectant solution or wipe\*. Do not use solvents. Unsuitable for sterilisation. Disposal of the pump / mattress / cushion in accordance with the local regulations including WEEE requirements.



\* In line with the MHRA Medical Device Alert (MDA/2013/019), Talley advises customers to use pH neutral, high level disinfectant cleaning products to sanitise reusable medical devices to prevent damage to materials and the degradation of plastic surfaces after prolonged use. The use of inappropriate cleaning and detergent materials on medical equipment could damage surfaces and may compromise the ability to decontaminate medical devices adequately or may interfere with device function. Talley recommends the use of TECcare® CONTROL antimicrobial wipes and fluid to clean and decontaminate all products it supplies to health and social care facilities. TECcare® CONTROL products provide class leading broad spectrum, high level disinfection with an exceptional safety profile. Being pH neutral TECcare® CONTROL can be universally used on all hard and soft surfaces without any detrimental effect. TECcare® CONTROL is CE marked for cleaning medical equipment.

## SERVICING

Once the initial guarantee period expires, Talley recommend that all power units should be serviced annually or as indicated by the 'hours to service' display. The unit contains no user serviceable parts and should only be serviced by either Talley or an authorised dealer. Talley will make available on request service manuals, component parts lists and other information necessary for Talley, an authorised dealer or a competent electrical engineer to repair or service the system. Talley's standard terms and conditions apply to all sales. A copy is available on request. For service, maintenance and any questions regarding this, or any other product, please contact Talley.

It is the customer's responsibility to ensure the following prior to collection:

- the system is cleaned of any obvious contaminants.
- contamination status is documented.
- assistance is given to Talley personnel to bag the equipment if the mattress has been in a known or suspected infectious environment.

## TRANSPORT AND STORAGE

Handle with care. Please report instances of damage or impact to Talley Service Department.  
Transport

–25 °C without relative humidity control; and

+70 °C at a relative humidity up to 93 %, non-condensing.

An atmospheric pressure range of 700 hPa to 1 060 hPa.

Suitable for all standard modes of transport when in the correct packaging.

## OPERATIONAL CONDITIONS

A temperature range of +5 °C to +40 °C;

A relative humidity range of 15% to 93%, non-condensing; and

Operational Atmospheric Pressure: 700 hPa to 1060 hPa

Suitable for pollution degree 2

Operational altitude ≤ 2 000 m

IP Rating: IP21 pump only



## TRANSPORTATION OF MATTRESS SYSTEM

Remove the side bolsters and foam base, then loosely roll the mattress lengthwise with the cover innermost, taking care not to strain the umbilical. The mattress can then be transported and stored in a carry bag with the power unit, mains cable and this booklet. Do not stack bagged mattresses more than two high to avoid strain on the umbilical.

## MANUFACTURER'S GUARANTEE


All power units, alternating mattresses/cushions and foam mattresses/cushions are covered by a 24 month manufacturer's guarantee. The products have no specified shelf life. The intended design life is 5 years if fully serviced.

# Fault Finding

The power unit can be reset by pressing the MUTE button once. This also silences the sounder and clears the message from the display screen. All systems have a fault log that records the last 5 faults via the DATA display mode. If problems reoccur contact Talley.

**AC FAIL fault** – indicates a mains power failure, a sounder will be heard if power is interrupted, e.g. power unit switched off, power cut, disconnection of mains lead. Press MUTE or re-connect to power supply.

**ROTOR SYSTEM fault** – indicates the automatic sequential cycle has stopped or there is a fault in the system. Switch power off, press MUTE button, then switch power on again. If the fault re-occurs, contact Talley.

**LOW PRESSURE fault** – indicates pressure has fallen below the minimum allowable levels. Check that the hose is connected to the power unit correctly. Check that the CPR device is fully closed, i.e. the dial of the CPR device is rotated clockwise until it 'clicks' into the closed position . Check that the internal cells are connected and that no cell is punctured. Press the MUTE button to clear the message and to silence the sounder. If the fault re-occurs, contact Talley.


**EMI fault** – indicates that the unit detects the pressure sensor amplifier is adversely affected by external RF fields. This will clear when interference ceases.

## OTHER FAULTS

**PUMP OR TRIAC fault** - indicates a pump control failure or an open pump coil fault. Should this occur, contact Talley.

**UNCALIBRATED** - contact Talley for recalibration.

If you have any queries relating to this system please contact Talley or your local authorised dealer.

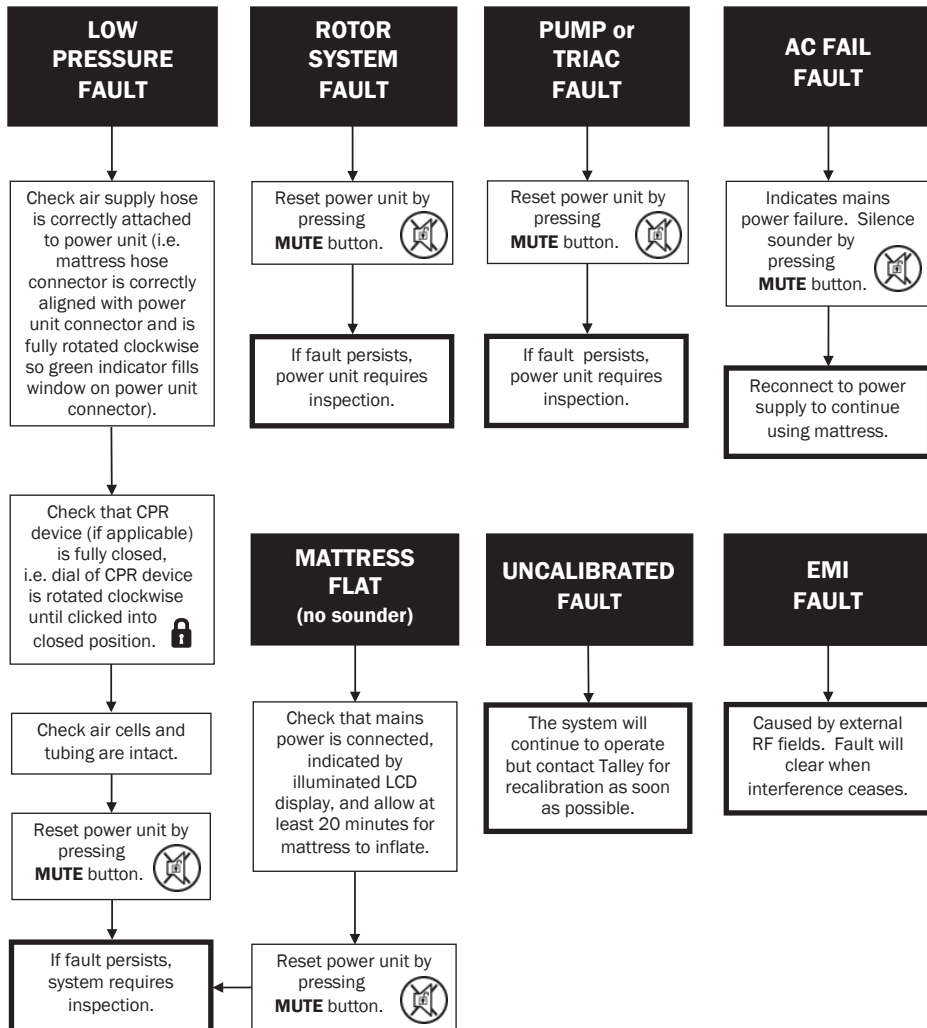



# Specifications

## QUATTRO SENTINEL ALTERNATING AIR PRESSURE POWER UNIT

(Medical Device Classification: Class IIa)



Model Ref.:	Type 19
Construction:	ABS Plastic
Dimensions:	350mm/13.8" x 233mm/9.2" x 165mm/6.5"
Weight:	3.6 kg / 7.9 lbs
Mains Cable:	5 metres / 16.5'
	For the USA:- only a hospital grade attachment plug with a 15A NEMAP 5-15 configuration and 18AWG hospital grade flexible cord is to be used, as supplied by Talley.
Electricity Supply:	230V ~ 50Hz (CE marked) 120V ~ 60Hz (cETLus listed) 230V ~ 60Hz
Rated Input:	9.4 VA
Fuse Rating:	T500mA 250V HRC (ceramic) 5 x 20 mm
IP Rating:	IP21
Cycle Time:	16 minutes (continuous)

## QUATTRO SENTINEL BARIATRIC ALTERNATING AIR PRESSURE MATTRESS

(Medical Device Classification: Class IIa)



Construction:	<b>BASE TRAY:</b> Woven nylon 940 DTEX PU coated both sides <b>BASE LAYER AND SIDE BOLSTERS:</b> CMHR foam <b>CELLS:</b> Reinforced PU coated Nylon <b>COVER:</b> PU coated stretch nylon 255g/m <sup>2</sup>
Options:	Standard
Type:	Orthodifferential air cells over a CMHR foam base operating in a 1-in-4 alternating air pressure cycle.
Dimensions*:	L2000mm x W1200mm (860mm without side bolsters) x H180mm
Weight:	23.4kg

### MANUFACTURER'S GUARANTEE

The system is covered by a 24 month manufacturer's guarantee.

\*UK standard size - approx.max. top surface, inflated +/- 15mm



# EMI/EMC Statement and Manufacturer's Declaration

This equipment has been tested and found to comply with the limits of EN 60601-1-2.

These limits are designed to provide reasonable protection against harmful interference in both a medical and residential environment. This equipment generates, uses and can radiate radio frequency energy and, if not used in accordance with manufacturer's instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception or other equipment, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment.
- Connect the equipment to an outlet on a circuit different from that to which the receiver or equipment was connected.

The equipment having been tested to operate within the limits of electromagnetic compatibility. (Immunity to interference from nearby sources radiating radio frequency energy). Sources exceeding these limits may give rise to operation faults. Where possible the system will sense the interference and if it is of short duration transparently take countermeasures whilst operating near normally, or failing this will issue a warning and take measures for the continued safety of the user. Further increased levels of energy may cause the system to stop operating, continuously generate random faults or continuous resets.

Try to ascertain the source of the interference by turning nearby or suspect equipment off, and see if the interference effects stop. In any such event the user is encouraged to try to correct the interference by one of the following measures:

- Have the interfering equipment repaired or replaced.
- Reorient or relocate the interfering equipment.
- Increase the separation between the equipment and the possible source of the interference.
- Connect the equipment to an outlet on a circuit different from that to which the interfering equipment was connected.

## Information regarding Electro Magnetic Compatibility (EMC) according to IEC60601-1-2

With the increased number of electronic devices such as PC's and mobile telephones, medical devices in use may be susceptible to electromagnetic interference from other devices. The EMC (Electro Magnetic Compatibility) standard IEC60601-1-2 defines the levels of immunity to these electromagnetic interferences. From the other hand, medical devices must not interfere with other devices. IEC60601-1-2 also defines the maximum levels of emissions for these medical devices. The QUATTRO® conforms to this IEC60601-1-2 standard for immunity and emission. Nevertheless, special precautions need to be observed:


- The QUATTRO® needs to be installed and put into service according to the EMC information below.
- The QUATTRO® is intended for use in the electromagnetic environment specified in the tables below. The user of the QUATTRO® should assure that it is used in such environment.
- In general, although the QUATTRO® complies too the EMC standards, it can be affected by portable and mobile RF communications equipment (such as mobile telephones).
- The QUATTRO® should not be used adjacent to or stacked with other equipment. In case adjacent or stacked use is necessary, the QUATTRO® should be observed to verify normal operation.

Guidance and Manufacturer's Declaration: Electromagnetic Emissions (IEC 60601-1-2)		
Emissions Test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Class B	The QUATTRO® systems are suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage pump supply network that supplies buildings used for domestic purposes.
Harmonics emissions 61000-3-2	Class A	
Voltage fluctuations / flicker emissions 61000-3-3	Complies	



Guidance and Manufacturer's Declaration: Electromagnetic Immunity (IEC 60601-1-2)			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8kV contact ± 15kV air	± 8kV contact ± 15kV air	The relative humidity should be at least 5%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV For mains supply lines 100kHz repetition frequency	± 2 kV For mains supply lines 100 kHz repetition frequency	Mains supply quality for the mains adapter should be that of a typical commercial and/or hospital environment.
Surge IEC61000-4-5	± 2kV Line(s) to ground ± 1kV line(s) to line	± 2kV Line(s) to ground ± 1kV line(s) to line	Mains supply quality for the mains adapter should be that of a typical commercial and/or hospital environment.
Voltage dips, short interruptions and voltage variations on mains supply IEC 61000-4-11	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle 70 % UT; 25/30 cycles Single phase: at 0°	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle 70 % UT; 25/30 cycles Single phase: at 0°	Mains power quality should be that of a typical commercial and/or hospital environment. If the user of the QUATTRO® requires continued operation during power mains interruption, it is recommended that the QUATTRO® be powered from an uninterruptible power supply or battery.
Voltage interruptions	0 % UT; 250/300 cycle	0 % UT; 250/300 cycle	
Mains frequency (50/60Hz) magnetic field IEC61000-4-8	30 A/m	30 A/m	Mains frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial and/or hospital environment.

Note: Ur is the A.C. mains voltage prior to application of the test level.

Guidance and Manufacturer's Declaration: Electromagnetic Immunity (IEC 60601-1-2)			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Conducted RF IEC 61000-4-6	3 V rms 150 kHz ~ 80 MHz  6 V rms 150 kHz to 80 MHz in ISM and amateur radio bands between 0,15 MHz and 80 MHz	3 V rms  6 V rms	Portable and mobile RF communications equipment should be used no closer to any part of the QUATTRO® including cables, than the recommended separation distance calculated from the equation appropriate to the frequency of the transmitter. <b>Recommend separation distance</b> $d = 1.2 \sqrt{P}$ $d = 0.35 \sqrt{P}$ 80 MHz to 800 MHz $d = 0.7 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).
Radiated RF IEC 61000-4-3	10 V/m 80 MHz ~ 2.7 GHz	10 V/m	Field strengths from fixed RF transmitters as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: 
Note1: At 80 MHz and 800 MHz, the higher frequency range applies. Note2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people. <sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the QUATTRO® is used exceeds the applicable RF compliance level above, the QUATTRO® should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the QUATTRO®. <sup>b</sup> Over the frequency range 150 kHz to 80MHz, field strengths should be less than 3 V/m.			

Products are free from TSE species derived materials, medicinal substances, human blood derivatives and phthalates.

Talley manufacture products to comply with National and International safety standards and are certified to ISO13485 and Directive 93/42/EEC.

Every care has been taken to ensure that the information contained in this manual was correct at the time of going to press. However, Talley reserves the right to modify the specification of any product without prior notice in line with a policy of continual product development.

Our standard terms and conditions apply.

This medical device is compliant with:

IEC 60601.1 3rd edition Medical electrical equipment safety and essential performance

IEC 60601.1.11 Home healthcare environment

## USER MANUAL PART NUMBER 10550/8



**Talley Group Limited**  
Premier Way, Abbey Park Industrial Estate, Romsey, Hampshire, SO51 9DQ England  
TEL: +44(0)1794 503500 FAX: +44(0)1794 503555  
E-MAIL: sales@talleygroup.com  
**www.talleygroup.com**



03/2020