

cura¹

2701 - Standard Falls Monitor with SafeTime



Compatible & Optional Accessories

All Premium, Safetime and Universal pads and mats. (Universal when using USB to RJ10 break cord).

3504 ActiveCare Bracket, 2406 Plug Pack 6V Power Adaptor, 2642 Caregiver Reset Key, 2636 Easy Release Seatbelt, 2742 Soft Monitor Cover, 2699 Fixing Strap.

Nurse Call Cables: 2643, 2646, 2460, 2461, 2493

cura¹


10 Gray Street, Kilkenny South Australia 5009
www.cura1.com

1300-12-CARE (2273)

UM-035e



All our products are supplied with a full warranty and are guaranteed to be free from defects in material and workmanship.

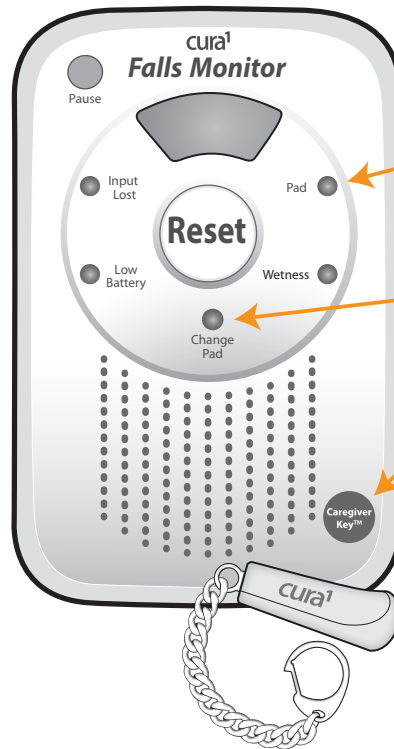


Pad light will blink when pad is in use.

Change Pad light will illuminate & Monitor will beep when SafeTime Bed or Chair Pad expires.

Caregiver Reset Key application point.

Caregiver Reset Key (Resets the Monitor after an alarm.)

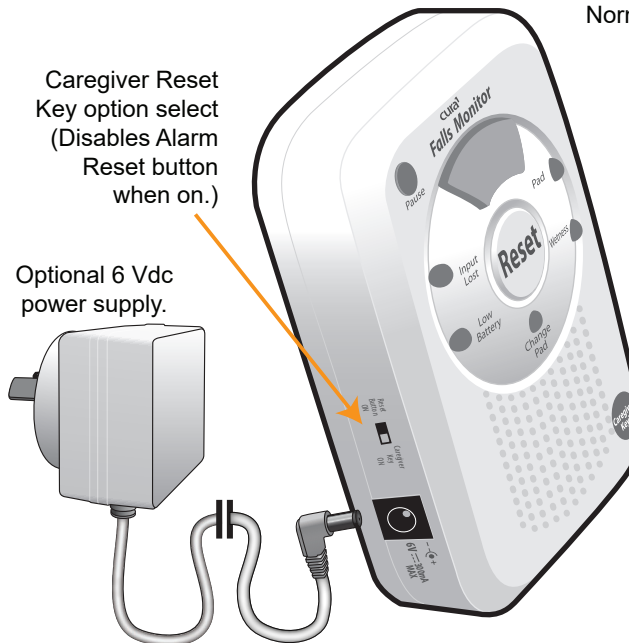


Alarm Reset Button




Monitor auto-mutes when connected to a nurse call system and volume is set to minimum.

Optional Nurse Call Cable.

NO / NC (Normally Open / Normally Closed) selector for Nurse Call Systems.

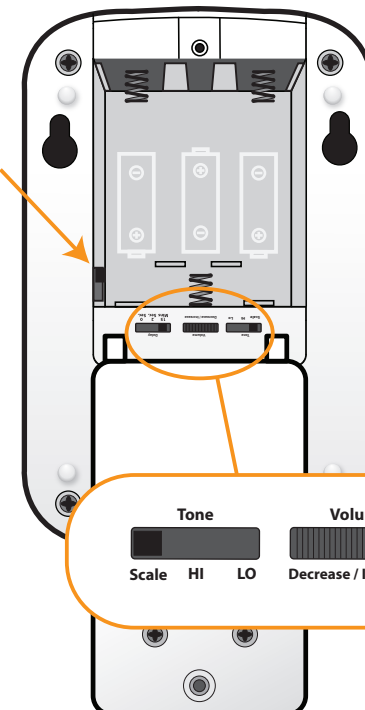

Adjust alarm tone, volume & delay settings.

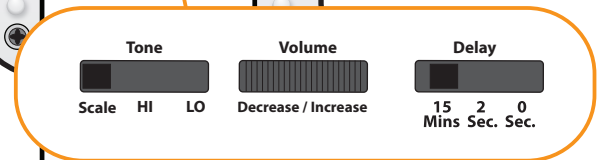


QUICK START GUIDE

2701 - Standard Falls Monitor with Safetime

Step 1

Insert 3 x AA alkaline batteries. (Do not use rechargeable batteries.)

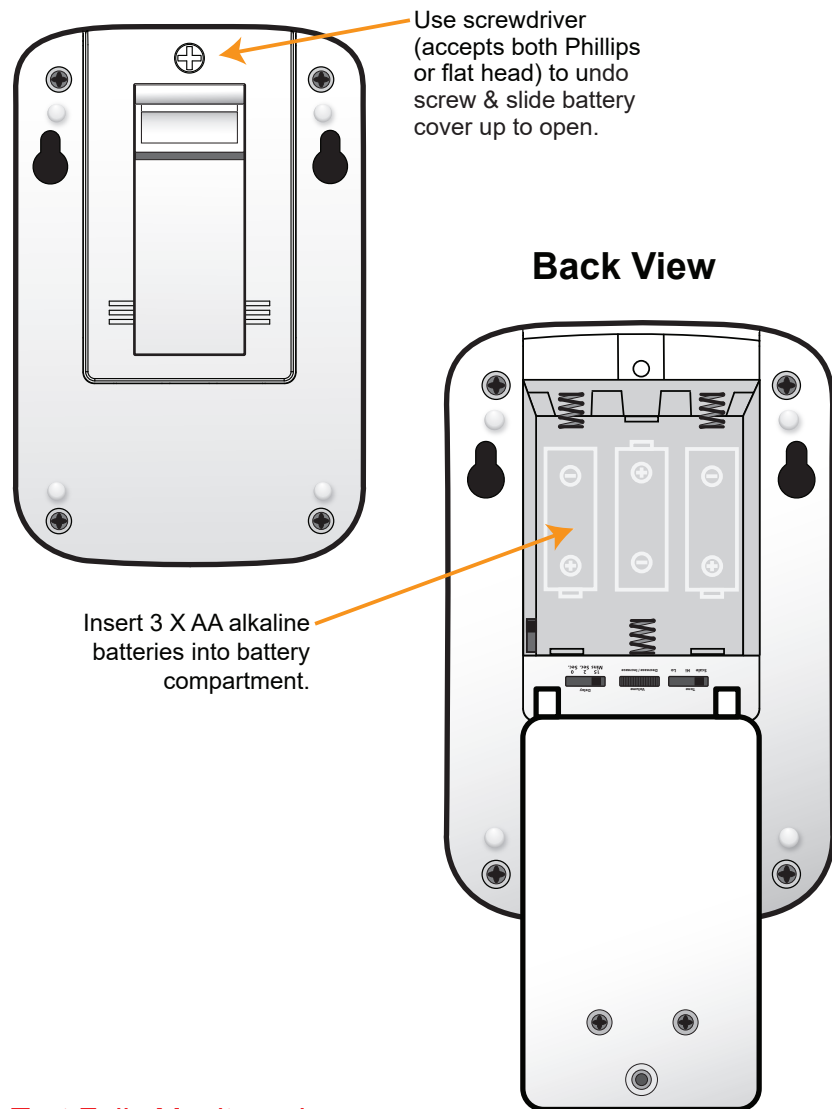


Test Falls Monitor prior to every use.

Step 2

Connect Pad or Floor Mat to Monitor & test prior to using.

Bottom View

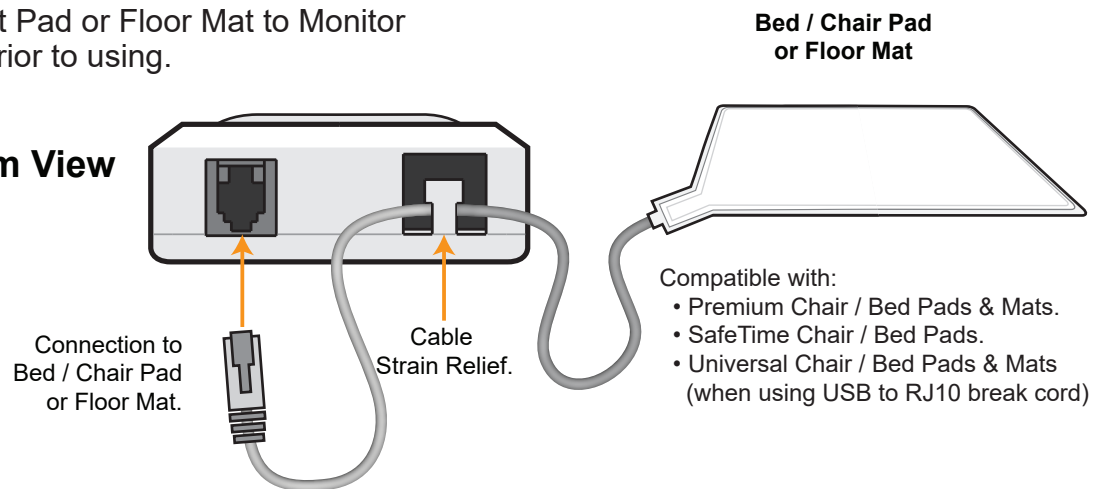

Step 3

Place bed pad on mattress or position mat on floor.

