

Reynard Shampoo cap.

A complete rinse-free hair care solution.

Removes dirt, EEG gel and blood while reducing the need for patient movement and risk of cradle cap. The cap is impregnated with a gentle conditioning shampoo designed to release when massaged into the scalp. Removing the need for water, bowls and other products whilst saving on valuable nursing time. Each shampoo cap is for single patient use and will cleanse the head and hair.

Preparation: Heating: heat the unopened pack in a microwave for 10-20 seconds. Alternatively, place the sealed pack in hot water for 3-5 minutes. Always check the temperature before use.

Chilling: Place sealed pack in refrigerator for 10 minutes.

Other: Use at room temperature.

Step 1: Place cap on the patient's head.



Step 2: Ensure all hair is covered.



Step 3: Massage the head all over for 2-3 minutes to ensure it is completely saturated.



Step 4: For patients with longer hair, remove cap and work through the ends to ensure all hair is properly washed.



Step 5: Towel dry hair if required.



Step 6: Dispose of the cap and packaging. Do not flush or put into a macerator.



Please note: we recommend following the hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

| | |
|-----------------|--------|
| Product code | RHS104 |
| Caps per pack | 1 |
| Caps per carton | 24 |



Reynard is reassurance.

