



ROYALE


Royale Over Chair & Bed Swivel Table
User Manual

Contents

SECTION	TITLE	PAGE
1	Introduction	2
2	Safety & Precautions	3
3	Features	4
4	Specifications	4
5	Instructions for Use	5
6	Labelling Information	11
7	Product Care & Maintenance	12
8	Recycling & Disposal	13
9	Warranty	13

No part of this user manual may be copied, distributed, reproduced, duplicated, scanned or stored whether in whole or part in any form or by any means without the expressed written consent from the officers of Independent Living Specialists Pty Ltd. Independent Living Specialists does not accept responsibility for any changes, errors or omissions. All information is subject to change without prior notice and products may vary from that of the actual images depicted.

1 Introduction

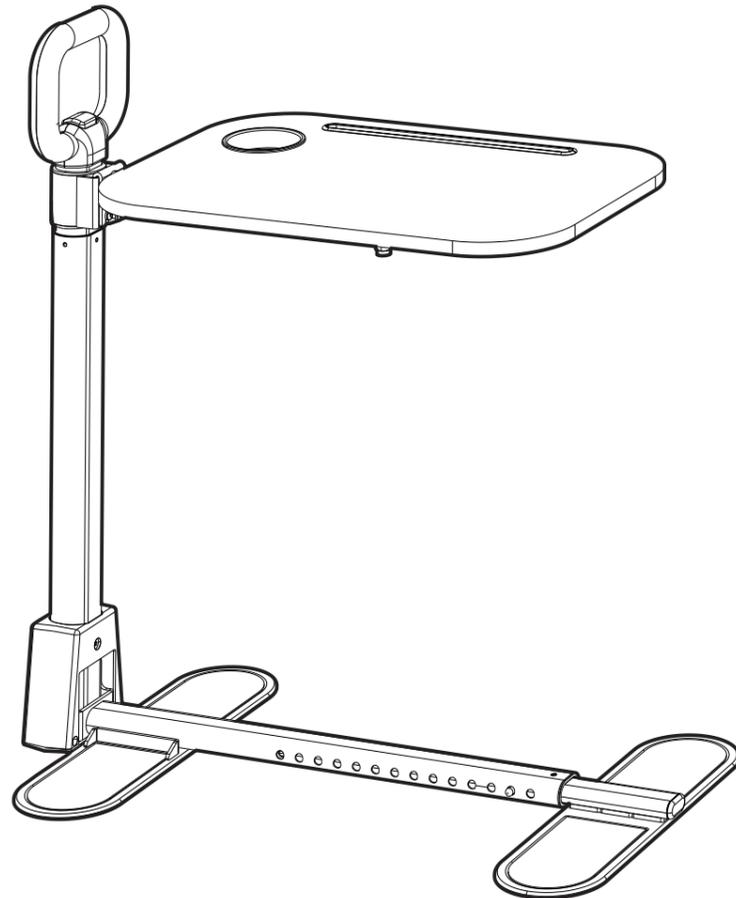

S339491 - Over Bed/Chair Table- Swivel Table with Adjustable Support Handle
- Black Ebony Royale Medical

S339492 - Over Bed/Chair Table- Swivel Table with Adjustable Support Handle
- Brown Mahogany Royale Medical

2 Safety & Precautions

- Please make sure to read and fully understand the user manual before installation. If you do not understand any of the installation steps, please contact your authorised distributor for clarification.
- The tabletop is designed to rotate 360°. For safe and stable operation, ensure that heavy loads are placed centrally on the tabletop and not on the sides or edges.
- The tabletop arm is designed to rotate 360°. It is recommended to put any load on the tabletop only when the tabletop is positioned above or adjacent to the base. For safe and stable use in any position, ensure that the base feet of the table is positioned underneath the chair base to secure the table and avoid risks of tipping.
- Ensure the base is positioned securely under a stable, chair or sofa. The furniture must have legs or a gap that allows the base to be positioned underneath.
- Adjust the base length to provide maximum contact with the furniture for stability.
- Ensure the rubber pads on the base are in good condition to maximise grip and prevent slipping or floor damage.
- Avoid leaning heavily on the swivel tabletop - it is designed as a utility surface, not a weight - bearing support.
- Keep hot items, liquids, and electronics secure when using the tabletop, particularly when rotating and adjusting.
- Always check the swivel function is clear of obstructions before rotating the tray.
- This product contains small parts that could pose a choking hazard if swallowed. Please keep these small accessories out of reach of children and store them in a safe place.
- Regularly inspect all components, including the handle, tabletop, swivel mechanisms, and base.
- Wipe clean with a damp cloth and mild detergent. Do not use abrasive cleaners or immerse in water.
- If any part is loose or damaged, please contact your authorised distributor for support or replacement parts.


3 Features

- **Swivel Tabletop** - generous sized tabletop with 360° rotation and swivel function on axis.
- **Adjustable Base Design** - base extension to suit the side of most couches or chairs.
- **Hydraulic Height Adjustment** - simple push button to adjust the height of the tabletop.
- **Handle Support** - ergonomic support handle to provide additional balance during sit-to-stand transfers.

4 Specifications

Specifications	
Maximum Support Handle load	130 kg
Maximum tabletop Load	9 kg
Tabletop Pivot	360°
Tabletop Rotation	360°
Tabletop Size	60 x 40 cm
Height Adjustment (Tabletop)	70 - 104.5 cm
Height Adjustment (Support Handle)	85 - 119.5 cm
Base Feet Adjustment (Inner Length)	49 - 81 cm

Instructions for Use 5

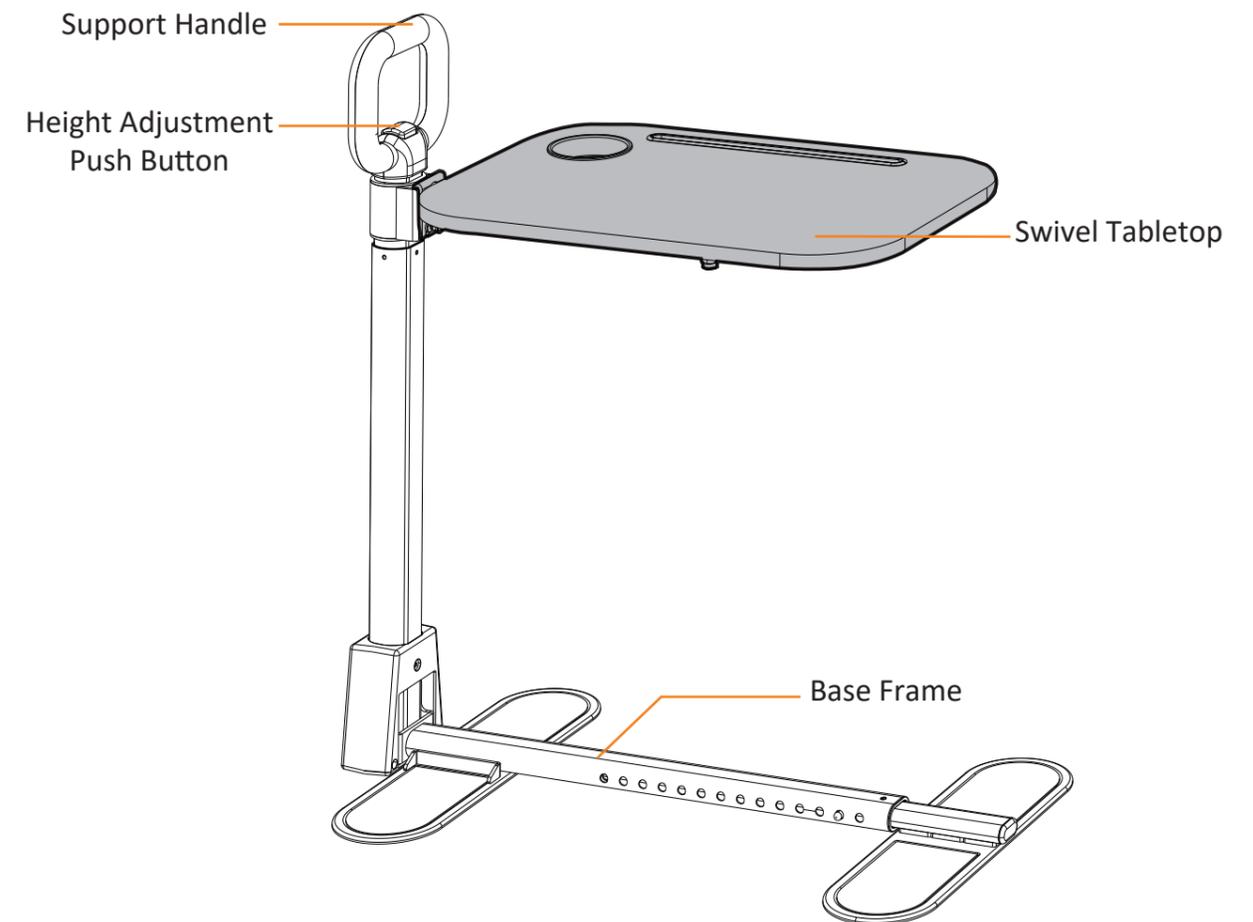


Intended Use


- This product is designed to assist with balance and support during sit-to-stand movements. It is not intended to bear full body weight. This product comes with a 360° swivel tabletop to provide a versatile surface for various items.
- Always use the ergonomic safety handle in a controlled and stable manner—avoid sudden movements or pulling heavily on the handle.
- Adjust the height of both the support handle and swivel tray to suit the user’s needs. Double-check that all locking mechanisms are secure before use.
- Do not use the product if any parts appear damaged, loose, or worn.

5.1 Assembling the Over Chair Swivel Table


Lie the base frame on a flat surface and lift the support handle to a 90° upright position.

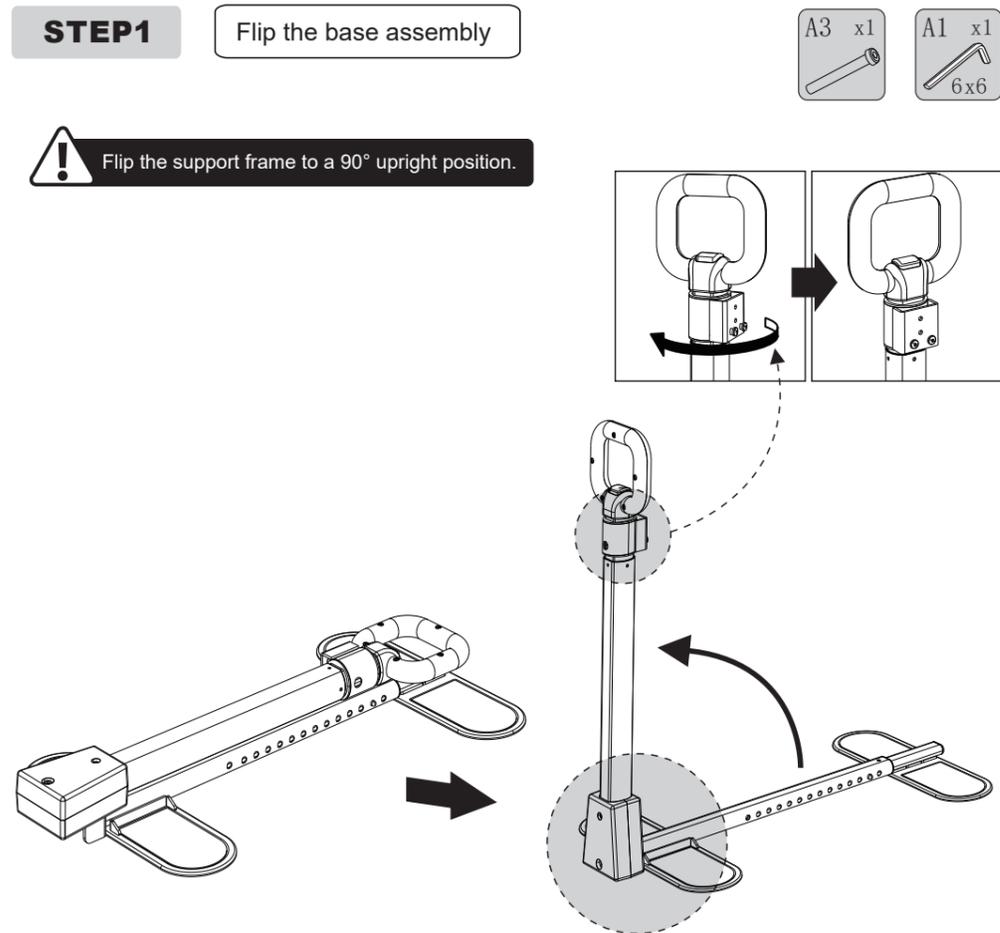


Secure the support handle by screwing in the bottom screw.

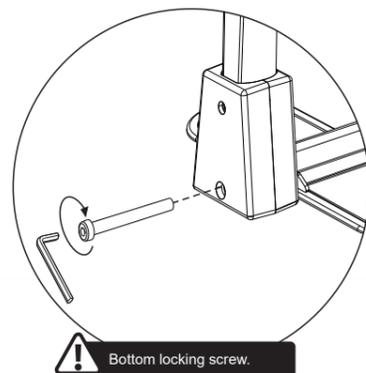


Position the swivel arm assembly into the support handle.

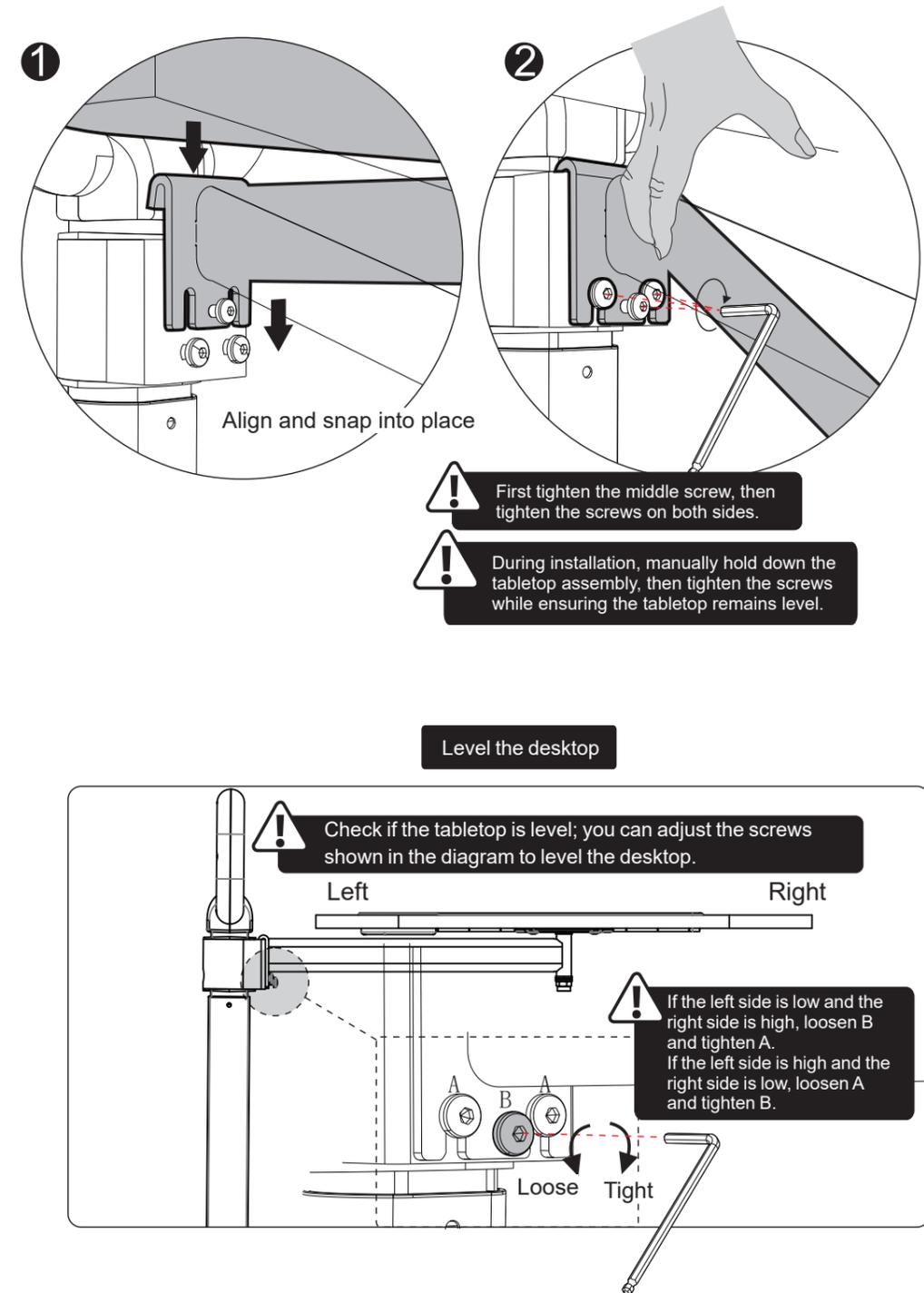


Adjust the tightness of the swivel table.

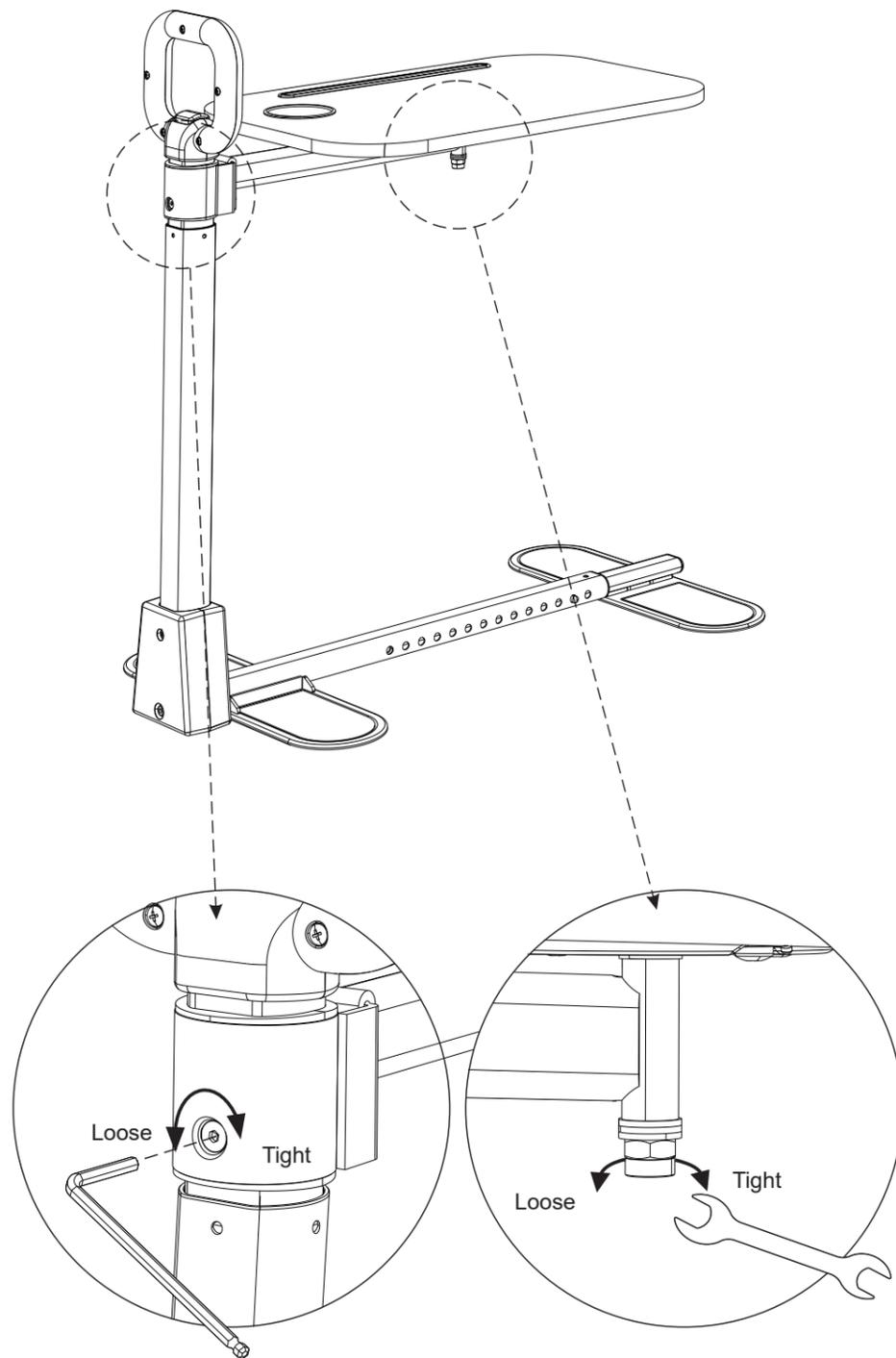

Adjusting the base frame extension.

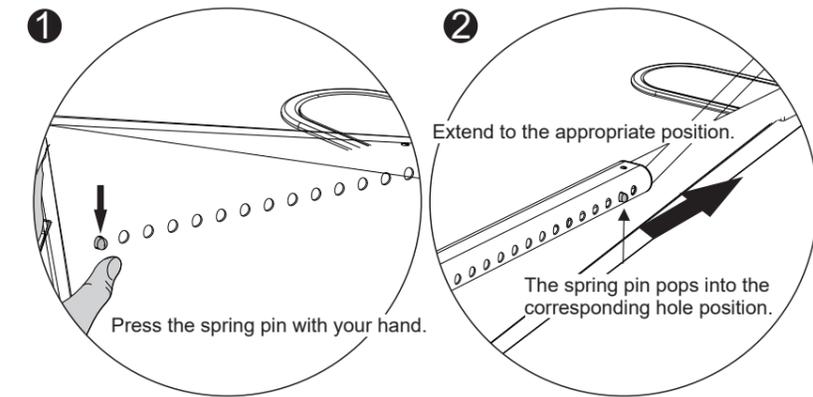


Positioning underneath furniture. Place the furniture legs evenly onto the rubber pads of the base frame.

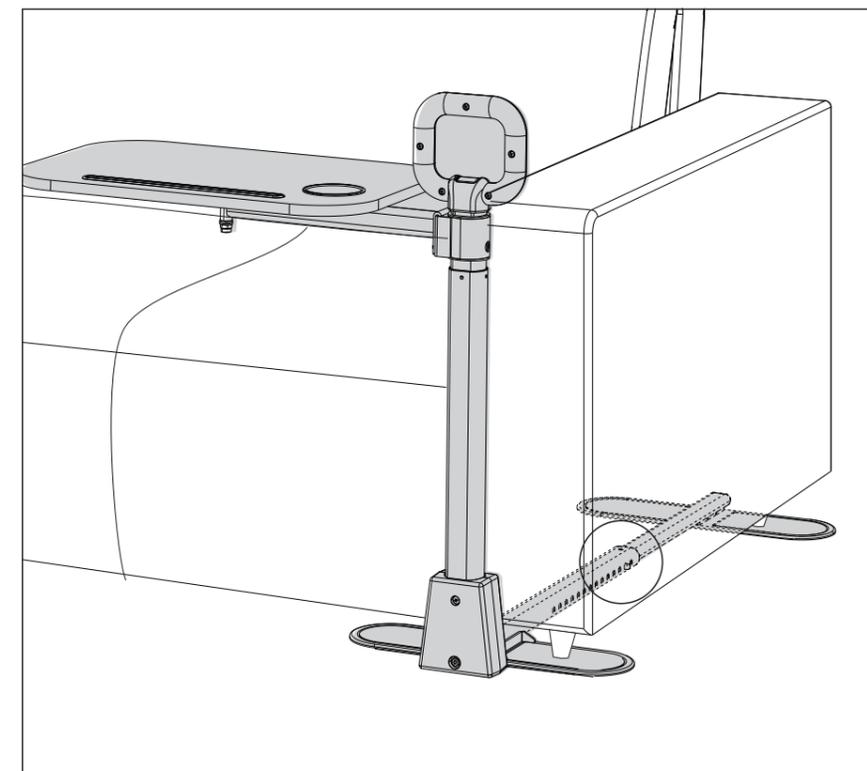



5.2 Adjusting the height of the Over Chair Swivel Table

To adjust the height of the tabletop and handle support, simply press the hydraulic push button located on the support handle.

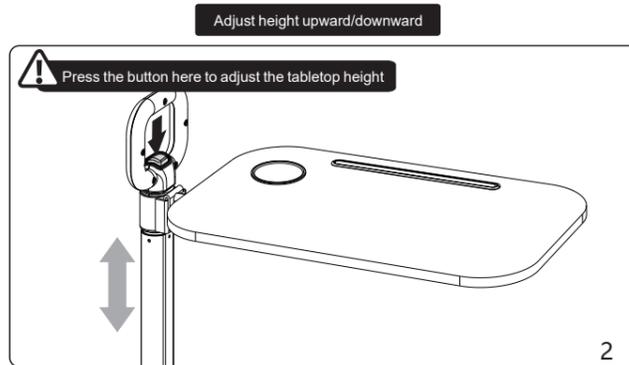

5.3 Adjusting the position of the tabletop

The tabletop can rotate a full 360° and can be swiveled in the one position.

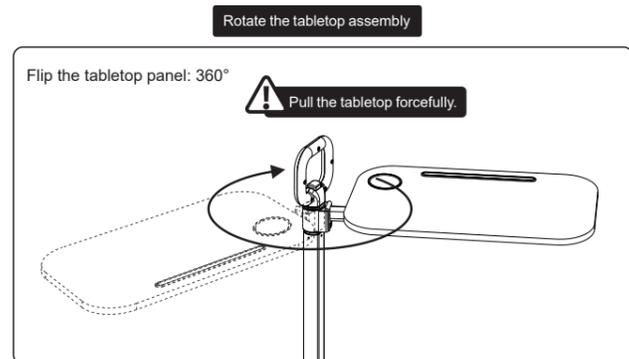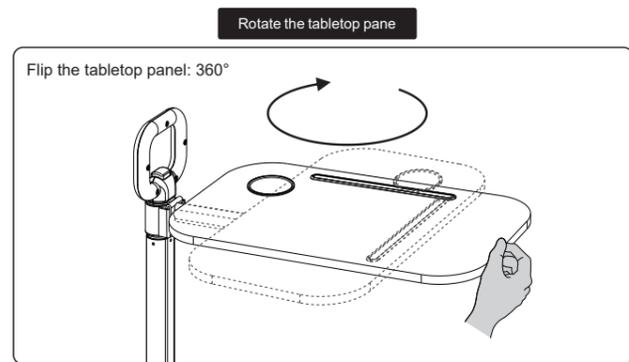


Labelling Information

6

The Over Chair Swivel Table has a product label on the base frame.

The product label includes the following information:

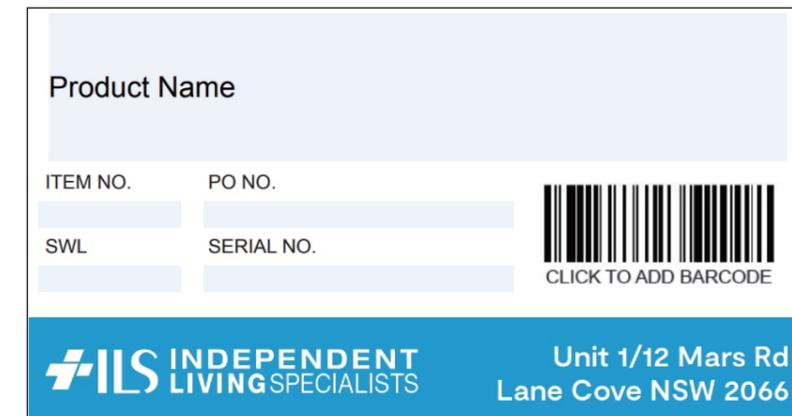

- Name and address of the company
- Name and code for the product
- Maximum support handle load and maximum tabletop load (SWL)
- Serial number of the product (which includes the date of manufacture and the batch serial number)

7 Product Care & Maintenance

- It is recommended that the Over Chair Swivel Table is regularly checked for signs of wear and tear—particularly on key components such as the base frame, height adjustment mechanism, swivel tabletop mechanism and table surface.
- Before each use, ensure the height adjustment column and any locking brackets are secure and functioning correctly.
- If the table has been exposed to moisture or spills, wipe it dry promptly to prevent corrosion or damage to moving parts.
- The table should be cleaned regularly using warm water or a mild detergent. Only use soft cleaning tools such as sponges, cloths, or soft-bristled brushes. Avoid using abrasive cleaners, scouring pads, or harsh chemicals, as these can damage the surface or frame.
- Inspect the height adjustment and the tabletop swivel mechanism to ensure they are operating safely and without signs of damage.
- Make sure all fixings and components remain tight and secure during regular use.

For replacement parts, after-sales support, or service enquiries, please contact your authorised **Royale Medical dealer**.

Recycling & Disposal

Please follow the local government rules regarding proper recycling and disposal. It should always be done through an authorised agent or licensed place of disposal. Please review the list of seat walker parts and packaging materials to ensure they are recycled or disposed of correctly.

- Frame: Steel
- Tabletop: Wood
- Packaging: Cardboard box and plastic

Warranty

The Royale Medical Royale Over Chair Swivel Table comes with 12 months warranty from the date of purchase on structural integrity.

- During the warranty period any parts that have become defective due to faulty workmanship or material will be repaired or replaced without charge by a Royale Medical dealer of where the product was purchased.
- The warranty is for normal domestic use only and excludes any defect or injury caused by or resulting from misuse, abuse, neglect, accidental damage, or any other alteration that affects the reliability or performance of the unit, not attributable to faulty manufacture, parts or labour.
- The warranty excludes components and parts that are subject to reasonable wear and tear.
- The warranty excludes faults that have been identified as misuse or user error.
- Unauthorized changes or modifications will forfeit your warranty.
- If a defect or fault is discovered, please contact the Royale Medical dealer of where the patient transfer device was purchased.



Independent Living Specialists

Unit 1, 12 Mars Road

Lane Cove West NSW 2066

1300 008 267 | ilsau.com.au

©2023 ROYALE MEDICAL. INDEPENDENT LIVING SPECIALISTS

PDT037-0424